

# Viking M

## Service Manual


English

Prod. No. 2040005, 2040015, 2040035


3EN370401-16  
2012-12-27

Proj. 1707


Prod. No. 2040005  
S/N 7 100 101-7 200 200

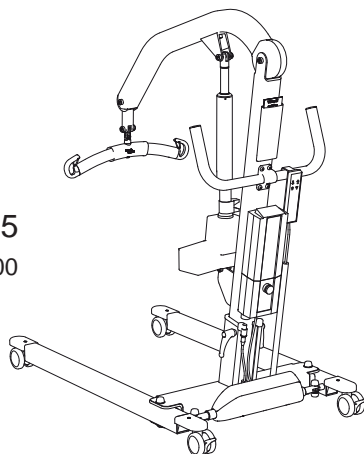

Prod. No. 2040015  
S/N 7 500 401 - 7 600 500

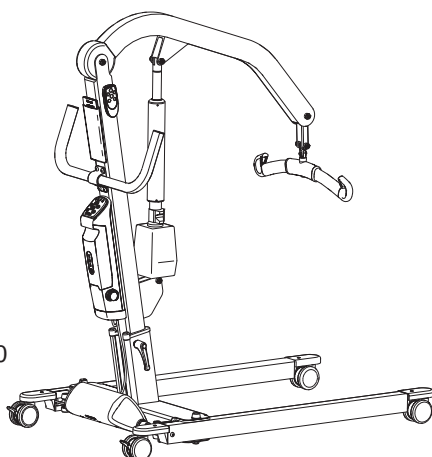

Prod. No. 2040035  
S/N 9 200 001 - 9 300 000

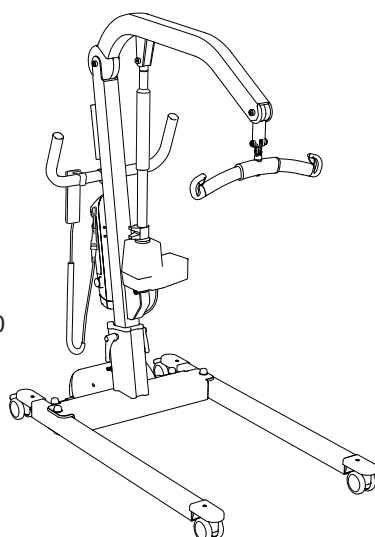


Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko  
and using original Liko spare parts.  
For information regarding Service and Maintenance education, contact your Hill-Rom/Liko representative.  
For contact information please visit our website: **[www.liko.com](http://www.liko.com)**

	Page
1.0 Table of contents .....	3
2.0 Spare Parts Positions.....	4
3.0 Technical Data	
3.1 Viking M 2040005 .....	5
3.2 Viking M 2040015 .....	6
3.3 Viking M 2040035 .....	7
4.0 Troubleshooting	
4.1 Viking M 2040005 .....	8
4.2 Viking M 2040015 .....	9
4.3 Viking M 2040035 .....	10
4.4 Control Unit CBL 20 .....	11
5.0 Spare Parts List.....	12
5.19 Attachment for Control Box .....	17
5.20 Hand Control 6 button .....	18
5.22-24, 248 Cable for Charging .....	19
5.29 Front Wheel 75 mm.....	20
5.30 Rear Wheel 100 mm .....	21
5.55 Hand Control EE 4-button .....	22
5.139 Extra Battery Box .....	23
5.159 Front Fork Cover .....	24
5.160 Extension Cable .....	25
5.194 Rear Fork Cover.....	26
5.196 Lift Arm Plastic Cover .....	27
5.197 Overhead Beam Restoration set .....	28
5.199 Locking Handle .....	29
5.263 Extension Cable US .....	30
5.264 Latch Universal Slingbar .....	31
5.266 Base Motor, complete.....	32
5.267 Actuator Viking M .....	34
5.268 Control Box Viking M.....	35
5.284 Front Wheel 75 mm Standard .....	36
5.285 Rear Wheel 75 mm Standard.....	37
5.292 Hook for hand control .....	38
5.307 Lift Arm Plastic Cover .....	39
5.308 Overhead Beam Restoration set.....	40
5.311 Latch, composite .....	42
5.328 Restoration set motor attachment .....	43
5.331 Outer Tube .....	45
5.333 Wireless hand control HBL20.....	47
5.334 Control Unit CBL, Vi M .....	48
5.334C Control Unit CBL, Vi M Cable .....	50
5.336 Control Unit CBL, by S/N.....	52
5.336C Control Unit CBL, by S/N.....	54
5.347 Actuator Vi M .....	56
5.348-349 Extension cable CBL20 .....	57
5.350 Extra battery box .....	58
5.351 Beltclip HBL20.....	59
5.352 Clips with hook HBL20 .....	60
5.353 Charge cable hook .....	61
5.354 Hand control hanger.....	62
5.410 Viking M Legs.....	63
5.418 Mast Viking M.....	65
5.422 Locking device CBL20.....	66
5.439 Bracket Cover.....	67
5.460 Leg Cover M.....	68
5.461 Flexlink .....	70
5.485 Fork locking attachment .....	71
5.486 Wheel Fork.....	72
5.490 SlingBar Assembly set.....	73
5.511 Locking Ring Hand Control CBL .....	74
6.0 Service,Control Unit CBL	
6.1 Parameter Code Key.....	75
6.2 Reset Service Interval Parameters .....	76
7.0 Liko Service System, LSS.....	77
8.0 Liko Diagnostic System - Service, LDS.....	78
9.0 Periodic Inspection.....	79
10.0 Preventative Maintenance.....	85
11.0 Product Changes .....	93


## Spare Parts Positions

PROD. NO.	PRODUCT
2040005	Viking M
2040015	Viking M (intelligent control box)
2040035	Viking M (intelligent control box and cable hand control)

**Product No. 2040005** (S/N 7100101-7200200)

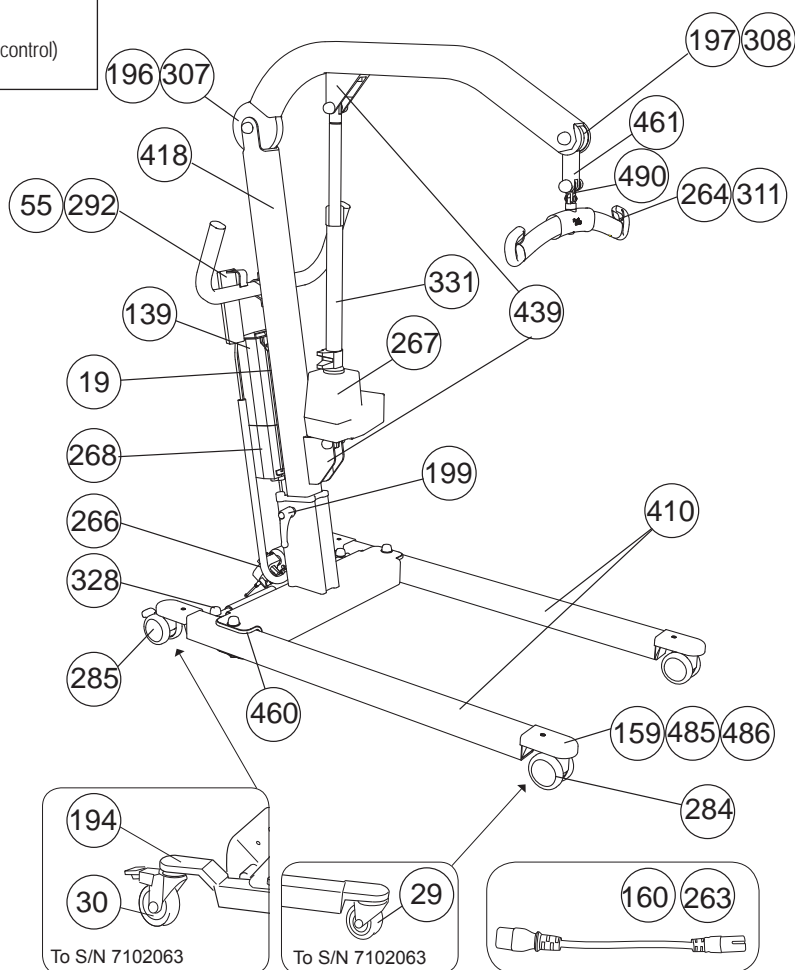


**Fits Viking M, all models:**

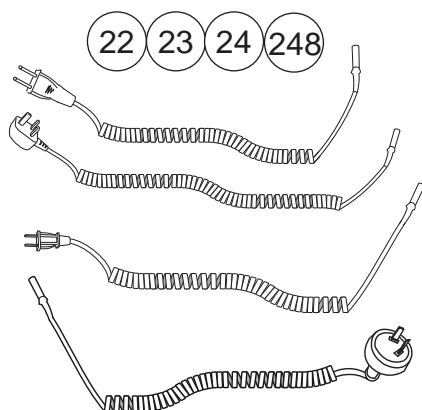

**Product No. 2040015** (S/N 7 500 401 - 7 600 500)

**2040035** (S/N 9 200 001 - 9 300 000)

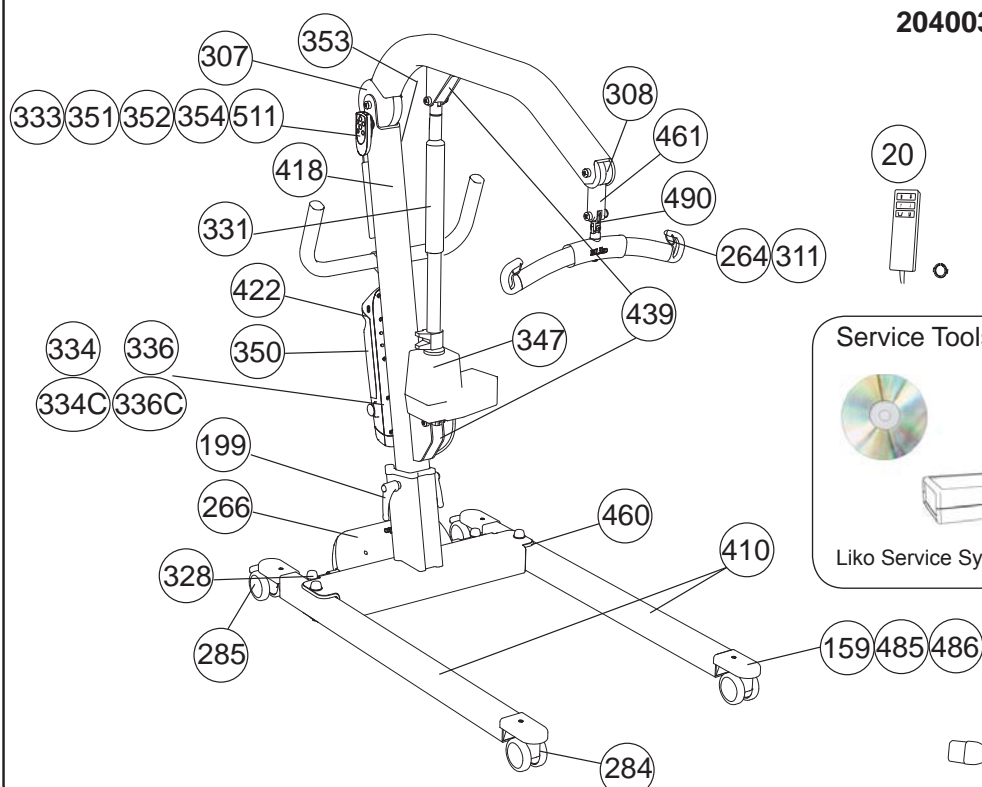

**Service Tools, see page 79**



Liko Service System



Liko Diagnostic System




## Technical Data


### Viking M, Prod. No. 2040005

<b>Lifting speed:</b>	3.5 cm/s (1.4 inch./s) without load
<b>Batteries:</b>	Two 12 V 2.9 Ah valve-regulated lead-acid gel-type batteries. New batteries are provided by the supplier.
<b>Battery charger:</b>	Built-in charger, 100-240 VAC, 50-60 Hz, max 400mA
<b>Motor (mast):</b>	24 V, 9.2 A, permanent magnetic motor with mechanical safety mechanism and safety nut.
<b>Motor (base):</b>	24 V, 5.5 A, permanent magnetic motor.
<b>Material:</b>	Aluminium
<b>Emergency lowering:</b>	Mechanical and electrical


<b>Wheels:</b>	Front: 75 mm twin castor wheels. Rear: 75 mm brake-equipped twin castor wheels.
<b>Intermittent operation:</b>	Int. Op 10/90, active operation max 2 min. Out of a time of 100, active must be less than 10, though not more than 2 min.
<b>Degree of protection:</b>	IP 43

 Built for indoor operation.

 Type B, according to the degree of protection against electrical shock.

 Class II equipment.

## Measurements

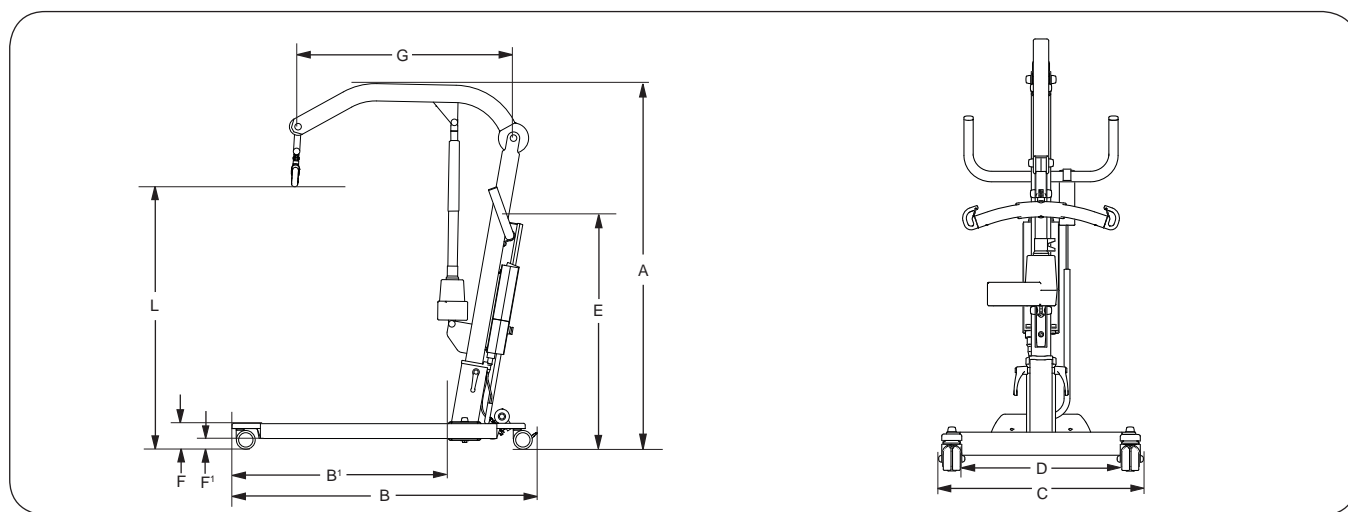

Viking M	Max. load	L* Max.	L* Min.	A* Max.	A* Min.	B	B'	C	D	E*	F	F'	G	Weight Total	Weight Part	Turning diameter
A:	205	179 174 169	52 47 42	202 197 192	144 139 134	123	87	69-111	56-97	100 105 110	10,5	4,5	88	35	18	140
B:	450	70 69 67	21 19 17	80 78 76	57 55 53	49	34	27-44	22-38	39 41 43	4.1	1.6	35	77	40	55

A: Max. load and weight in kg. Measurement in cm.

B: Max. load and weight in lbs. Measurement in inch.

\* These measures depend on in which hole the mast is levelled and secured.

## Technical Data

### Viking M, Prod. No. 2040015

**Maximum load:** 205 kg (450 lbs)

**Material:** Aluminium

**Weight:** *Total weight:* 34.0 kg (75 lbs) *Heaviest part:* 15.3 kg (34 lbs)

**Wheels:** Front: 75 mm (3 inch.) twin wheels.  
Rear: 75 mm (3 inch.) twin wheels with brakes.

**Turning diameter:** 1400 mm (55.2 in)

**Em. Lowering:** Manual and electrical

**Lifting interval:** 1270 mm (50.0 in)

**Lifting Speed (no load):** 42 mm/s and 38 mm/s, respectively  
(1.7 in/s and 1.5 in/s, respectively)

**Sound Level:** 46 dB(A)

**Protection Class:** IP 54

**Op. Forces of Controls:** HandControl: 4 N  
Operation Panel: 4 N

**HandControl IR:** Batteries: 2 AAA LR03 1.5 V.  
Reach: 2 m (80 in)

**Electrical Data:** 24 V

**Intermittent operation:** Int. Op 10/90, active operation max 2 min.  
Only 10% of a given length of time may be active, yet no more than 2 min.

**Batteries:** NiMH cells, 2.2 Ah. New batteries provided by the supplier.


**Battery Charger:** CBL20001, built-in, 100-240 V AC, 50-60 Hz, max 600 mA.


**Lift Motor:** Permanent magnet motor with mechanical safety mechanism; 24 V, 8.5 A

**Motor for base-width adjustment:** Permanent magnet motor  
24 V, 6 A

 The device is intended for use indoors

 Type B, in accordance with the electrical shock protection class.

 Class II equipment.

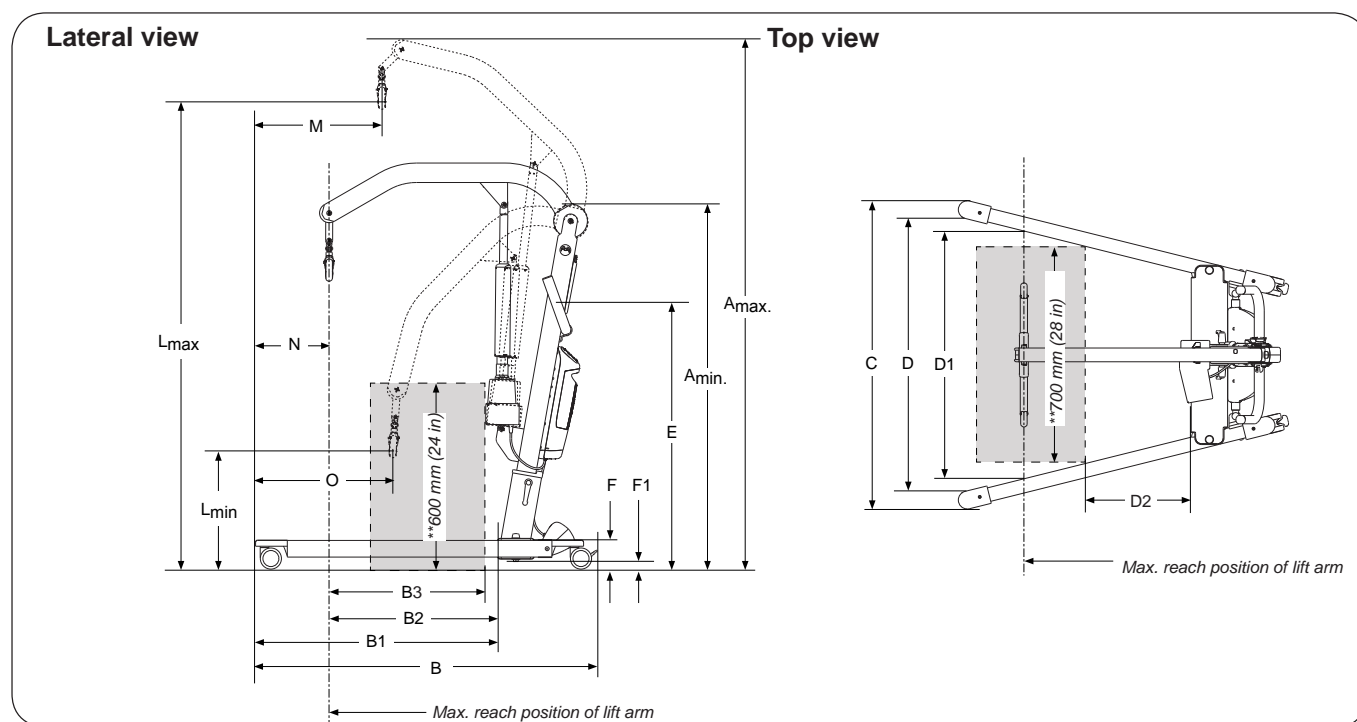

Measurement in mm

<i>A<sub>max</sub></i> *	<i>A<sub>min</sub></i> *	<i>B</i>	<i>B<sub>1</sub></i>	<i>B<sub>2</sub></i>	<i>B<sub>3</sub></i> **	<i>C</i>		<i>D</i>		<i>D<sub>2</sub></i> **	<i>E</i> *	<i>F</i>	<i>F<sub>1</sub></i>	<i>L<sub>max</sub></i> *	<i>L<sub>min</sub></i> *	<i>M</i>	<i>N</i>	<i>O</i>
						max.	min.	max.	min.									
2020	1440	1230	870	660	650	1110	690	970	560	195	1100	105	30	1790	520	385	220	495
1970	1390				650						1050			1740	470			
1920	1340				595						1000			1690	420			

Measurement in inch.

<i>A<sub>max</sub></i> *	<i>A<sub>min</sub></i> *	<i>B</i>	<i>B<sub>1</sub></i>	<i>B<sub>2</sub></i>	<i>B<sub>3</sub></i> **	<i>C</i>		<i>D</i>		<i>D<sub>2</sub></i> **	<i>E</i> *	<i>F</i>	<i>F<sub>1</sub></i>	<i>L<sub>max</sub></i> *	<i>L<sub>min</sub></i> *	<i>M</i>	<i>N</i>	<i>O</i>
						max.	min.	max.	min.									
79.5	56.7	48.4	34.3	26.0	25.6	43.3	27.2	38.2	22.0	7.7	43.3	4.2	1.2	70.5	20.5	15.2	8.7	19.5
77.6	54.7				25.6						41.4			68.5	18.5			
75.6	52.8				23.4						39.4			66.5	16.5			

Note: The measurements are based on the lift being equipped with standard sling bar and standard wheels. When changing to other lifting accessories, or other wheels, check that the lift still achieves desired lifting height.

\* Different measurements apply to Viking M depending on the height-setting position

\*\* Reference measurement according to Standard EN ISO 10535:2006.


## Technical Data


### Viking M, Prod. No. 2040035


<b>Maximum load:</b>	205 kg (450 lbs)	
<b>Material:</b>	Aluminium	
<b>Weight:</b>	Total weight:	Heaviest part:
	30,2 kg (66.6 lbs)	15,8 kg (34.8 lbs)
<b>Wheels:</b>	Front: 75 mm (3 inch.) twin wheels Rear: 75 mm (3 inch.) twin wheels with brakes	
<b>Turning diameter</b>	1400 mm (55.2 in)	
<b>Emergency lowering device:</b>	Mechanical and electrical	
<b>Lifting interval:</b>	1270 mm (50.0 in)	
<b>Lifting speed (no load)</b>	42 mm/s and 38 mm/s, respectively (1.7 in/s and 1.5 in/s, respectively)	
<b>Sound level:</b>	46 dB(A)	
<b>Protection class:</b>	IP 54	

<b>Operating Forces of Controls:</b>	HandControl: 2.4 N Operation Panel: 4 N
<b>Electrical data:</b>	24 V
<b>Intermittent power:</b>	Int. Op 10/90, active operation max 2 min. Only 10% of a given length of time may be active, but no more than 2 min.
<b>Batteries:</b>	NiMH cells, 2.2 Ah. New batteries provided by the supplier.
<b>Battery charger:</b>	CBL20002, built-in, 100-240 V AC, 50-60 Hz, max 600 mA.
<b>Lift motor:</b>	Permanent magnet motor with mechanical safety mechanism 24 V, 8.5 A
<b>Motor for base-width adjustment:</b>	Permanent magnet motor 24 V, 6 A

 The device is intended for use indoors

 Type B, in accordance with the electrical shock protection class.

 Class II equipment.

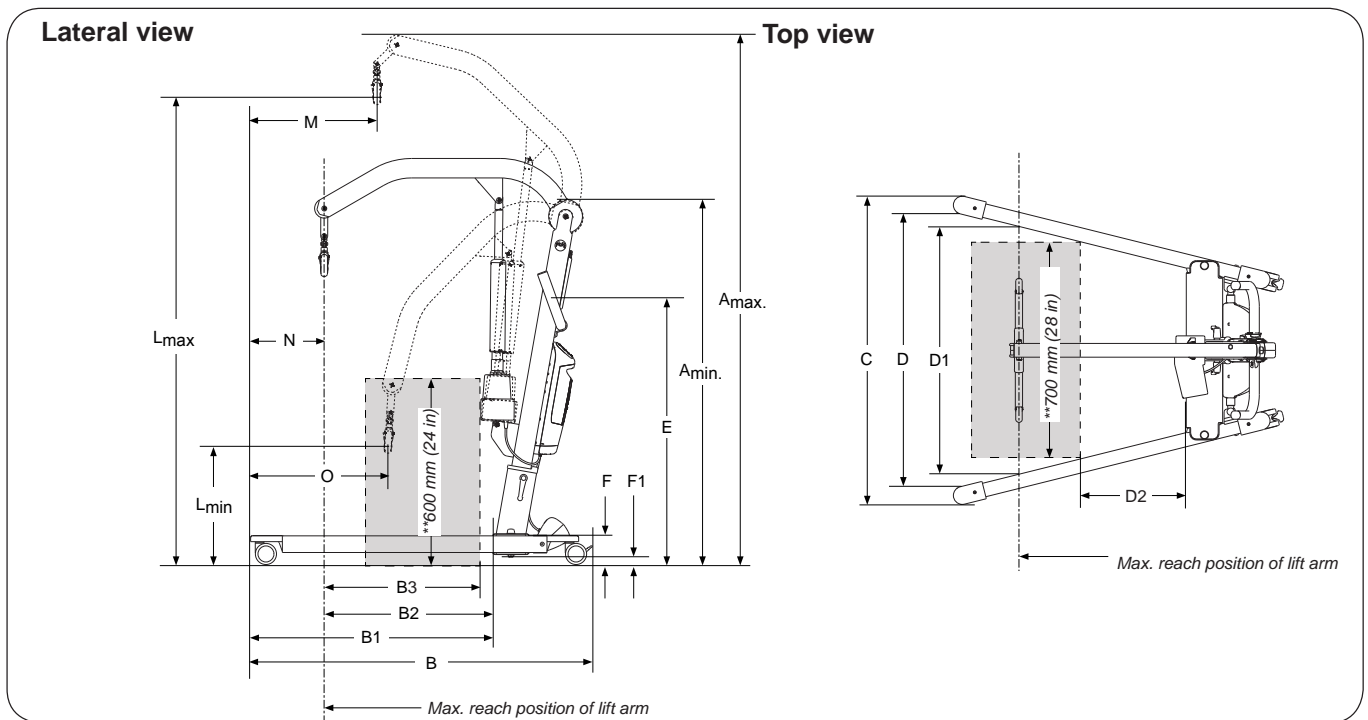

Measurement in mm

$A_{max}^*$	$A_{min}^*$	$B$	$B1$	$B2$	$B3^{**}$	$C$		$D$		$D2^{**}$	$E^*$	$F$	$F1$	$L_{max}^*$	$L_{min}^*$	$M$	$N$	$O$
						max.	min.	max.	min.									
2020	1440	1230	870	660	650	1110	690	970	560	195	1100	105	30	1790	520	385	220	495
1970	1390				650						1050			1740	470			
1920	1340				595						1000			1690	420			

Measurement in inch.

$A_{max}^*$	$A_{min}^*$	$B$	$B1$	$B2$	$B3^{**}$	$C$		$D$		$D2^{**}$	$E^*$	$F$	$F1$	$L_{max}^*$	$L_{min}^*$	$M$	$N$	$O$
						max.	min.	max.	min.									
79.5	56.7	48.4	34.3	26.0	25.6	43.3	27.2	38.2	22.0	7.7	43.3	4.2	1.2	70.5	20.5	15.2	8.7	19.5
77.6	54.7				25.6						41.4			68.5	18.5			
75.6	52.8				23.4						39.4			66.5	16.5			

Note: The measurements are based on the lift being equipped with standard sling bar and standard wheels. When changing to other lifting accessories, or other wheels, check that the lift still achieves desired lifting height.

\* Different measurements apply to Viking M depending on the height-setting position

\*\* Reference measurement according to Standard EN ISO 10535:2006.

## Trouble shooting

### Viking M, Prod. No. 2040005

**The lift does not work up/down.  
Width adjustment does not  
work in/out.**



1. Check that the emergency stop button is not pushed in.
2. Check that the cables are connected correctly.
3. Check that the charging cable is not connected to the wall socket.
4. Check that the battery is charged.
5. Check that the contact plates of the battery pack are not defective or broken off.
6. *If the lift still does not work satisfactorily, contact Liko/Hill-Rom.*

**Battery charging does not  
work.**



1. Check that the emergency stop button is not pushed in.
2. Check that the contact plates of the battery pack are not defective or broken off.
3. *If the lift still does not work satisfactorily, contact Liko/Hill-Rom.*

**The lift stops in the elevated  
position.**



1. Check that the emergency stop button is not pushed in.
2. Use the electrical emergency lowering to safely lower the patient.
3. Use the mechanical emergency lowering device (marked) to safely lower the patient.
4. Check that the battery is charged.
5. *If the lift still does not work satisfactorily, contact Liko/Hill-Rom.*

**If you hear unusual sounds.**



*Contact Liko/Hill-Rom.*



## Trouble shooting

### Viking M, Prod. No. 2040015

***The lift does not work up/down with the HandControl IR.***

***The base-width adjustment doesn't work (in/out) with the HandControl IR.***



1. Make sure that the emergency stop is not engaged.
2. If the lift has not been used for 24 hours, enable the system.
3. Check that the HandControl IR is enabled.
4. Check that the charger cable is not connected to an electric outlet.
5. HandControl IR indicates battery change.
6. If the lift works via the operation panel, change the HandControl IR.
7. If the problem persists, please contact Liko/Hill-Rom.

***The lift does not work up/down with the operation panel.***

***The base-width adjustment doesn't work (in/out) with the operation panel.***



1. Make sure that the emergency stop is not engaged.
2. Make sure that the cables to the control box are connected correctly.
3. Check that the charger cable is not connected to an electric outlet.
4. Check the battery capacity.
5. Check that the battery's contact plates are not defective or broken.
6. *If the problem persists, please contact Liko/Hill-Rom.*

***The charger doesn't work.***



1. Make sure that the emergency stop is not engaged.
2. Check that the charging cable is correctly connected
3. Make sure that the battery is properly attached.
4. Check that the battery's contact plates are not defective or broken.
5. If the problem persists, please contact Liko/Hill-Rom.

***The lift is stuck in the high position.***



1. Make sure that the emergency stop is not engaged.
2. Check the battery capacity.
3. HandControl IR indicates battery change.
4. Use the electrical emergency lowering device to lower the patient onto a firm surface.
5. Use the manual emergency lowering device to lower the patient onto a firm surface.
6. *If the problem persists, please contact Liko/Hill-Rom.*

***If you hear unusual sounds.***



*Contact Liko/Hill-Rom.*

## Trouble shooting

Viking M, Prod. No. 2040035

**The lift doesn't work up/down with Hand Control.**

**The base-width adjustment doesn't work (in/out) with the Hand Control.**



1. Check that the emergency stop button has not been pressed.
2. Check the battery capacity.
3. Check that the charger cable is not connected to an electric outlet.
4. Check that the hand control cable is correctly connected.
5. If the lift works via the operation panel, change the Hand Control.
6. *If the problem persists, please contact Liko/Hill-Rom.*

**The lift does not work up/down with the operation panel.**

**The base-width adjustment doesn't work (in/out) with the operation panel.**



1. Check that the emergency stop button has not been pressed.
2. Check that the cables to the control box are connected correctly.
3. Check that the charger cable is not connected to an electric outlet.
4. Check the battery capacity.
5. *If the problem persists, please contact Liko/Hill-Rom.*

**The charger doesn't work.**



1. Check that the emergency stop button has not been pressed.
2. Check that the charger cables are connected correctly.
3. Make sure that the battery is properly attached.
4. *If the problem persists, please contact Liko/Hill-Rom.*

**The lift is stuck in the high position.**



1. Check that the emergency stop button has not been pressed.
2. Check the battery capacity.
3. Check that the hand control cable is connected correctly.
4. Electrical emergency lowering, use the operation panel to lower the patient onto a firm surface.
5. Use the mechanical emergency lowering device to lower the patient onto a firm surface.
6. *If the problem persists, please contact Liko/Hill-Rom.*

**If you hear unusual sounds.**




Contact Liko/Hill-Rom.

## Trouble shooting


### Control Unit CBL

**Note!** Begin with Simple trouble shooting from Service manual for the actual product. If problem remains continue as described below.

#### Tools Required

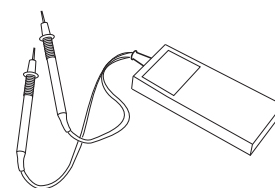

Multimeter

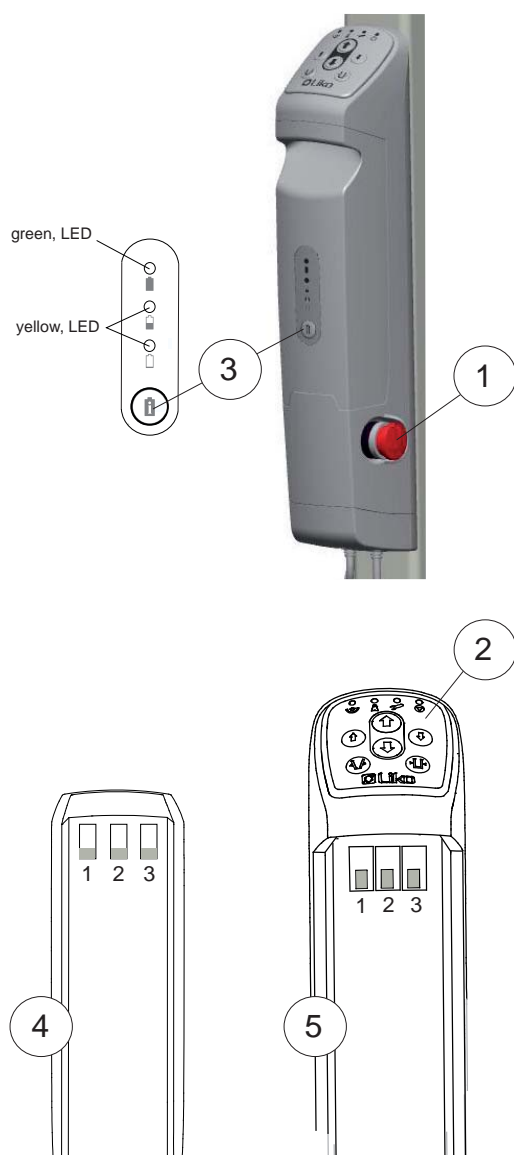

#### Check point:

1. Make sure that:
  - the emergency stop ① is not activated.
  - the battery is assembled correctly.
  - the charging cable is not connected to mains.
2. Check operation panel function's for actuator and base motor
 

Push control buttons on operation panel ② at control box

  - If both actuator and base motor functions response, continue to step 5.
  - If no function, continue to step 3.
  - If only one of either the actuator or base motor function response: change position on actuator and base motor cables then check control function of operation panel, to see if fault is on output channels or the motors. If control box function is ok! Continue to step 5. If control box is faulty. **Replace the control box.**
3. Check battery capacity
 

3a) Push in the battery capacity button ③:

  - If any of the three LED's on the battery start's the battery is OK, continue to step 4.
  - If not, continue to step 3b.

3b) Measure battery voltage: release battery, ④ measure on contact 1 and 3:

  - If voltage over 0V the battery is ok! Continue to step 4.
4. Verify contact between control box and battery.
 

4a) Remove the battery and wait for green LED to turn off.

4b) Put back the battery and see if green LED start automatically.

  - If green LED starts, battery contact is ok!
  - If not, try to ensure that the battery's contact plates are not defective or broken and try to bend them out carefully, then repeat step 4.
  - If still no contact, continue to step 5.
5. Check charging function
 

5a) Connect the charging cable to mains

  - If charging starts within 1 minute, charge and then continue to step 6.
  - If charging doesn't start, continue to step 5b.





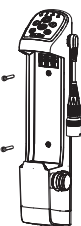
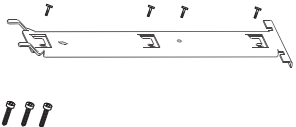
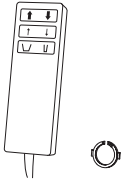
5b) Measure output voltage on the control box ⑤ on contact 1 and 3.

  - If voltage is 3-4V charging is OK! **Replace the battery**, continue to step 6.
  - If voltage is 0V when the charging cable is connected to mains the control box needs to be replaced! **Replace the control box.**
6. Charge the battery until green LED is flashing, on the battery.
  - Check that all function's on the control box is ok!
  - If not , **Replace control box.**














#### Customers FAQ:

- **The liftarms up/down function or base width in/out function doesn't work?**
  - a) Change position on actuator and base motor cables.
  - b) Perform step 2 and see if fault is on output channel or motors.
- **The control panel function's on controlbox is ok, but the hand control has no function?**
  - a) Check to ensure that battery contact plates in the hand control are not defective or broken.
- **There's no charging function?**
  - a) Verify contact between control box and battery. Perform step 4.
  - b) Check charging function. Perform step 5.

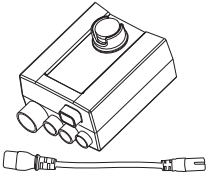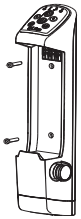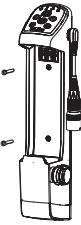



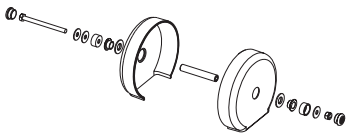
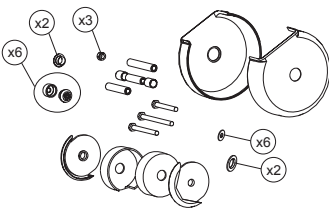

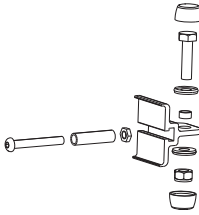
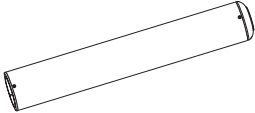

## Spare parts

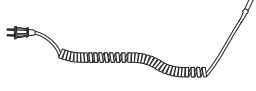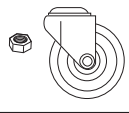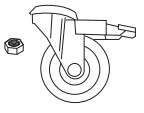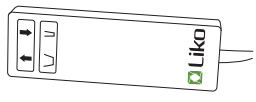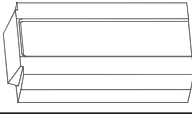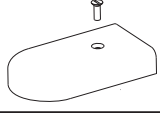Viking M							
<p>△ There are two models of Viking M; Prod. No. 2040005 and 2040015. Which spare part that fits to each model, can be read in the columns to the right. (x=fits with this model)</p>							
Viking M Control Units							
PROD. NO.	POS. NO.	NAME	UNIT		2040005	2040015	2040035
20490013	268	Control box Viking M S/N 7100101-7200200	pce		x		
20490013US	268	Control box Viking M , US S/N 7100101-7200200 <i>only for the US-market</i>			x		
20490032	334	Control Unit CBL, Viking M <b>Before use; requires Software and, needs to be programmed with Liko Service System, Prod. No. 21090024</b>	pce			x	
20490053	334C	Control Unit CBL Viking M, Cable <b>Before use; requires Software and, needs to be programmed with Liko Service System, Prod. No. 21090024</b>	pce				x
20490039	336	Control Unit CBL, by S/N <b>Can only be ordered if S/N (serial number) can be provided to Liko Customer Service. Will be programmed at Liko before delivery. 2 weeks delivery time from Liko.</b>	pce			x	
20490054	336C	Control Unit CBL by S/N, Cable <b>Can only be ordered if S/N (serial number) can be provided to Liko Customer Service. Will be programmed at Liko before delivery. 2 weeks delivery time from Liko.</b>	pce				x
Viking M Spare Parts							
20090023	19	Attachment for Control Box	pce		x		
20090089	20	Hand control 6-Button	pce				x

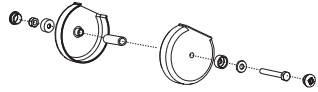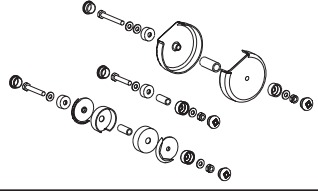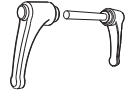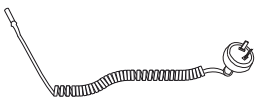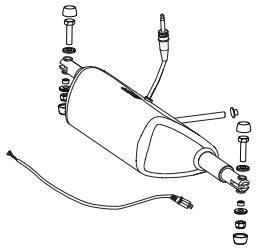




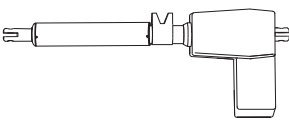


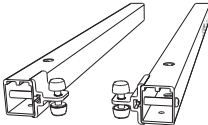
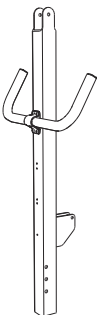

## Spare parts

Viking M								
PROD. NO.	POS. NO.	NAME	UNIT		2040005	2040015	2040035	
20090026	22	Cable for charging EU	pce		X	X	X	
20090027	23	Cable for charging UK			X	X	X	
20090028	24	Cable for charging US			X	X	X	
20090034	29	Front wheel 75 mm to S/N 7102063	pce		X			
20090035	30	Rear wheel 100 mm to S/N 7102063	pce		X			
20190017	55	Hand control EE 4 buttons	pce		X			
2006106	139	Extra Battery	pce		X			
20390006	159	Front Fork Cover	pair		X	X	X	
21090001	160	Extension cable	pce		X			
20490005	194	Rear fork cover to S/N 7102063	pair		X			
20490007	196	Lift Arm Plastic Cover to S/N 7106200	set		X			
20490008	197	Restoration set Overhead beam to S/N 7106200	set		X			
20490010	199	Locking handle Viking XL	pair		X	X	X	
20090050	248	Cable for charging AU/NZ	pce		X	X	X	
21090011	263	Extension Cable US	pce		X			
31590011	264	Latch Universal SlingBar	pair		X			
20490011	266	Base Motor Viking M, L (Viking M 2040005 up to S/N 7107499 also need spare part 20490028, Pos 328)	pce		X	X	X	







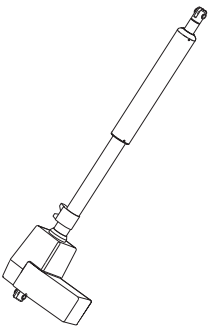
## Spare parts

Viking M								
PROD. NO.	POS. NO.	NAME	UNIT		2040005	2040015	2040035	
20490012	267	Actuator Viking M <i>S/N 7100101-7200200</i>	pce		x			
20490014	284	Front Wheel 75 mm Standard <i>from S/N 7102064</i>	pce		x	x	x	
20490015	285	Rear Wheel 75 mm Standard <i>from S/N 7102064</i>	pce		x	x	x	
20090058	292	Hook for Hand control	pce		x		x	
20490023	307	Lift Arm Plastic Cover <i>from S/N 7106201</i>	set		x	x	x	
20490024	308	Overhead Beam Rest. set <i>from S/N 7106201</i>	set		x	x	x	
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		x	x	x	
20490028	328	Restoration set motor attach- ment	set		x	x	x	
20490030	331	Outer Tube	set		x	x	x	
20490031	333	Wireless hand control HBL20	set			x		

## Spare parts

Viking M								
PROD. NO.	POS. NO.	NAME	UNIT		2040005	2040015	2040035	
20490033	347	Actuator Vi M <i>S/N 7500401 - 7600500</i>	pce			X	X	
21090018	348	Extension cable CBL20	pce			X	X	
21090019	349	Extension cable CBL 20 US,	pce			X	X	
2006107	350	Extra battery box	pce			X	X	
21090020	351	Beltclip HBL20	pair			X		
21090021	352	Clips with hook HBL20	pair			X		
21090022	353	Charge cable hook , 10 pce	set			X	X	
21090023	354	Hand control hanger, 10 pce	set			X		
20490036	410	Viking M Legs <i>from S/N 7100308</i>	pair		X	X	X	
20490038	418	Mast Viking M <i>including handle and motor attachment</i>	set		X	X	X	
20090077	422	Locking device CBL20, 10 pce	set			X	X	

## Spare parts

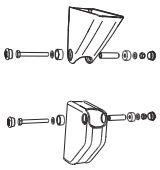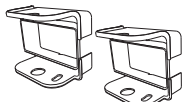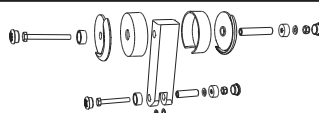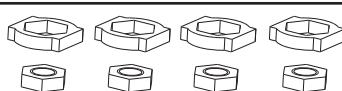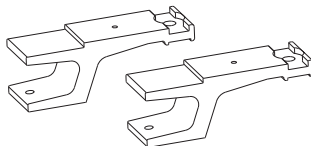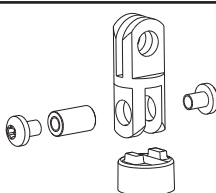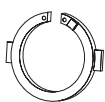Viking M								
PROD. NO.	POS. NO.	NAME	UNIT		2040005	2040015	2040035	
20490040	439	Bracket Cover	set		X	X	X	
20490048	460	Leg Cover M	set		X	X	X	
20490049	461	Flexlink	set		X	X	X	
20090080	485	Fork locking attachment, 4 pair	set		X	X	X	
20090081	486	Wheel Fork	pair		X	X	X	
20490052	490	SlingBar Assembly set	set		X	X	X	
20490057	511	Locking Ring CBL, 10 pce	set				X	



## Assembly Instruction Monteringsanvisning

Prod. No. 20090023

3SE630238-05  
2011-02-08

### Contains:

Screw MRT-GF M5x16

### Qty:

3

Screw MCL6S M5x8

4

Control box attachment

1

### Fits:

Golvo 7000ES/7007ES, 7007 LowBase

Uno 100 EM/EE

Uno 102 EM/EE/ES

Sabina EM/EE

Sabina II EM/EE

Viking XL (Prod. No. 2040003)


Viking L (Prod. No. 2040004)

Viking M (Prod. No. 2040005)

Liko M220 / M230




### Tools required:

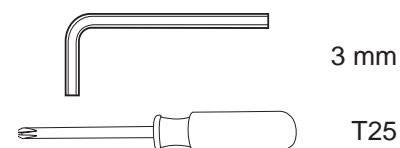


1.

 x 3 (Viking M, L, XL, Golvo 7000ES/7007ES)

 x 3 (Sabina, Uno, Liko M220/M230)

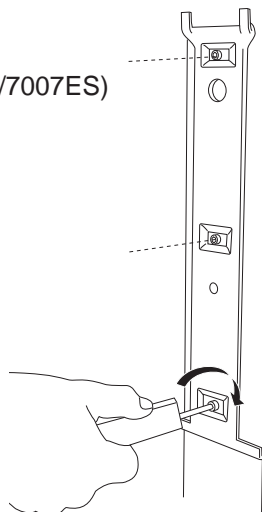


2.

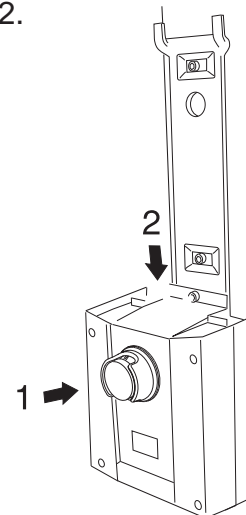


3.


 x 1

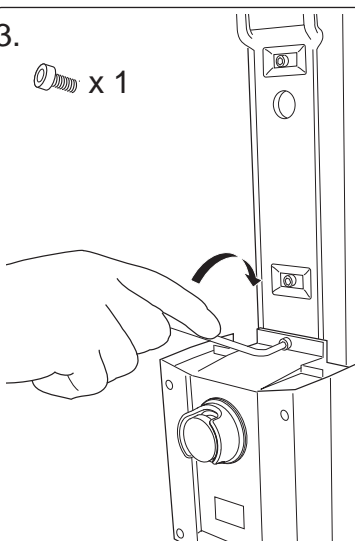

4.

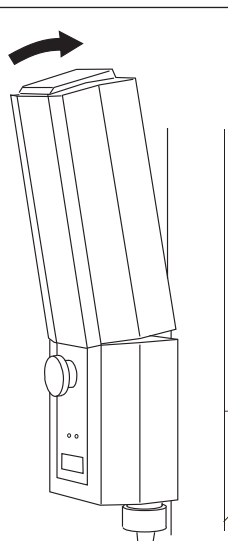

5. (Holder for the charging cable, is not standard on every lift model)

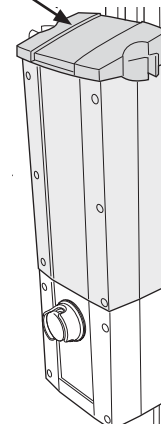

### NOTE


- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

# Assembly Instruction Monteringsanvisning

Prod. No. 20090089

3SE690232-02  
2013-01-17





<b>Contains:</b>	<b>Qty:</b>
Hand control ES	1
Locking Ring	1

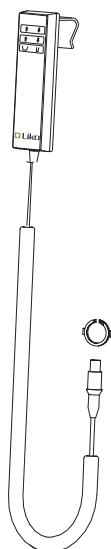

## Tools required




<b>Product:</b>	<b>Prod. No.</b>
Viking XL	2040033
Viking L	2040034
Viking M	2040035
Golvo 8000	2000034
Golvo 8008	2000035
Golvo 8008 LowBase	2000039

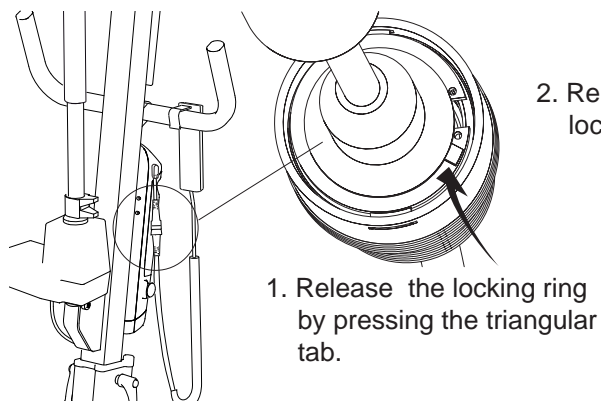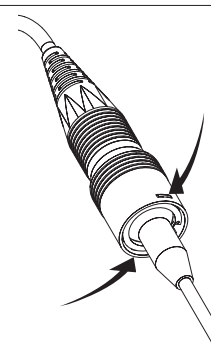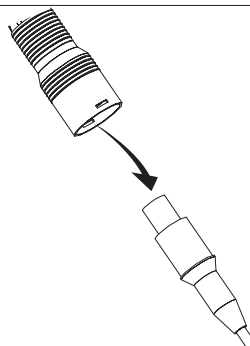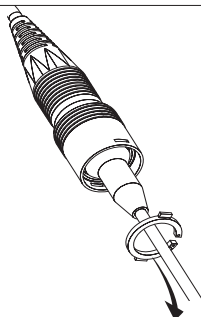

2. Remove the locking ring.


3. Remove the hand control. Assemble in reverse order.

4. Make sure the rectangular tabs at the locking ring are fixed.

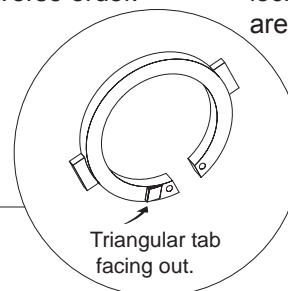

<b>Product:</b>
Golvo 7000ES/7007ES
Golvo 7007 LowBase
Uno 102 ES
Sabina EE
Sabina II EE

Use no locking ring on these products.

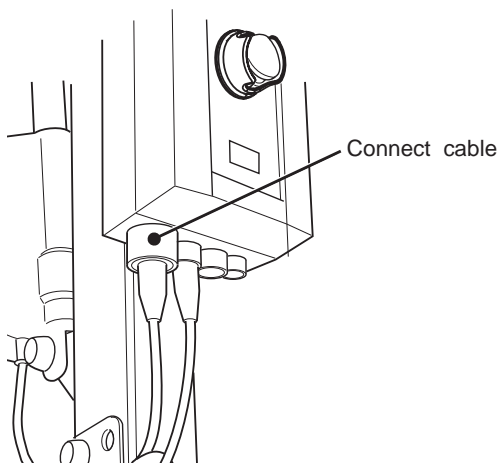

## NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

### Assembly Instruction Monteringsanvisning

**Contains:**      **Qty:**

Charging cable

1

**Fits:**

Golvo 7000ES / 7007ES / 7007 LowBase

Golvo 8000 / 8008 / 8008 LowBase

Uno 100 EM/EE

Uno 102 EM/EE/ES

Sabina EM/EE


Sabina II EM/EE

Viking XL

Viking L

Viking M

Prod. No. 20090026 EU

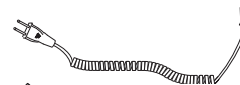

20090027 UK



20090028 US




20090050 AU/NZ




3SE630237-06

2012-09-11

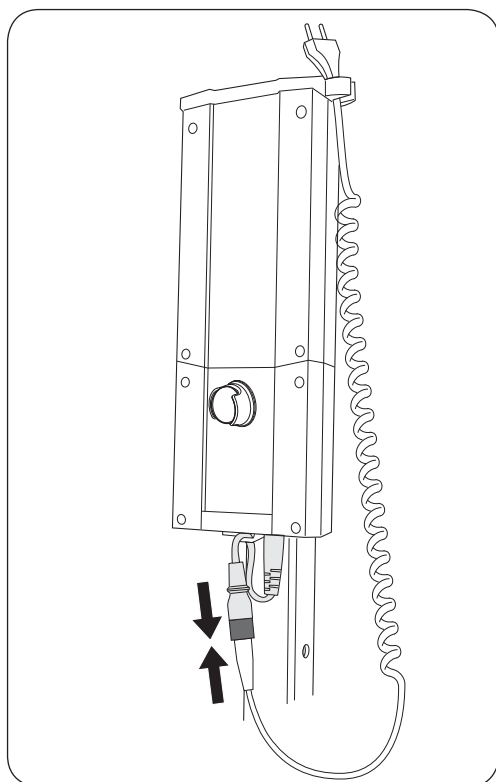

Lift equipped with Liko Diagnostic System™

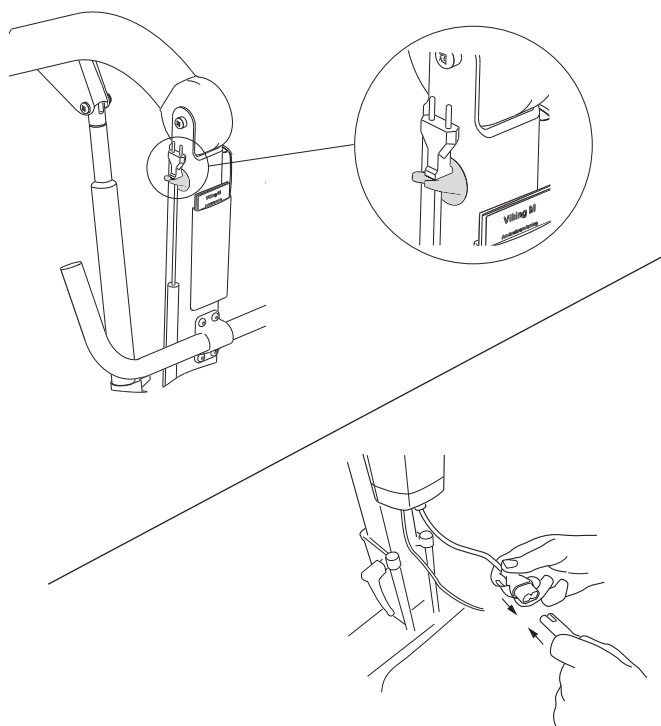

**The lift cannot be used when the charger cable is plugged into an electrical outlet.**

**△ NOTE**

- Never charge batteries in a wet area.
- The lift cannot be charged with the emergency stop engaged.
- When charging, a yellow indicator light on the control box will illuminate. The yellow light turns off when the battery is fully charged. If the light has not turned off after 8 hours of charging, the battery may need to be replaced. Stop charging and replace the battery.
- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20090034


3SE640273-02  
2010-04-21

### Contains:

Front wheel 75mm  
Locking nut M10 nyloc

### Qty:

1  
1

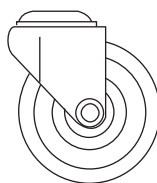


### Fits:

Golvo 1000, 2000, 3000  
Golvo 7000/7007, 7000 S/7007 S  
Golvo 7000 ES, to S/N 7024205  
Golvo 7007 ES, to S/N 7729721  
Viking M Prod. No. 2040005, to S/N 7102063


### Tools required:




1.

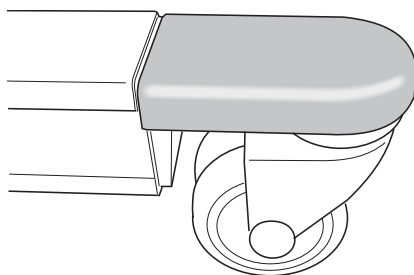

2.

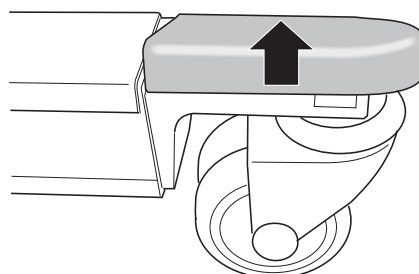

3.

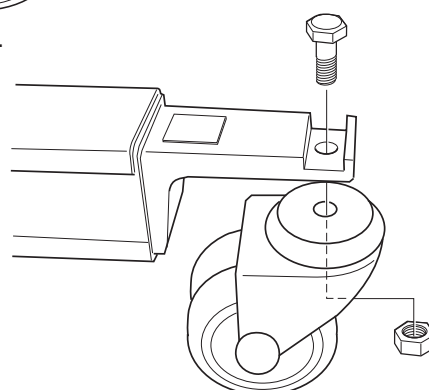

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20090035

3SE650201-04


2012-11-09

### Contains:

Lockable castor, d=100 mm  
Nut M12

### Qty:

1  
1

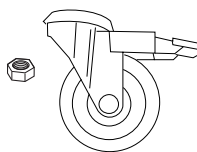


### Fits:

Golvo 1000, 2000, 3000  
Golvo 7000/7007, 7000 S/7007 S  
Golvo 7000/7007  
Golvo 7000 ES, to S/N 7024205  
Golvo 7007 ES, to S/N 7729721  
Viking M, Prod. No. 2040005, to S/N 7102063


Tools required:

 19 mm


1.

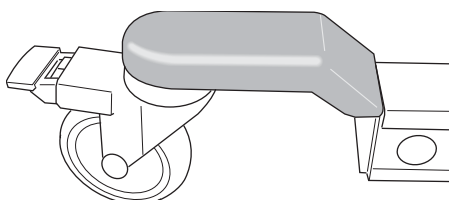

2.

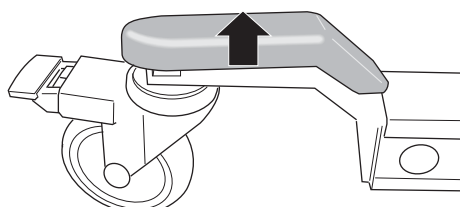

3.

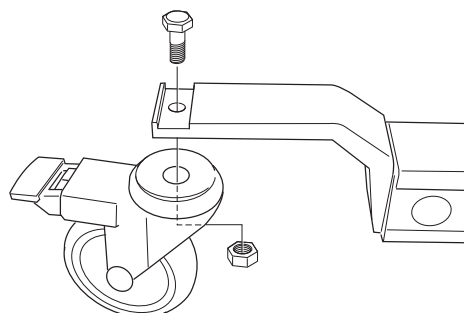


### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20190016, 20190017

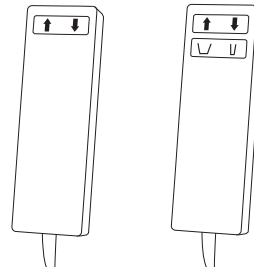

3SE630204-11  
2012-09-11

**20190016 Contains:** Qty:  
Hand control EM 2-Button 1



**20190017 Contains:** Qty:  
Hand Control EE 4-Button 1

**20190016 Hand control EM 2-Button, Fits:**

LikoLight  
Sabina EM  
Sabina II EM  
Uno 100 EM  
Uno 102 EM  
Viking S  
Viking XS

**20190017 Hand control EE 4-Button, Fits:**

Uno 100 EE  
Uno 102 EE  
Viking XL Prod. No. 2040003  
Viking L Prod. No. 2040004  
Viking M Prod. No. 2040005

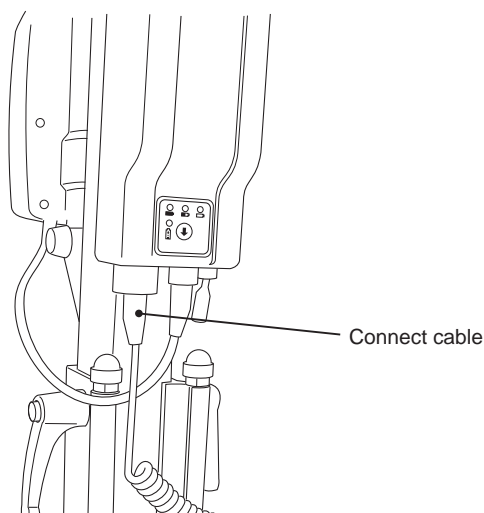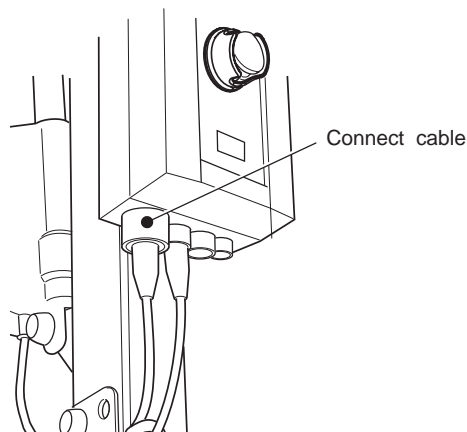

**△ NOTE**


- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 2006106




3SE630224-06  
2011-02-24

**Contains:**  
Battery box

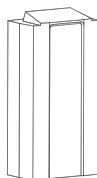**Qty:**  
1

**Fits:**

Caroli  
 Capella  
 Golvo 7000 ES/7007 ES (all models), 7007 Low Base  
 LikoLight  
 Liko M220 / M230  
 Sabina  
 Uno 100EM,EE,102EM,EE,ES  
 Viking XL (Prod. No. 2040003)  
 Viking L (Prod. No. 2040004)  
 Viking M (Prod. No. 2040005)

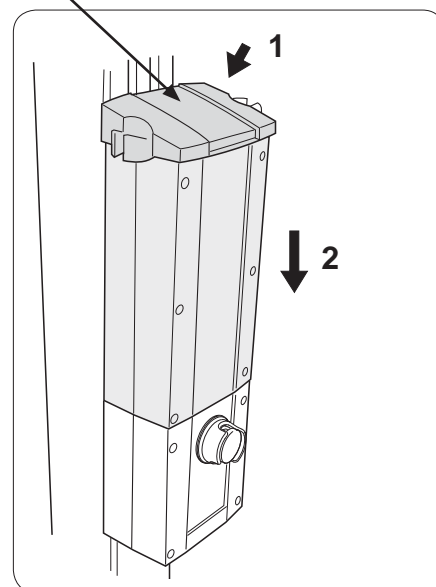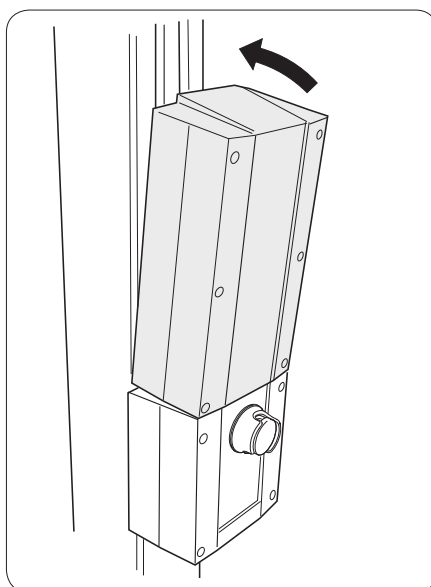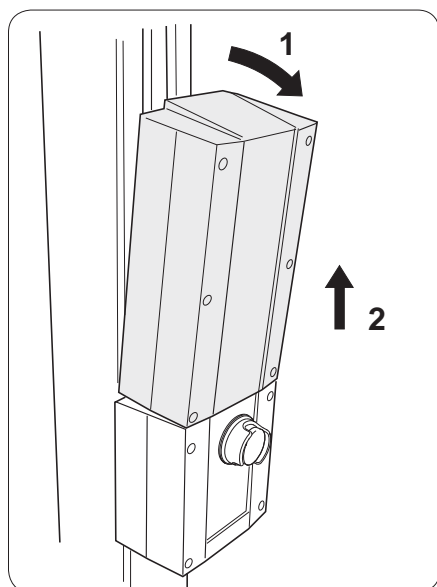(Holder for the charging cable, is not standard on every lift model)


**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.
- Old batteries are to be left at the nearest recycling station or given to personnel authorized by Liko.



## Assembly Instruction Monteringsanvisning

Prod. No. 20390006


3SE380202-04  
2010-05-04

### Contains:

Nylon screw M5x16 Nature  
Front fork cover grey light

### Qty:


2  
2

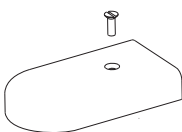


### Fits:

Golvo 7000 ES from S/N 7024206  
Golvo 7007 ES from S/N 7729722  
Golvo 7000 ES Classic / 7007 ES Classic  
Golvo 8000 / 8008 (all models)  
LikoLight  
Viking XL  
Viking L  
Viking M  
Viking S  
Viking XS  
Roll On


### Tools required:

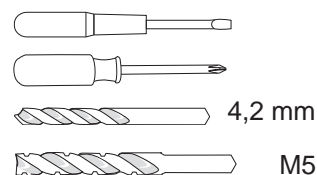


1.

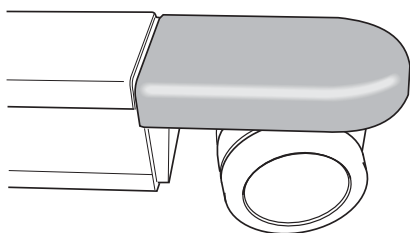


2.

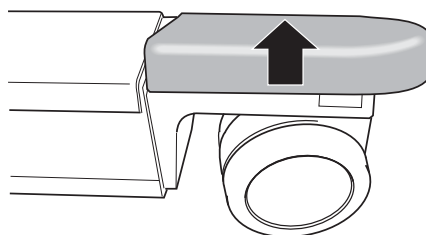


3. If not prepared with hole, drill [mm.]

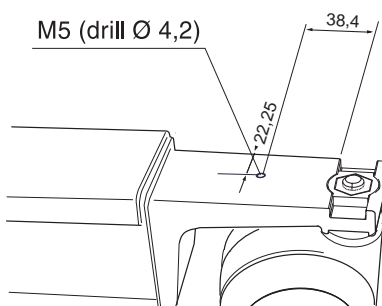


4.

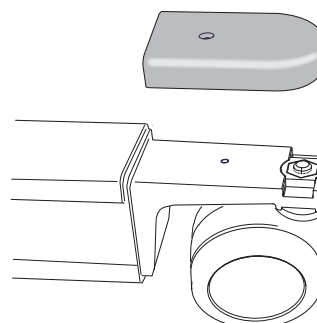

5.

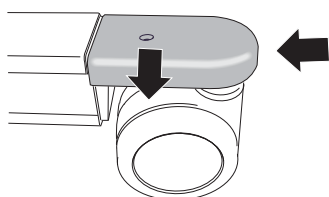

6.

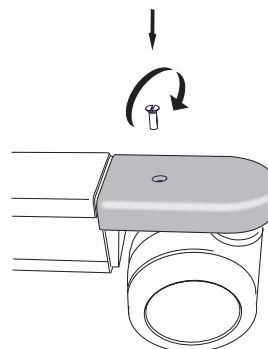

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.



## Assembly Instruction Monteringsanvisning

Prod. No. 21090001

3SE630225-05  
2011-03-02

### Contains:

Extension cable EU

### Qty:

1

### Fits:

Golvo 7000ES/7007ES

Golvo 7007 LowBase

Uno 100 EM/EE

Uno 102 EM/EE/ES

Sabina EM/EE


Sabina II EM/EE

Viking XL (Prod. No. 2040003)


Viking L (Prod. No. 2040004)

Viking M (Prod. No. 2040005)


Length 21 cm/ 8.5 inch

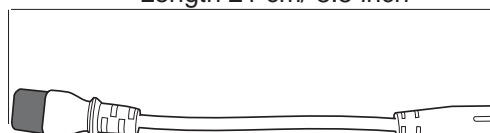


1.

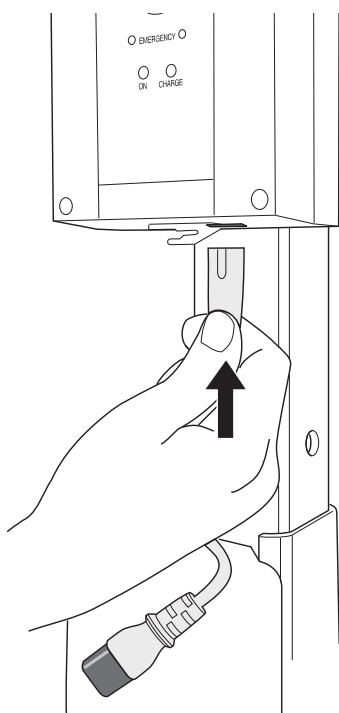

2.

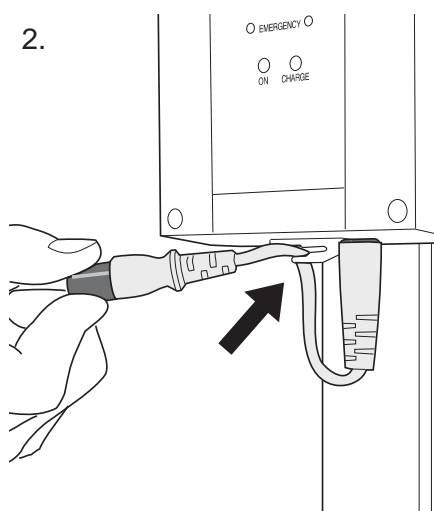

3.

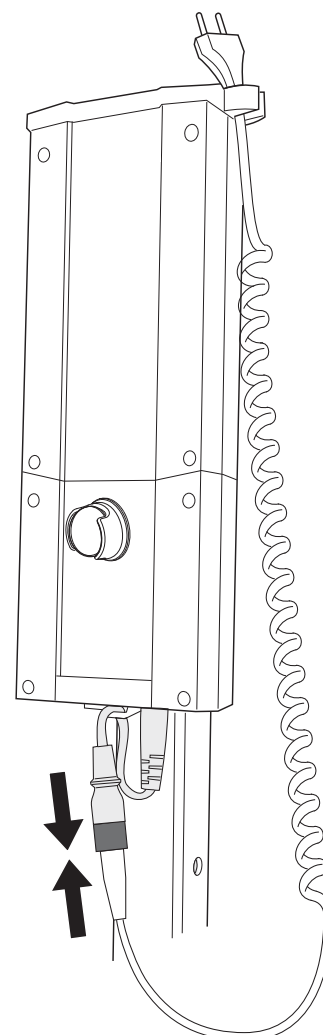

### NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490005


3SE630295-03  
2010-04-14

**Contains:**

Rear wheel cover, grey

**Qty:**

2

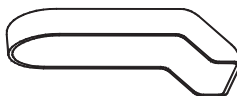



**Fits:**

Viking XL

Viking L

Viking M, Prod No. 2040005 (Up to S/N 7102063)

**Tools required:**

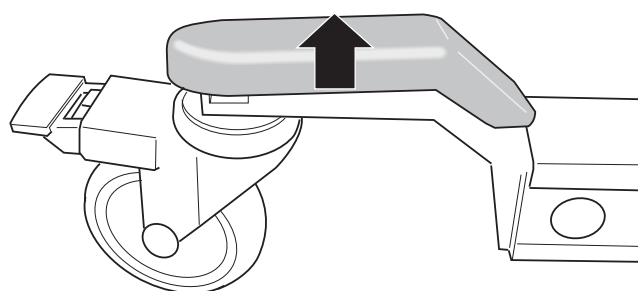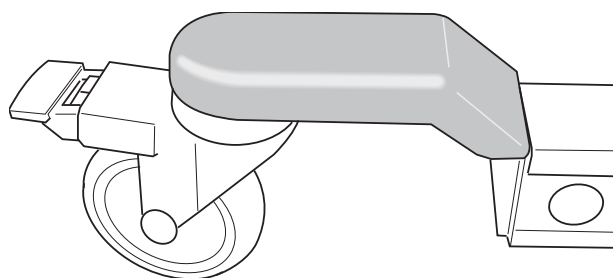

**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490007

3SE630240-05

2011-03-11

### Contains:


	Qty:
Plastic cover mast Viking	2
Cover hat Liko	2
Cover hat Liko bottom	2
Sleeve bearing Mob.62	1
Locking nut M6 Nyloc	1
Washer TBRB 6,4x12x3	3
Screw M6S M6x75	1

### Fits:

Viking XL (Prod. No. 2040003), to S/N 801999  
Viking L (Prod. No. 2040004), to S/N 7200794  
Viking M (Prod. No. 2040005), to S/N 7106200

Tools required:

2X  10 mm

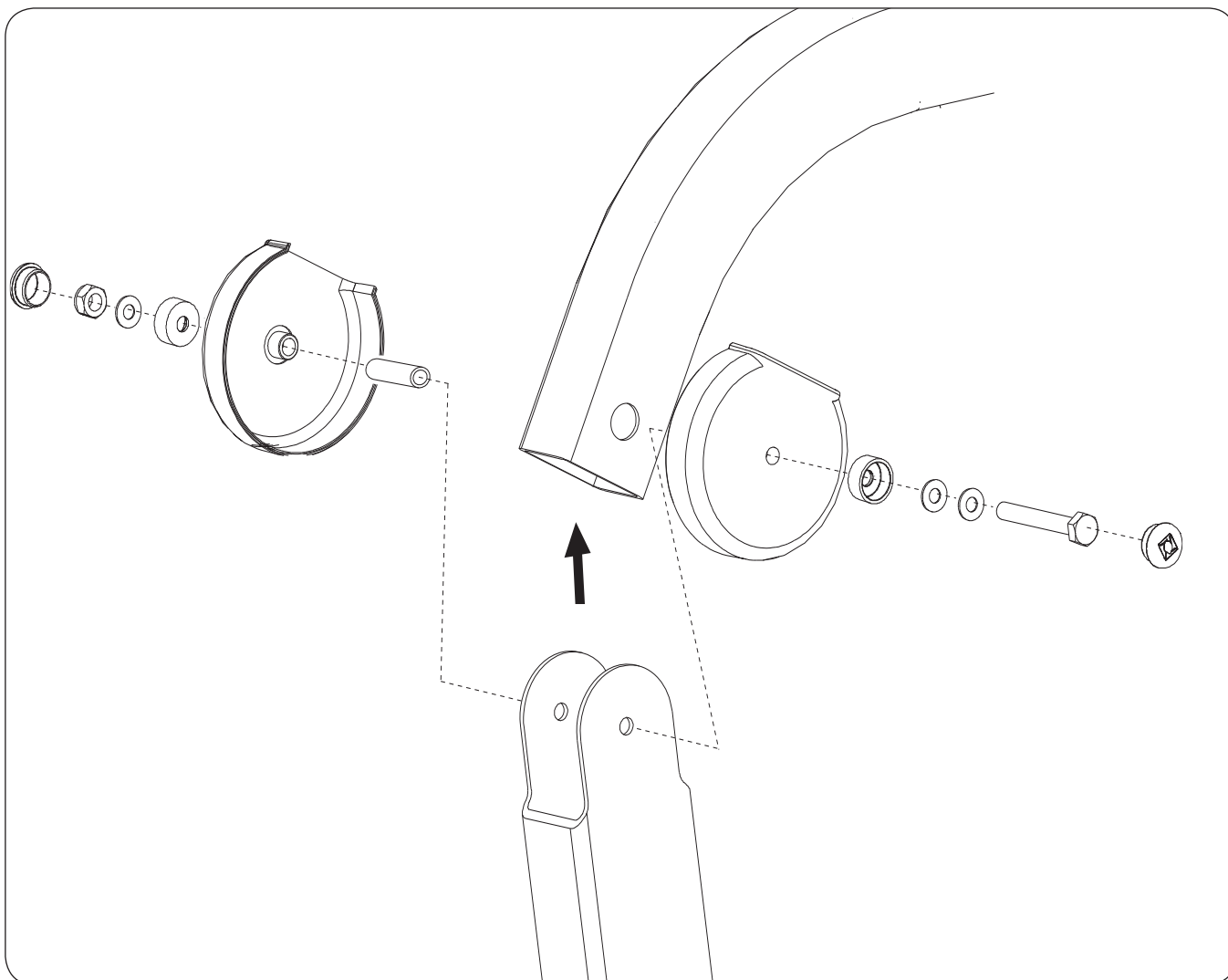

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

# Assembly Instruction Monteringsanvisning

Prod. No. 20490008

3SE630228-06  
2011-03-03

## Contains:

## Qty:

Plastic cover mast Viking	2
Cap Y Viking	2
Cap I Viking	2
Cover hat Liko & bottom	6
Sleeve bearing Mob.38	1
Sleeve bearing Mob.50	1
Sleeve bearing Mob.62	1
Locking nut M6 Nyloc	3
Screw M6S M6x50	1
Screw M6S M6x60	1
Screw M6S M6x75	1
Washer PS 6x12x1	7



## Fits:

Viking XL (Prod. No. 2040003), to S/N 801999  
Viking L (Prod. No. 2040004), to S/N 7200794  
Viking M (Prod. No. 2040005), to S/N 7106200

## Tools required:



2X 10 mm

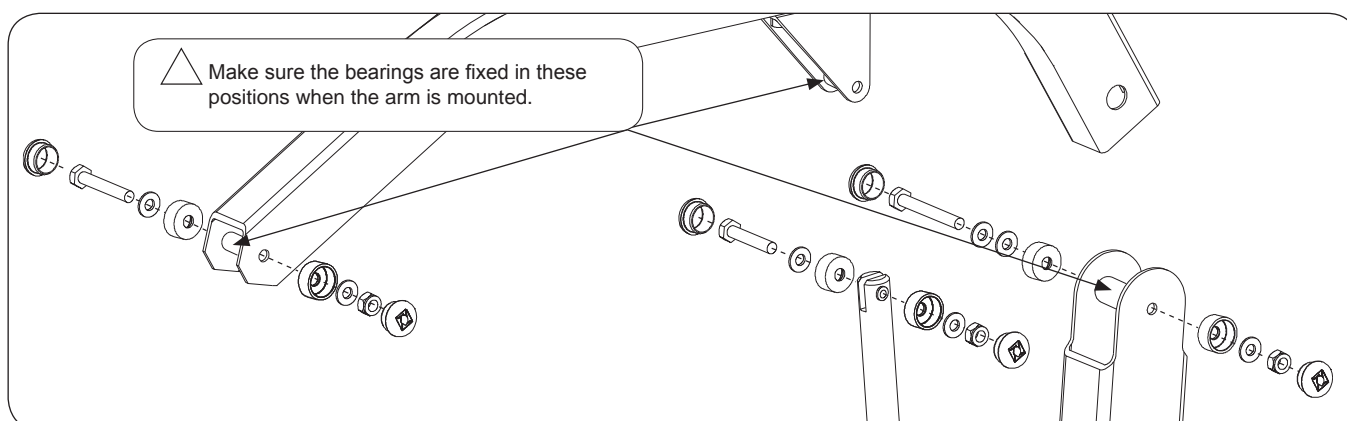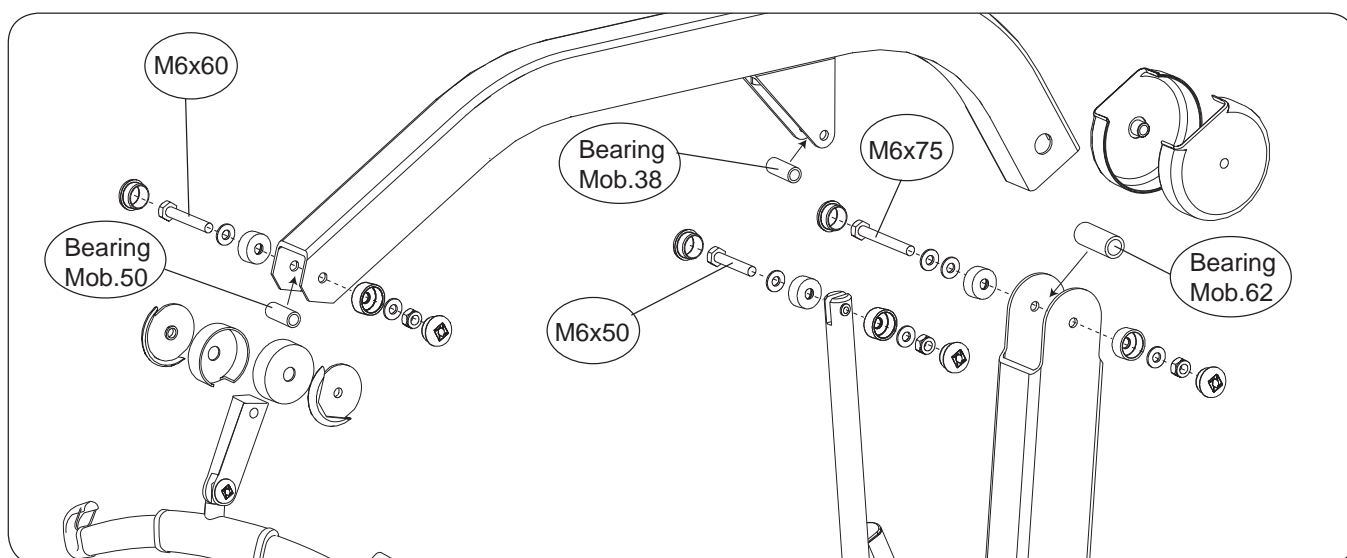

## NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490010


3SE630244-03  
2009-06-26

### Contains:

Lever M10 grey  
Lever M10x85 grey

### Qty:


1  
1

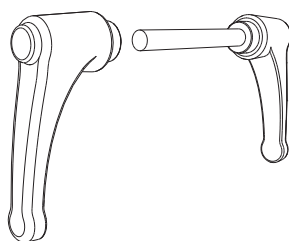

### Fits:

Viking XL  
Viking L  
Viking M  
Viking XS

Tools required:

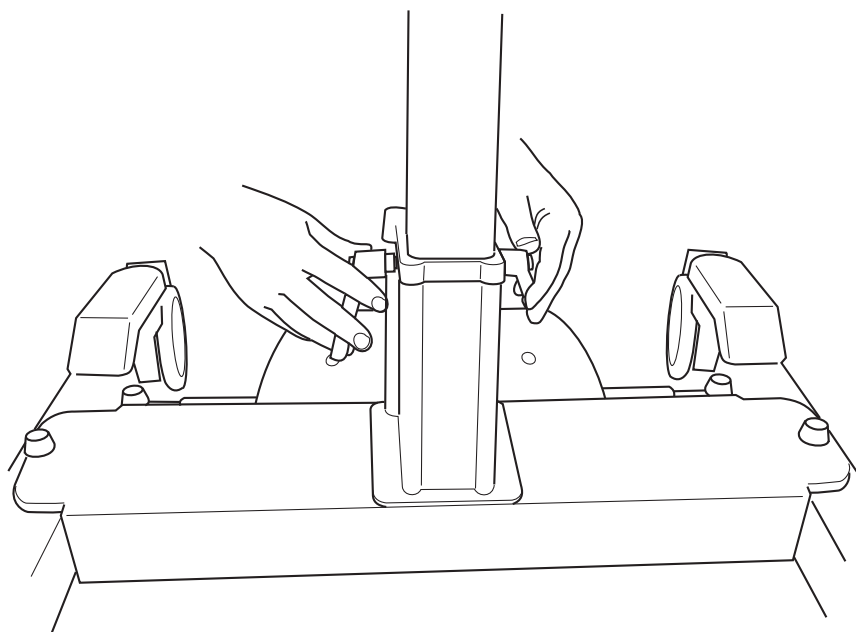

Place locking handle in lower hole and tighten.

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 21090011

3SE630218-06  
2011-05-02

### Contains:

Extension cable US

### Qty:

1

### Fits:

Golvo 7000ES/7007ES

Golvo 7007 LowBase

Uno 100 EM/EE

Uno 102 EM/EE/ES

Sabina EM/EE

Sabina II EM/EE

Viking XL Prod. No. 2040003


Viking L Prod. No. 2040004

Viking M Prod. No. 2040005


Length 26.5 cm/ 10.5 inch




1.

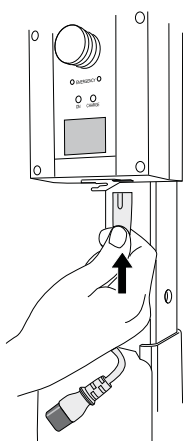

2.

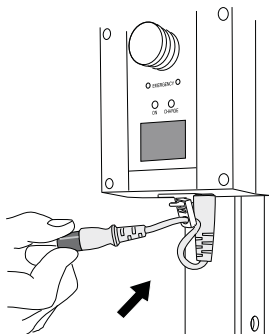

3.

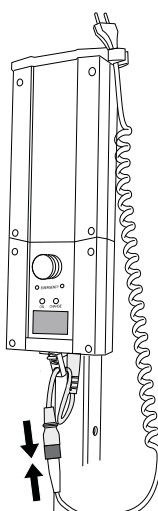

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

**Assembly Instruction  
Monteringsanvisning**

Prod. No. 31590011

3SE630229-04  
2011-05-25

**Contains:**

Latch

**Qty:**

2

**Fits:**

3156074 Universal SlingBar 350

3156075 Universal SlingBar 450

3156076 Universal SlingBar 600

3156077 Universal TwinBar 670

3156079 Universal SideBars 450

3156084 Universal SlingBar 350 with Quick-release Hook


3156085 Universal SlingBar 450 with Quick-release Hook

3156086 Universal SlingBar 600 with Quick-release Hook

3156087 Universal TwinBar 670 with Quick-release Hook

3156094 Universal SlingBar 350 R2R

3156095 Universal SlingBar 450 R2R

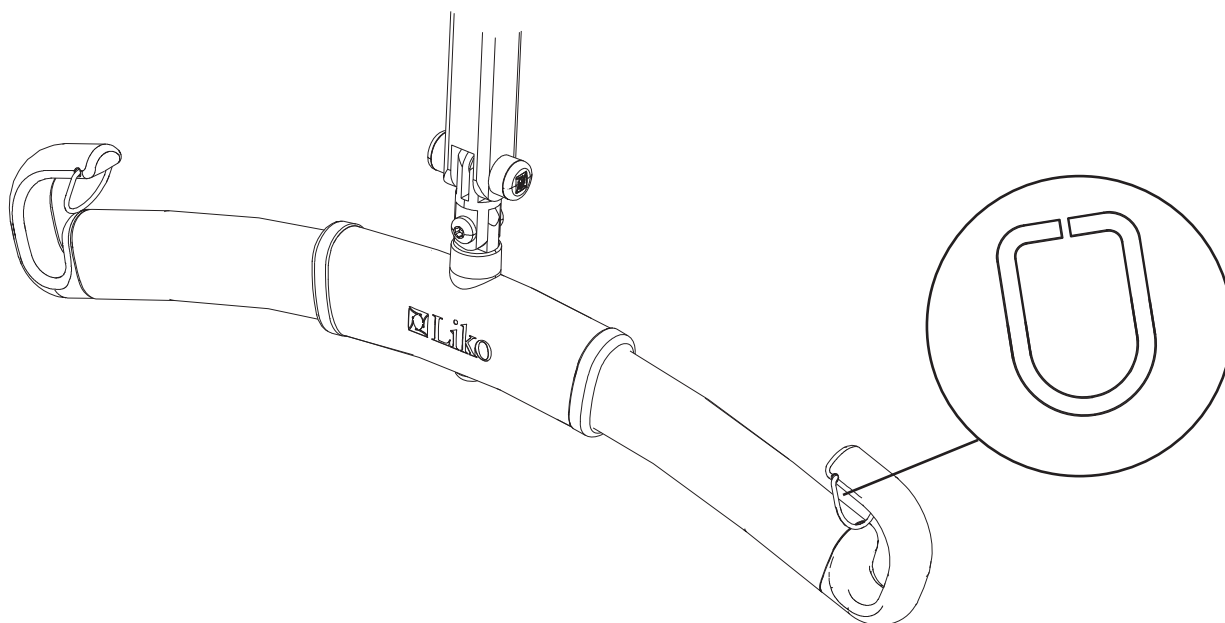
**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

# Assembly Instruction Monteringsanvisning

Prod No. 20490011

3SE630296-08  
2012-12-17

## Contains:

Actuator LA30 Viking L/M complete	Qty: 1
Motor cable Viking M Mini Fit	1


## Pos:

A	Slide bearing 10/12x6	2
B	Nylon bearing with collar	2
C	Cover Hat	4
D	Screw M10x40	2
E	Locking Nut M10	2


## Fits:

Viking M	2040005, from S/N 7107500*
Viking M	2040015
Viking M	2040035
Viking L	2040004, from S/N 7202000*
Viking L	2040014
Viking L	2040034

\*Fits Viking M/L with lower S/N, together with spare part 20490028;  
Rest. set Motor attachment.

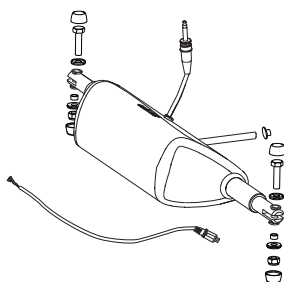

## Tools required:

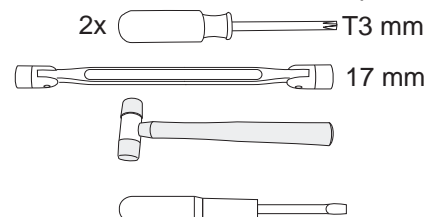


## Viking M 2040015 + 2040035 Viking L 2040014 + 2040034:


Assemble the enclosed motor cable.


Assembly of cable,  
see next page.


## Viking M 2040015 + 2040035 Viking L 2040014 + 2040034:

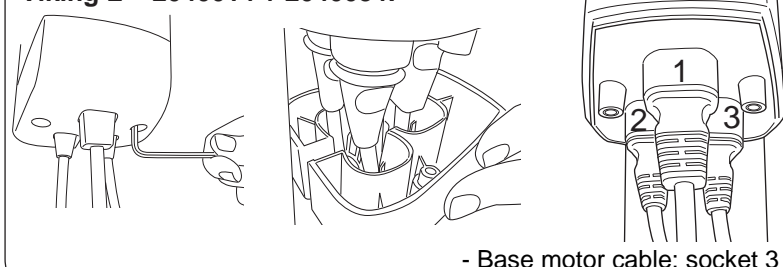


- Base motor cable; socket 3

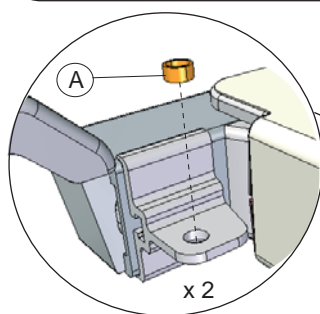


x 2

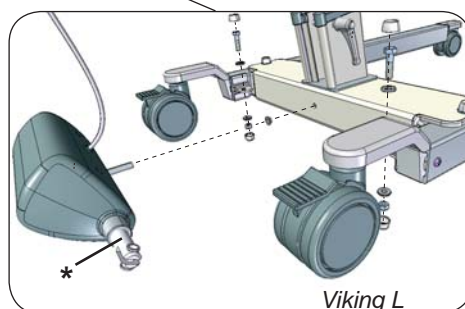

Viking L

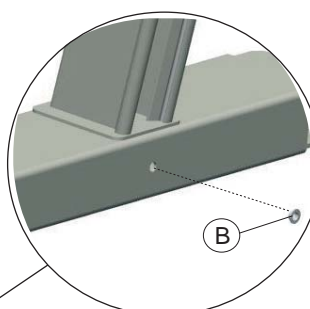

B

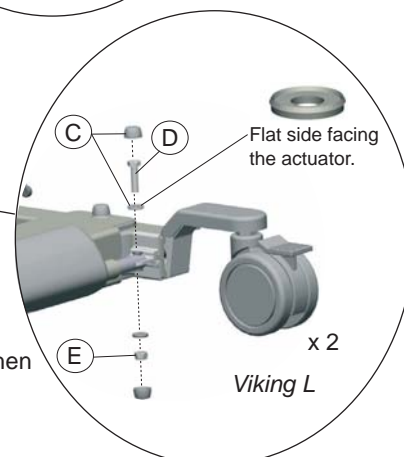

x 2

Viking L





- Unfasten the base motor by removing the bolt connections.
- Attach the slide bearings (A)
- Fasten the Nylon bearing with collar (B)
- Change the Base motor by pushing the motor axle in to the middle beam hole.  
Before mounting the Base Motor, unscrew the piston rods\* to its end position and then back until it fits in the brackets at the legs in closed position.
- Finish the assembly by mounting the bolt connections, assemble the cable and attach the cable cover.




Assembly Instruction  
Monteringsanvisning

Prod No. 20490011

## Assembly of cable for:

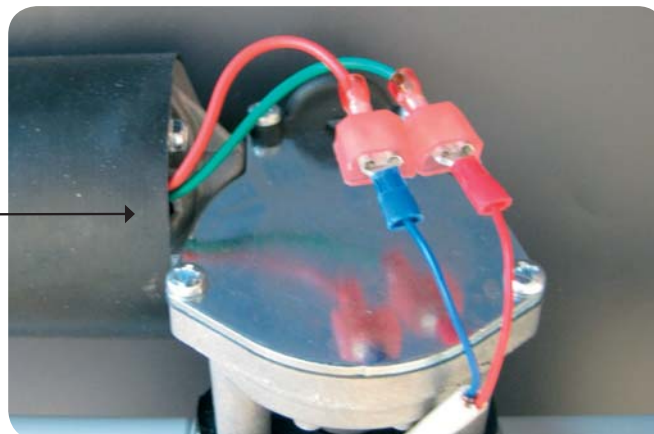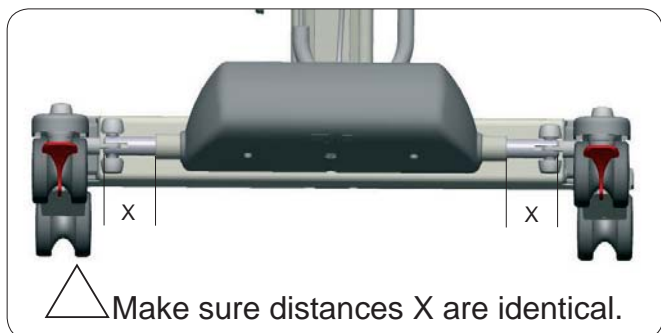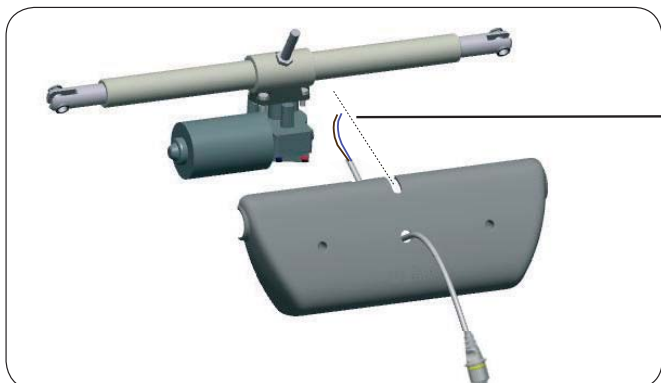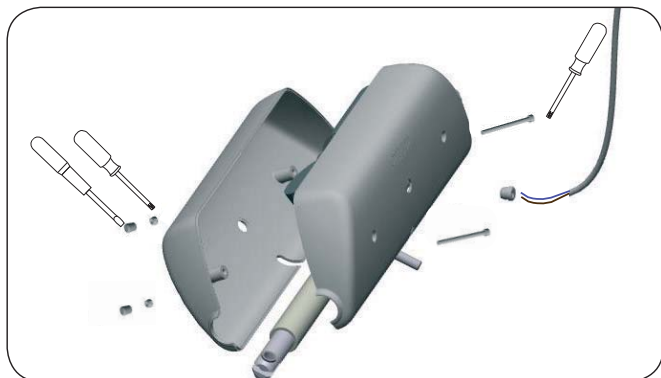Viking M  
Viking LProd No:  
2040015 and 2040035  
2040014 and 2040034Cable connection:

Red		Green	
Blue		Red	

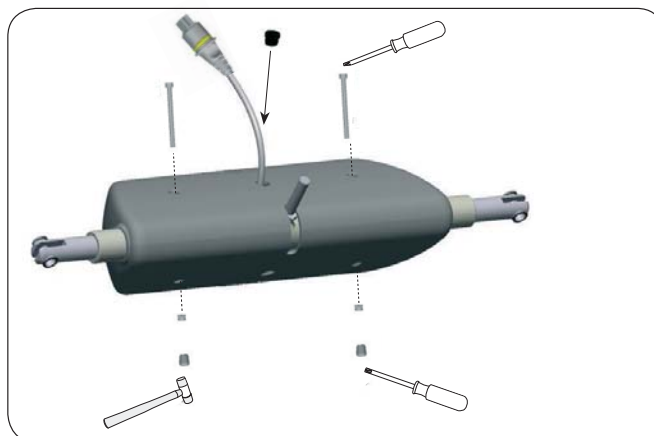

## △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490012

3SE630297-04

2010-08-02

**Contains:**

Actuator Viking M	1
Outer Tube	1
Locking nut M10 Nyloc	2
Locking nut M6 Nyloc	2



**Fits:**

Viking M (2040005)

**Tools required:**

2X  10mm

or

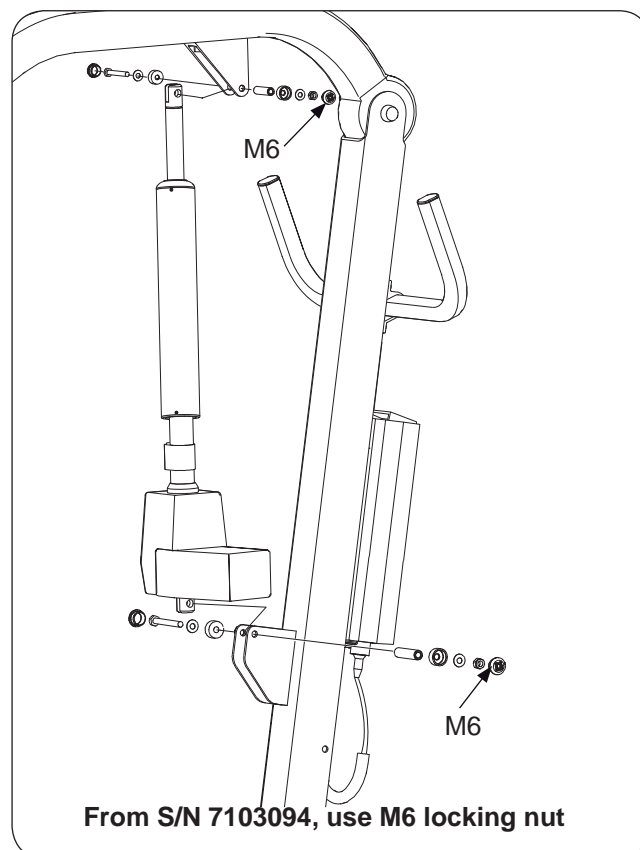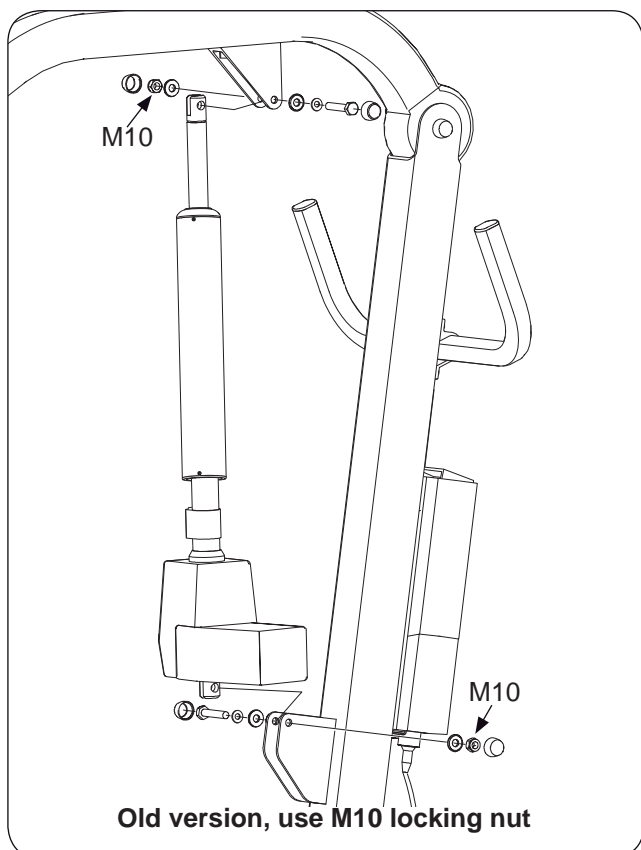2X  17 mm

**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

### For the North American market

Spare Part No:

20090022US, 20190014US, 20190076US, 20290011US, 20290024US, 20390007\*, 20490009US, 20490013US


3SE630223-10

2012-08-09

**Contains:** Qty:

Control Box 1

Extension cable, US 1

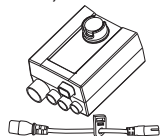

### For other markets:


Spare Part No:

20090022, 20190014, 20190076, 20290011, 20290024, 20390007\*, 20490009, 20490013


**Contains:** Qty:

Control Box 1

Extension cable 1


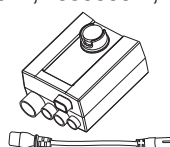


Part No.	Fits:
<b>20090022,</b> <b>20090022US</b>	Golvo 7000ES / 7007ES, 7007 LowBase
<b>20190014,</b> <b>20190014US</b>	Uno 102 ES
<b>20190076,</b> <b>20190076US</b>	Uno 100 EM/EE Uno 102 EM/EE
<b>20290011,</b> <b>20290011US</b>	Sabina EM/EE
<b>20290024,</b> <b>20290024US</b>	Sabina II EM/EE
<b>20390007</b>	LikoLight (*Spare part without extension cable)
<b>20490009,</b> <b>20490009US</b>	Viking XL (Prod. No. 2040003) Viking L (Prod. No. 2040004)
<b>20490013,</b> <b>20490013US</b>	Viking M (Prod. No. 2040005)

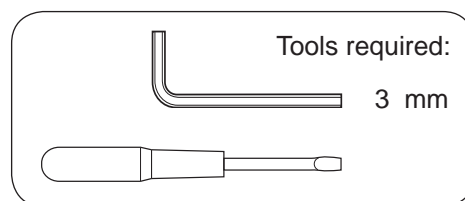

**NOTE:** Uno 100 EM  
Uno 102 EM  
Sabina EM  
Sabina II EM

Use plug from the replaced control box!

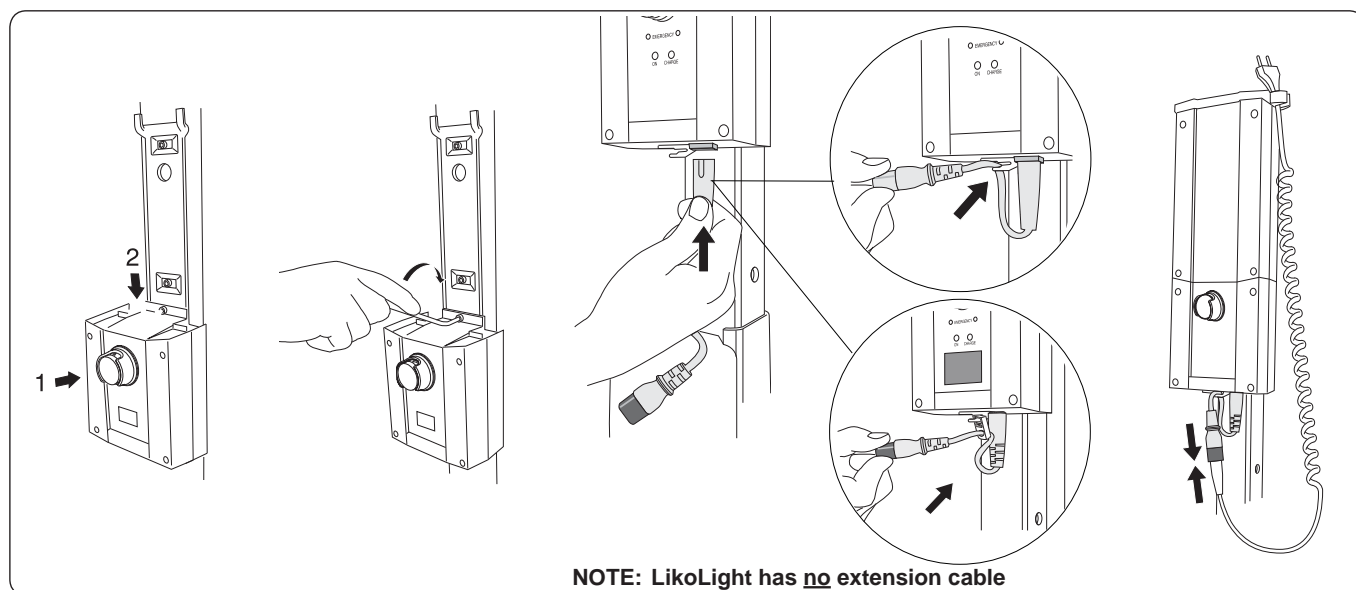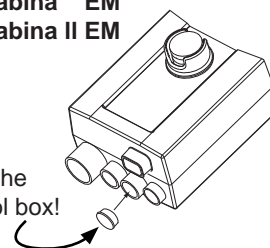

**NOTE:** LikoLight has no extension cable

### NOTE

- Make sure that the spare part is approved for the lift, and that it is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20390001, 20490014

3SE630220-05  
2010-05-20


**Contains:**  
Front wheel 75mm

**Qty:**  
1




**20390001 Fits:** LikoLight  
**20490014 Fits:** Golvo 7000 fr S/N 7024206  
Golvo 7007 fr S/N 7729722  
Golvo 7000ES Classic / 7007 ES Classic  
Golvo 7007 LowBase  
Golvo 8000/8008  
Golvo 8000 Classic / 8008 Classic  
LikoLight (from S/N 2503102)  
RollOn  
Viking M 20490005 (from S/N 7102064)  
Viking M 2040015  
Viking S  
Viking XS


Tools required:

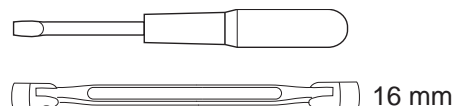


**A)**

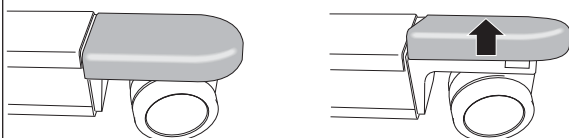

**B)**

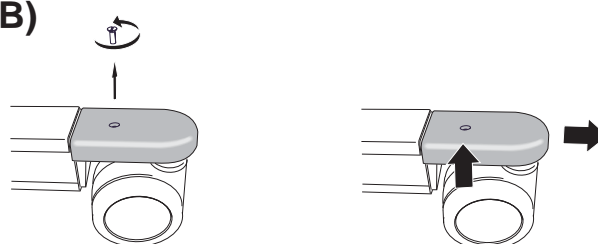

**A&B)**

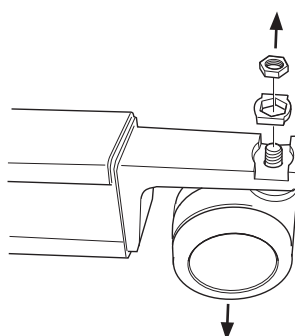

Mount the new spare part. Assemble in reverse order.

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.


## Assembly Instruction Monteringsanvisning

**Contains:**

Rear wheel 75mm

**Qty:**

1

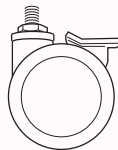
**20390002 Fits:** LikoLight up to S/N 2503101

**20490015 Fits:** Golvo 7000 fr S/N 7024206

Golvo 7007 fr S/N 7729722

Golvo 7000ES Classic / 7007 ES Classic

Golvo 7007 LowBase

Golvo 8000/8008 (all models)

LikoLight (from S/N 2503102)

RollOn

Viking M 20490005 (from S/N 7102064)

Viking M 2040015

Viking S

Viking XS

Prod. No. 20390002

20490015




3SE630221-05

2010-05-20

Tools required:



16 mm

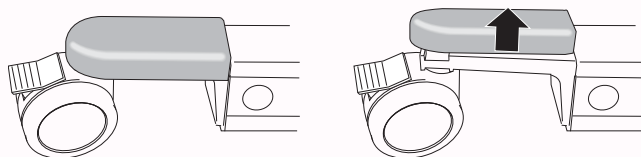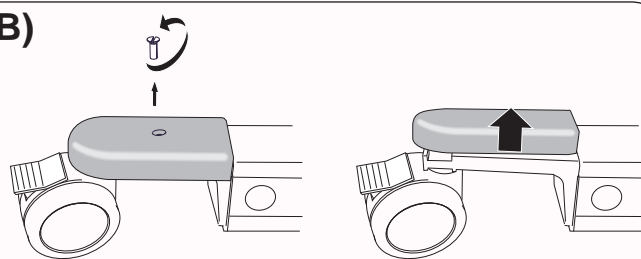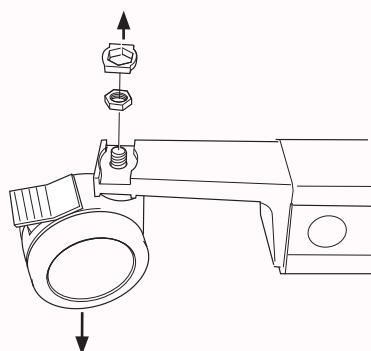**A)**

**B)**

**A&B)**


Mount the new spare part. Assemble in reverse order.

**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20090058


3SE630210-06  
2012-10-16

**Contains:**

Hook for Hand control 2,4,6

**Qty:**

1

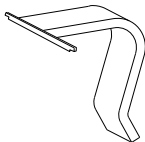



**Fits:**

20090089 HandControl 6-button  
20190016 HandControl EE 4-button  
20190017 HandControl EM 2 button

**Tools required:**



T10

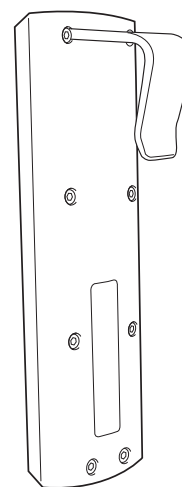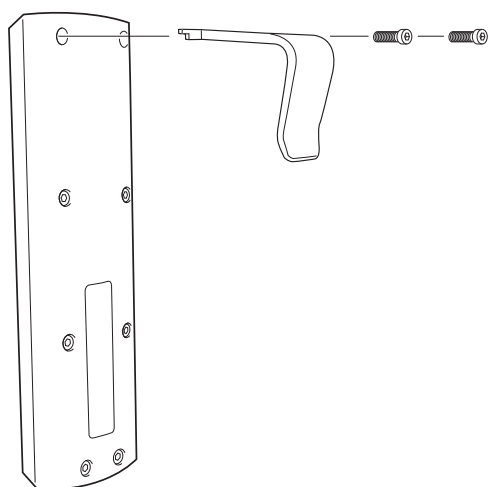

**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.


## Assembly Instruction Monteringsanvisning

Prod. No. 20490023

3SE630250-07  
2012-08-22

**Contains:**


	Qty:
A Cover hat Liko	2
B Screw M6S M6x75	1
C Washer PS 6x12x1	3
D Cover hat Liko bottom	2
E Flanged washer	2
F Distance washer PS 12x18x1	2
G Plastic cover mast Vi XS	2
H Sleeve bearing Mob.62	1
I Locking nut M6 Nyloc	1

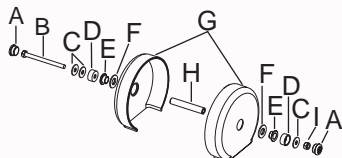
**Tools required:**

2x  10 mm

**Fits:**
**Prod. No.**

Viking XL	2040003 from S/N 802000
Viking XL	2040013
Viking XL	2040033
Viking L	2040004 from S/N 7200795
Viking L	2040014
Viking L	2040034
Viking M	2040005 from S/N 7106201
Viking M	2040015
Viking M	2040035
Viking XS	2040007

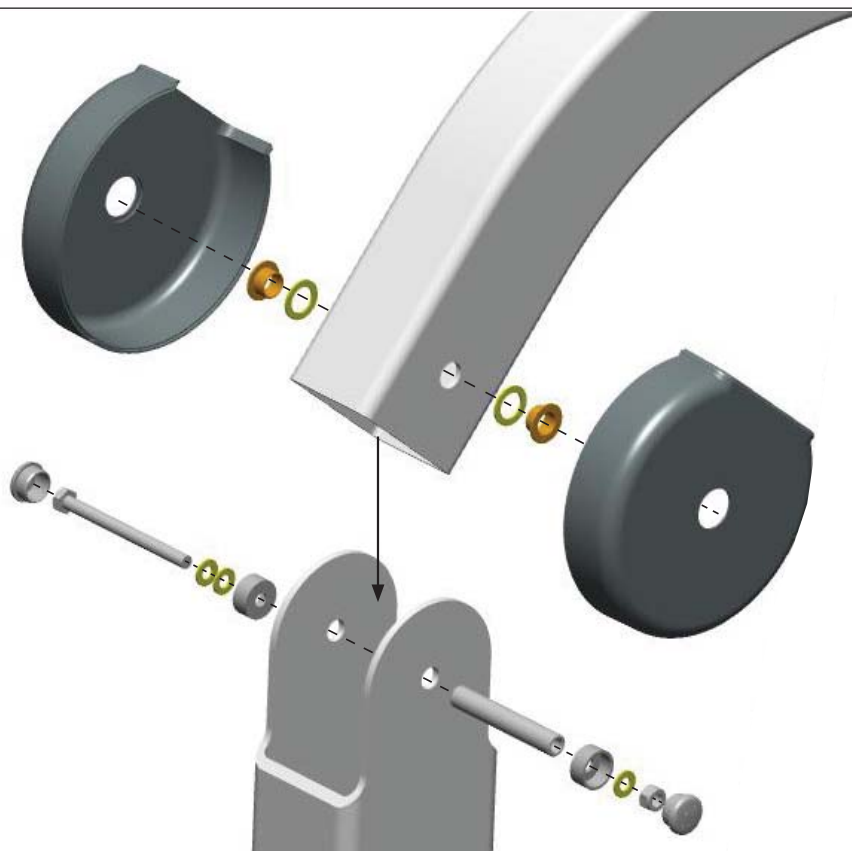
**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction



### Monteringsanvisning

Prod. No. 20490024


3SE630251-08  
2012-09-10

Pos	Contains:	Qty:
A	Cap Y Viking	2
B	Cap I Viking	2
C	Cover hat Liko	6
D	Cover hat Liko bottom	6
E	Sleeve bearing Mob.38	1
F	Sleeve bearing Mob.50	1
G	Sleeve bearing Mob.62	1
H	Plastic cover mast Vi XS	2
I	Nylon bearing 10/12x15	2
J	Locking nut M6 Nyloc	3
K	Distance washer PS 12x18x1	2
L	Screw M6S M6x50	1
M	Screw M6S M6x60	1
N	Screw M6S M6x75	1
O	Washer PS 6x12x1	6
P	Flanged washer	2

Fits:	Prod. No.
Viking XL	2040003 from S/N 802000
Viking XL	2040013
Viking XL	2040033
Viking L	2040004 from S/N 7200795
Viking L	2040014
Viking L	2040034
Viking M	2040005 from S/N 7106201
Viking M	2040015
Viking M	2040035
Viking XS	2040007

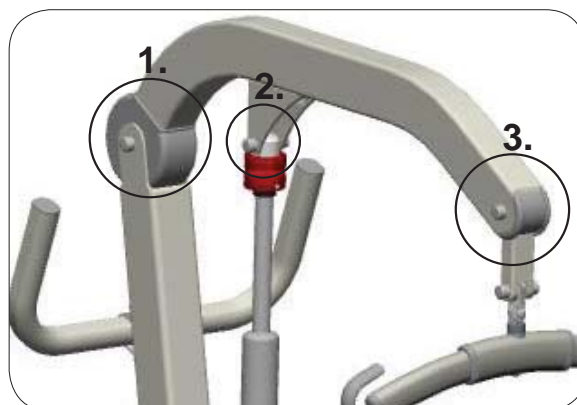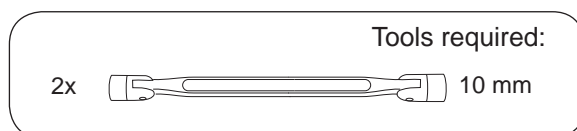

Viking XS



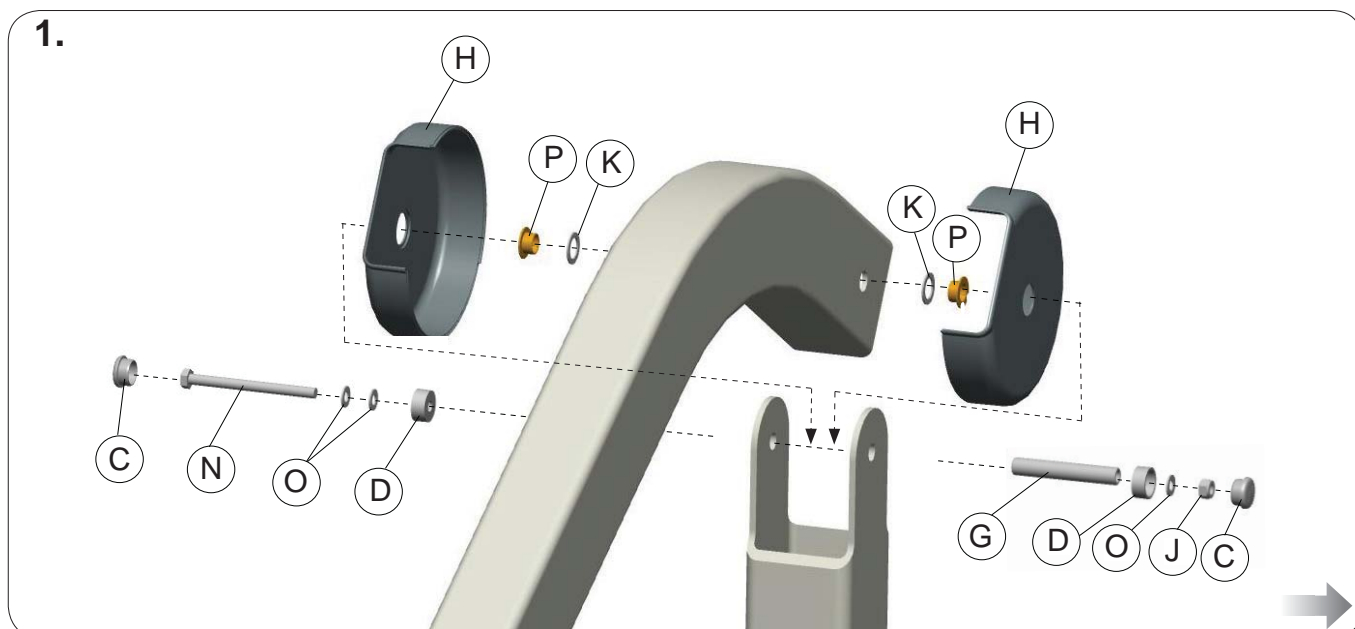




Assembly Instruction  
Monteringsanvisning

Prod. No. 20490024

Pos	Contains:	Qty:
A	Cap Y Viking	2
B	Cap I Viking	2
C	Cover hat Liko	6
D	Cover hat Liko bottom	6
E	Sleeve bearing Mob.38	1
F	Sleeve bearing Mob.50	1
G	Sleeve bearing Mob.62	1
H	Plastic cover mast Vi XS	2
I	Nylon bearing 10/12x15	2
J	Locking nut M6 Nyloc	3
K	Distance washer PS 12x18x1	2
L	Screw M6S M6x50	1
M	Screw M6S M6x60	1
N	Screw M6S M6x75	1
O	Washer PS 6x12x1	6
P	Flanged washer	2

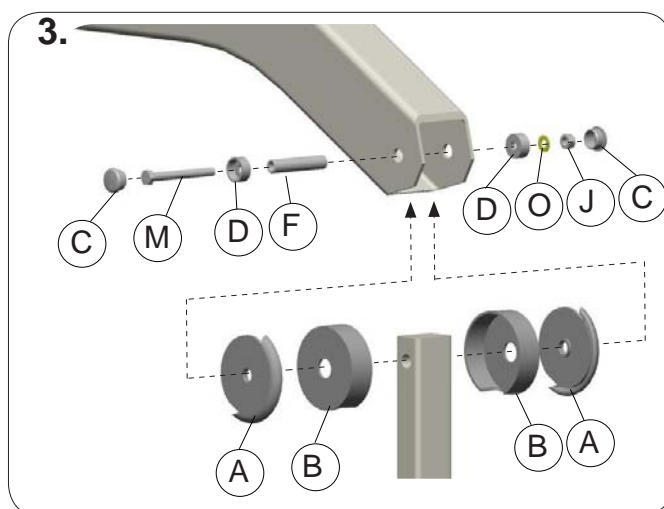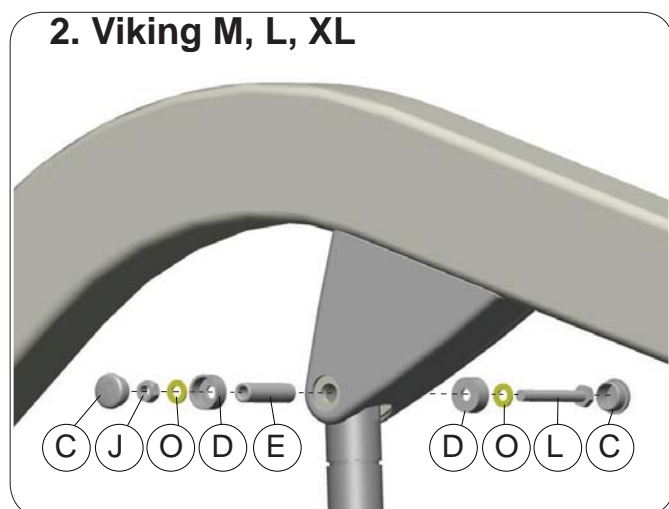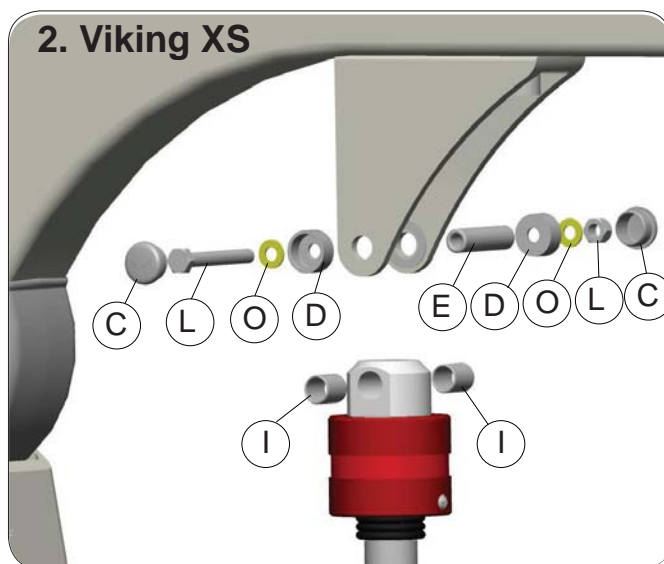

Viking XS



Viking M



Viking L



Viking XL

## NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 31590014

3SE640242-05  
2011-02-22

**Contains:**  
Latch composite



**Qty:**  
2



### Fits:

		<b>From S/N:</b>
3156074	Universal SlingBar 350	1299511
3156075	Universal SlingBar 450	1299511
3156076	Universal SlingBar 600	1299511
3156077	Universal TwinBar 670	1299511
3156079	Universal SideBars 450	1299511
3156084	Universal SlingBar 350 with Quick-release Hook	1299511
3156085	Universal SlingBar 450 with Quick-release Hook	1299511
3156086	Universal SlingBar 600 with Quick-release Hook	1299511
3156087	Universal TwinBar 670 with Quick-release Hook	1299511
3156094	Universal SlingBar 350 R2R	1299511
3156095	Universal SlingBar 450 R2R	1299511
20590006	SlingBar Liko 450	1700101

### From S/N:

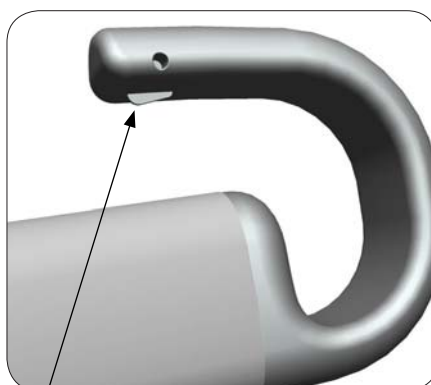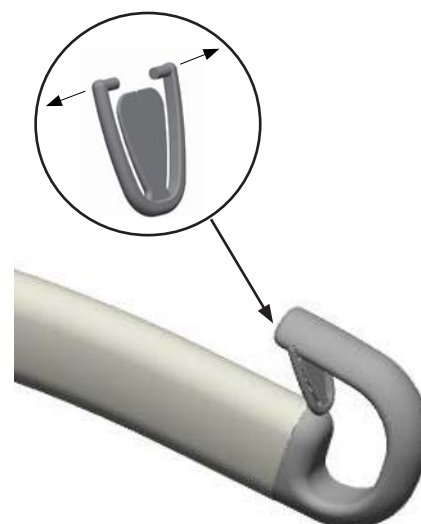

The Latch can only be used on composite hooks equipped with a locator.



### NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490028





3SE640258-05  
2010-04-21

**Contains:**


	Qty:
Link attachment Viking M Alu	2
Distance Leg 12/9	2
Screw M6S M10x40 GL=15mm	2
Nylon bearing 10x12x6	2
Locking nut M10 nyloc	2
Screw K6S M8x60	2
Nut M6LM M8	2
Coverhat M10 grey	4

**Fits:**


Viking M

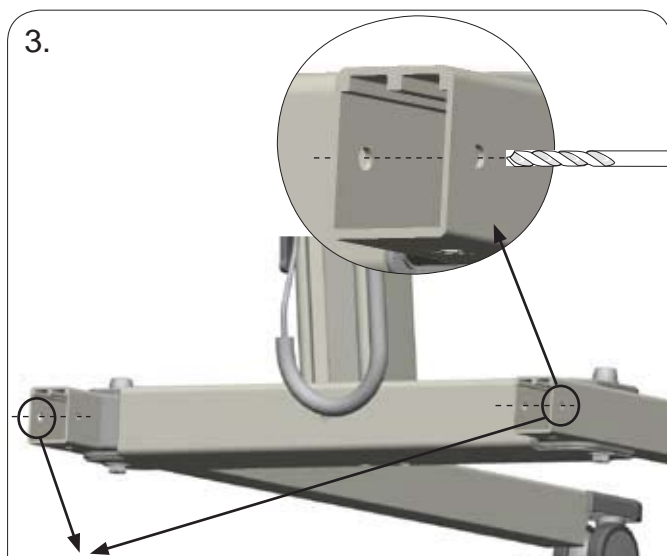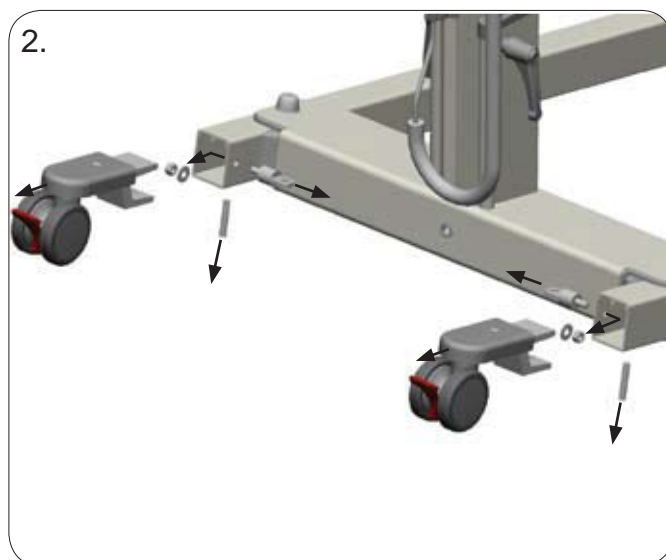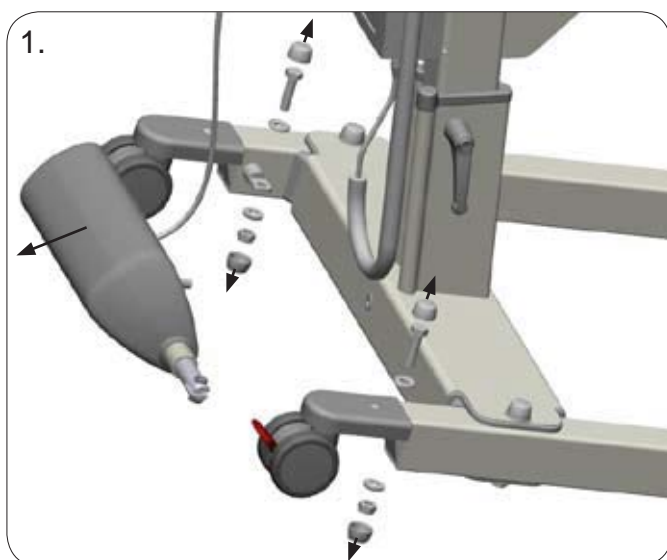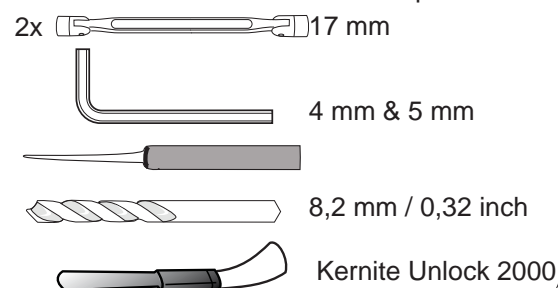**Tools required:**


Specify center of existing hole. Mark the same distance on opposite side with a centre punch. Drill one hole, Ø 8.2 mm / 0,32 inch on each leg.

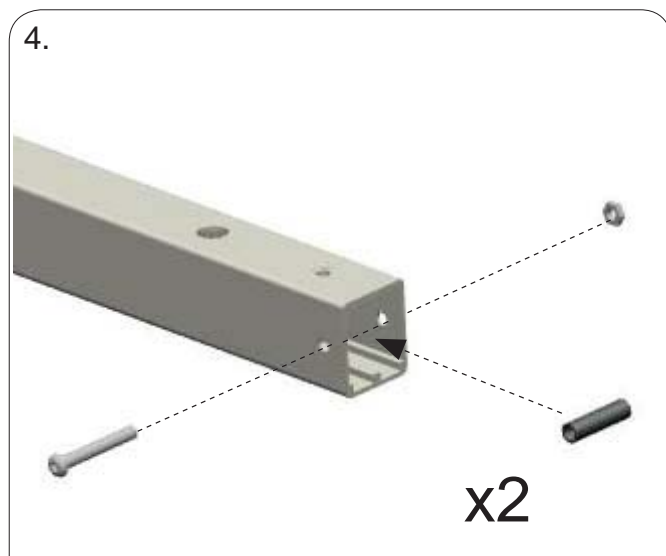


5.

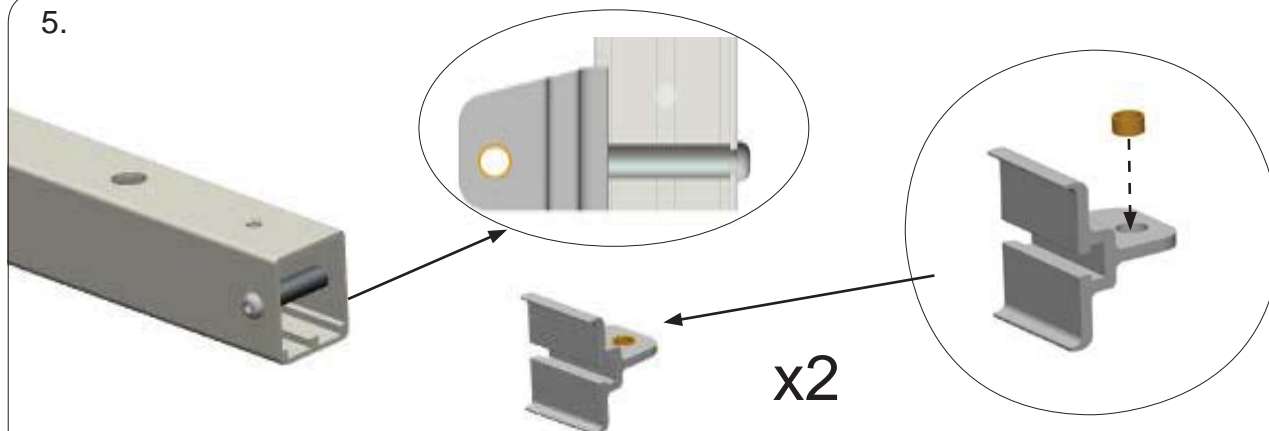


The link attachment (together with Nylon bearing 10x12x6) is assembled so that the nut fits into the groove, in line with the leg. Then - tighten the screw.

6.

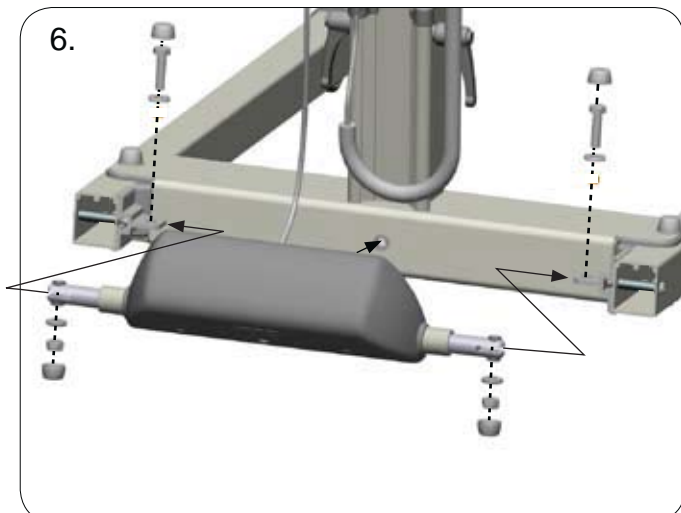


7.

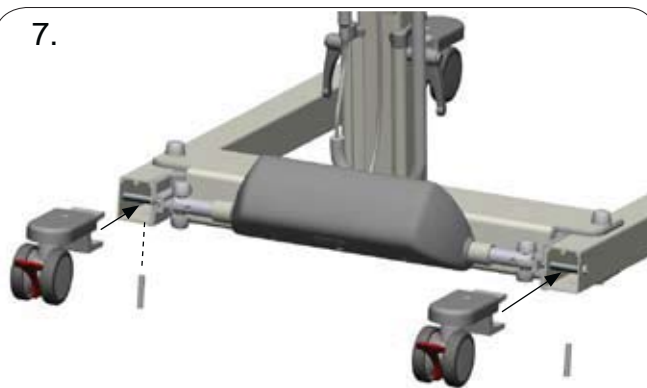

Apply Kernite Unlock 2000 on each screw tip.

8.

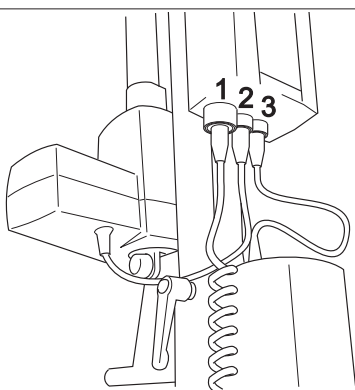

Connect the cables as follows:

- Handcontrol cable to socket No.1
- Cable for lift arm motor to socket No.2
- Base motor cable to socket No.3

9.



# NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490030

3SE640260-05  
2011-03-03

### Contains:


	Qty:
a Top cover	1
b Screw P6SS M4x5	6
c Tube	1
d Snap bushing	2
e Two sided adhesive	2
f Bushing	1
g Compression spring	1

### Fits:


Viking M

Viking L Prod. No. 2040004







Viking XL Prod. No. 2040003

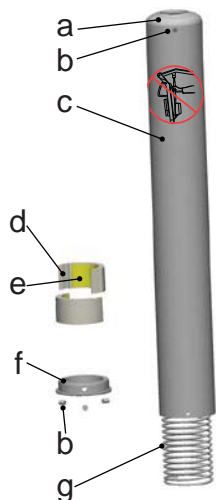








### Tools required:

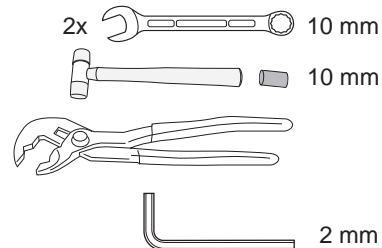

Isopropyl alcohol and water + rag

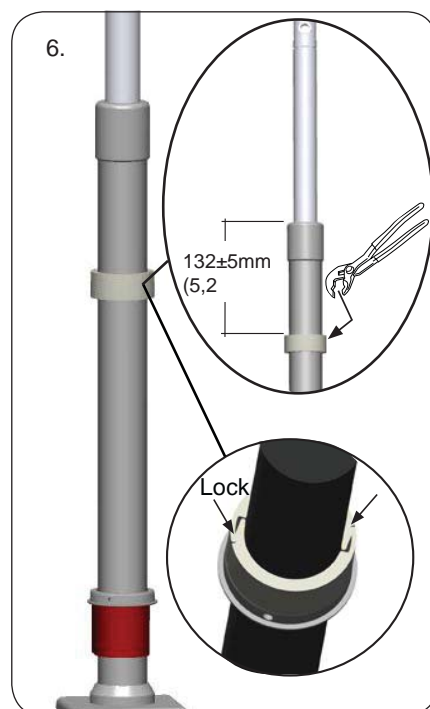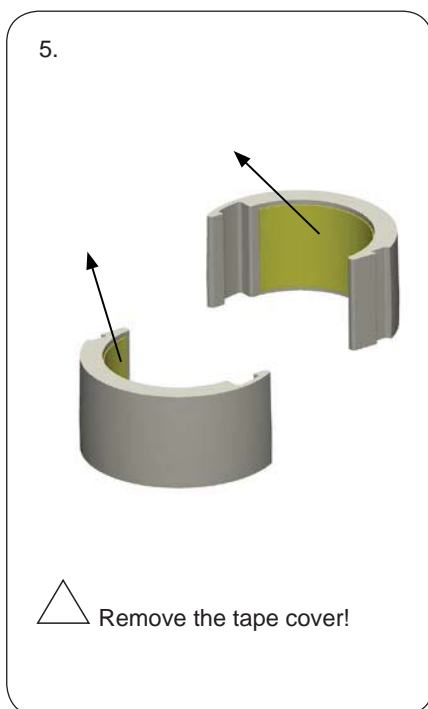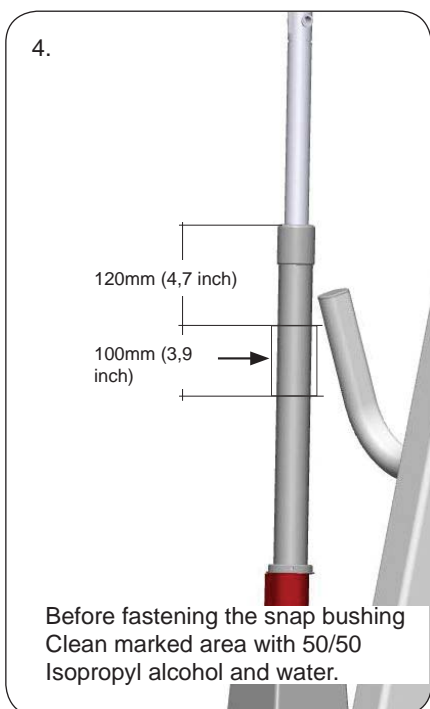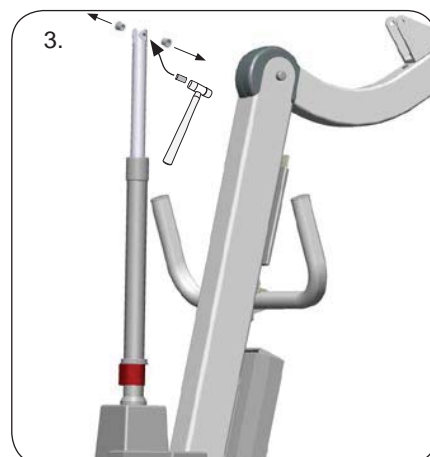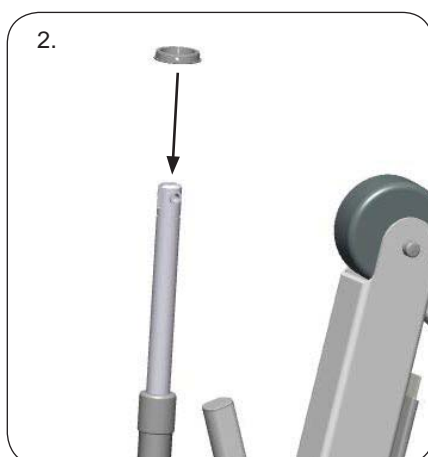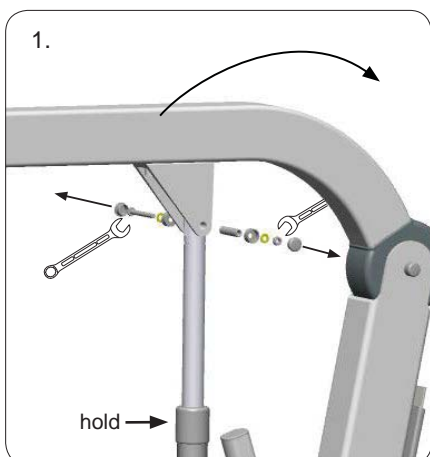

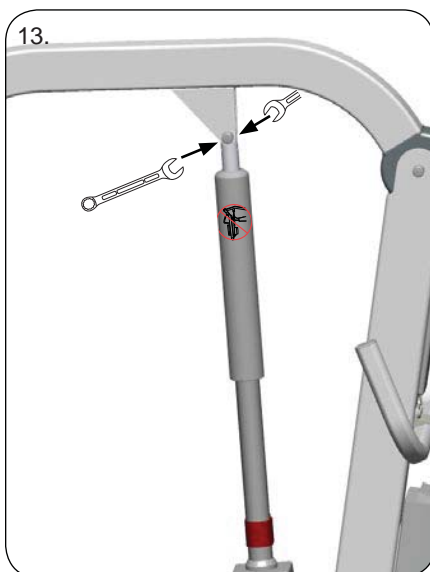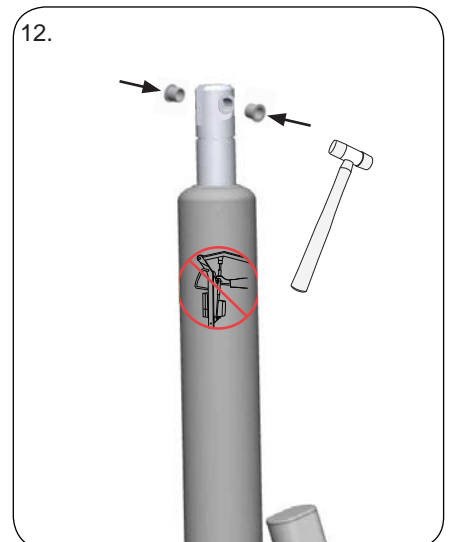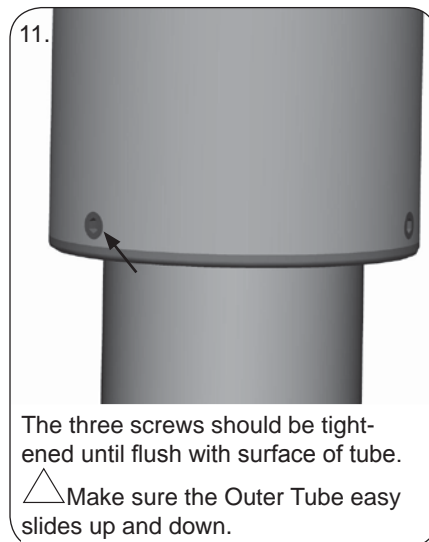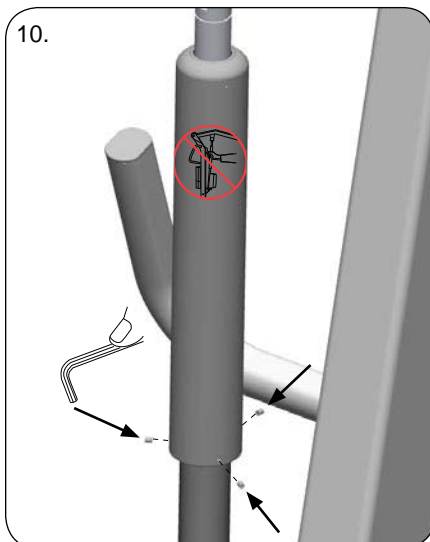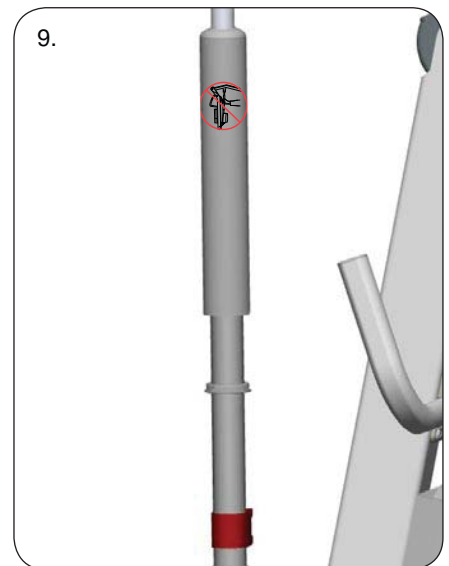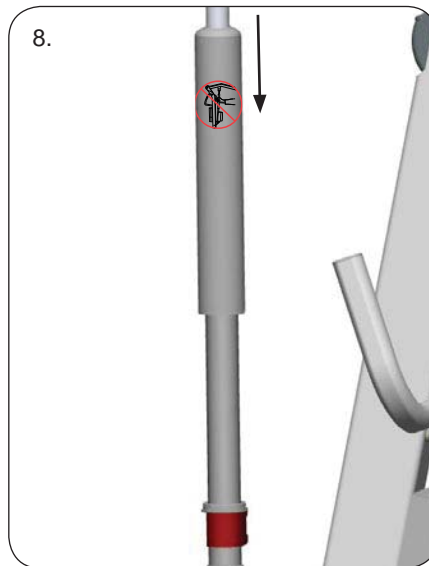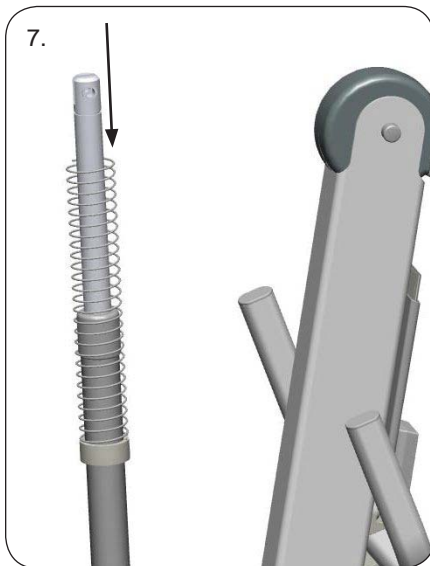


**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490031

3SE640298-04  
2011-03-03

**Contains:**


Handcontrol HBL20	1
Clips with hook HBL20	1
Charge Cable Hook	2

**Fits:**

Golvo 8000 / 8008	
Viking M	Prod. No. 2040015
Viking L	Prod. No. 2040014
Viking XL	Prod. No. 2040013

**Tools required:**



1.

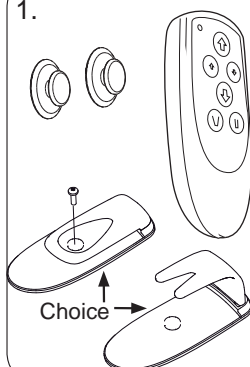


2.




3. Golvo

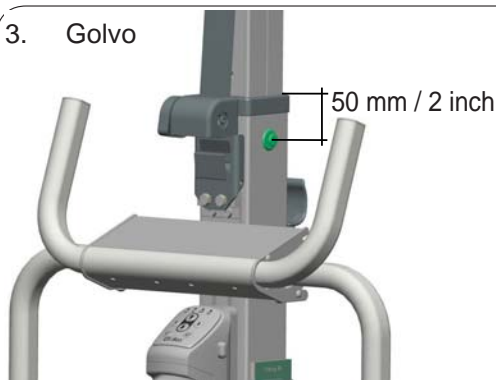

Viking M, Viking L, Viking XL

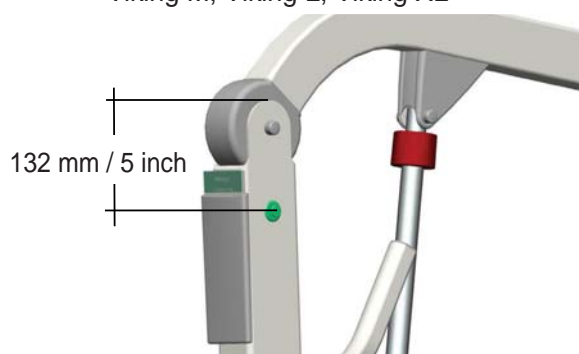


4.

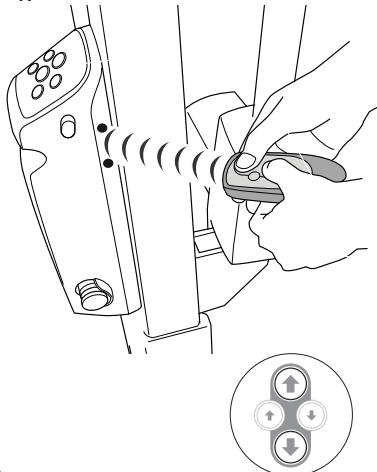

**A.** To activate IR control: Hold the hand control IR about 20 cm / 8 inch. from the control unit's IR receiver and press the large UP and DOWN buttons on the hand control simultaneously for about 2 seconds. A signal is heard when the function is activated. The lift is now ready to use with Handcontrol IR.

**B.** Check to determine if other lifts in the room are affected by your hand control. If other lifts are affected by your hand control: Use the large UP and DOWN buttons on the operation panel of the lift that is to be used. Press both buttons simultaneously until a signal sounds. The lift has now been assigned a unique communication code. Now, activate IR control according to procedure A, above.

5.

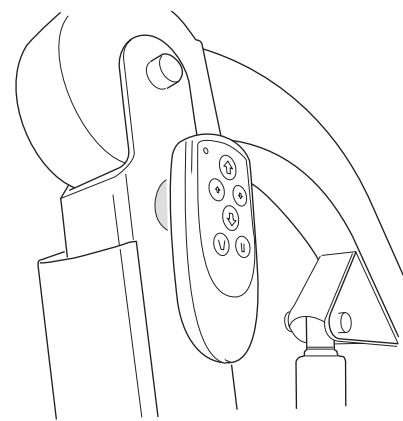
**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

# Assembly Instruction

## Monteringsanvisning

Prod. No. 20490032

3SE640299-04  
2012-10-25

**Contains:**

Control Unit CBL20  
Cable cover  
Screw cable cover  
Allen Key 2,5 mm  
Screw M5x35

**Qty:**

1  
1  
2  
1  
2

**Fits:**

Viking M

**Prod. No.**

2040015

**Requires:**

Liko Service System  
(for programming)

**Prod. No.**

21090024

Tools required:





2,5 &amp; 3 mm


**1.**

**Before use!**





**Control Unit CBL Vi M must be programmed with Liko Service System, Prod. No. 21090024 according to the Instruction Guide for the Service System.**

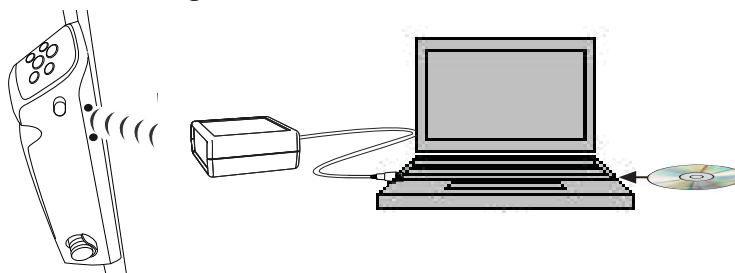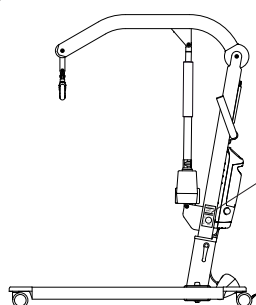
**2.**


S/N:

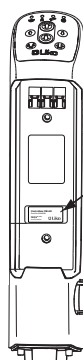




**With permanent marker, fill in the S/N at the product label (same as on the Viking M)**

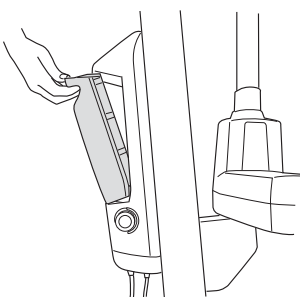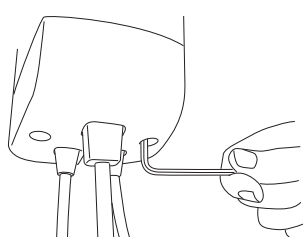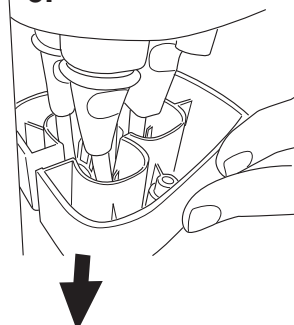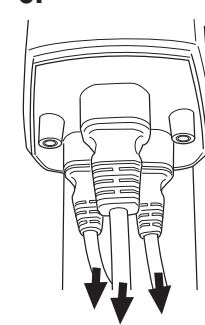**3.**

**4.**

**5.**

**6.**

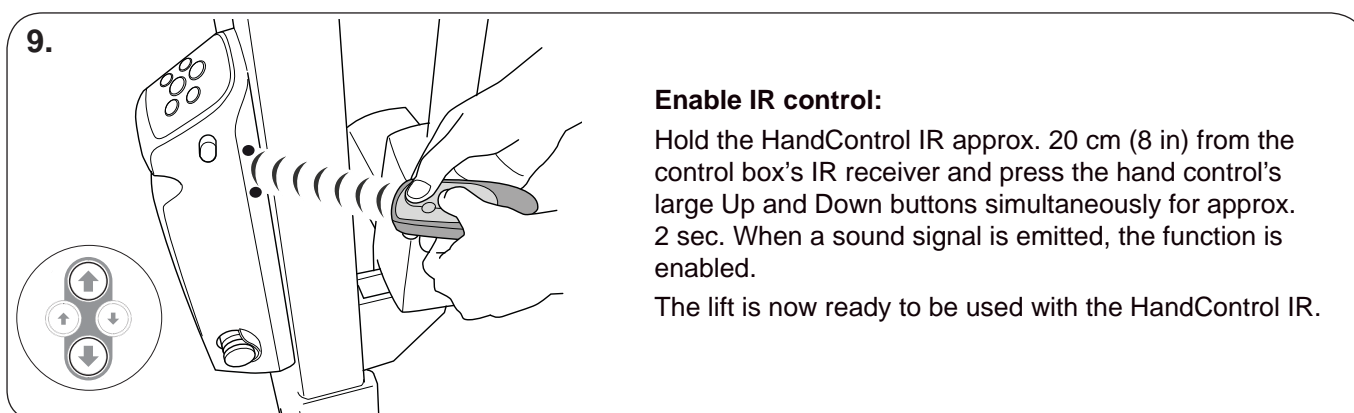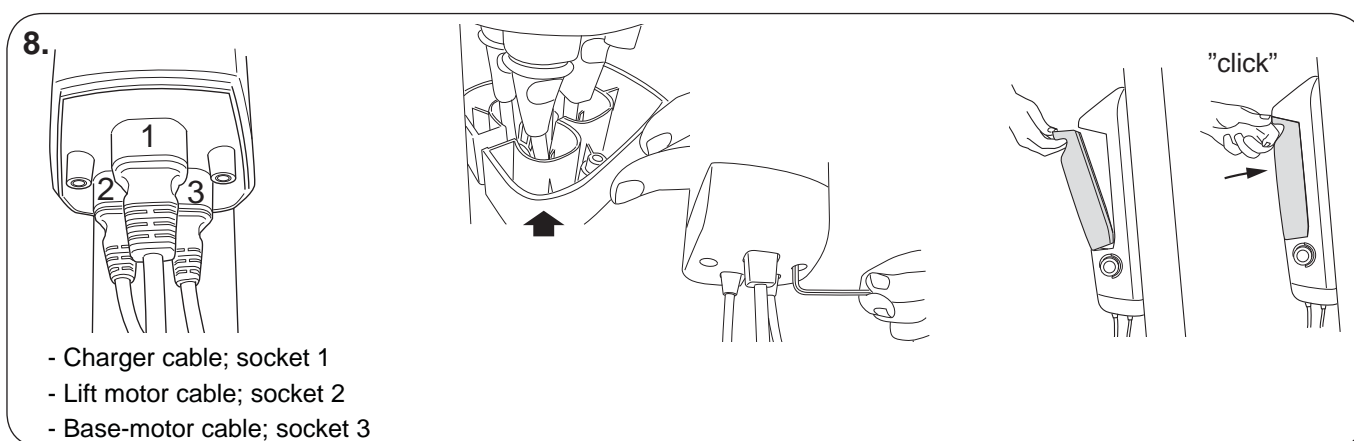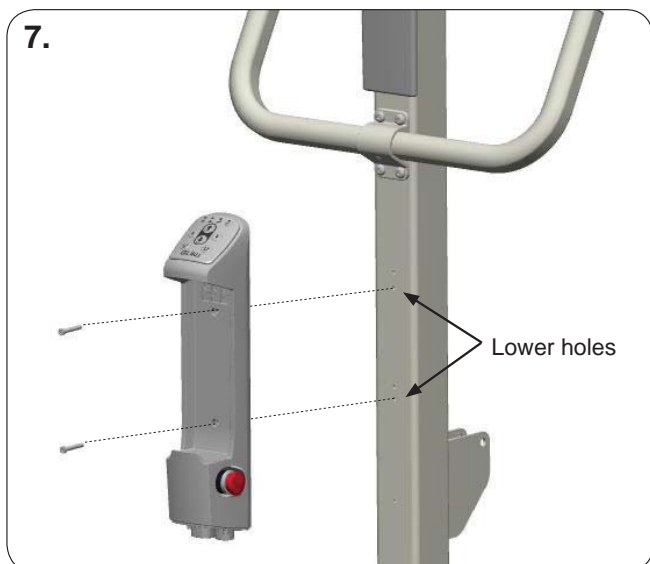


## Assembly Instruction Monteringsanvisning

Prod. No. 20490032



**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning


Prod. No. 20490053

3SE690226-01  
2012-10-25

**Contains:**

Control Unit CBL20 with cable  
Cable cover  
Screw cable cover  
Screw M5x35  
Allen Key 2,5 mm

**Qty:**

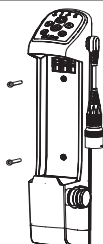1  
1  
2  
2  
1

**Fits:**

Viking M

**Prod. No.**  
2040035

**Requires:**

Liko Service System  
(for programming)

**Prod. No.**  
21090024


**Tools required:**


2,5 &amp; 3 mm




### 1. Before use!

**Control Unit CBL Vi M must be programmed with Liko Service System,  
Prod. No. 21090024 according to the Instruction Guide for the Service System.**

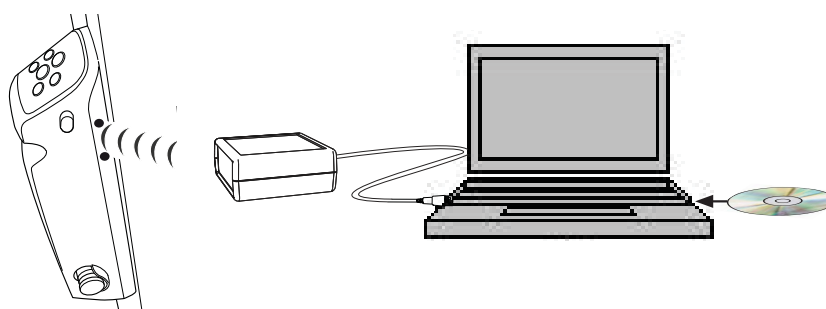

### 2.

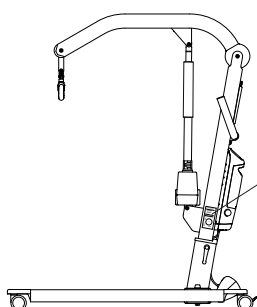


S/N:




**With permanent marker,  
fill in the S/N at the  
product label  
(same as on the Viking M)**


### 3.

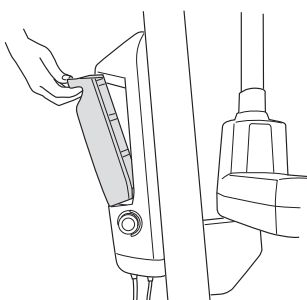


### 4.

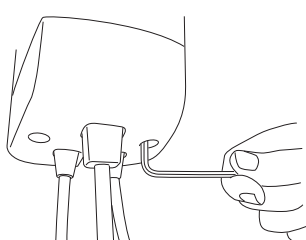

### 5.



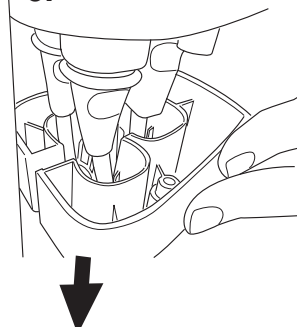


### 6.

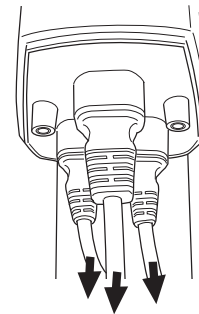

## Assembly Instruction Monteringsanvisning

Prod. No. 20490053

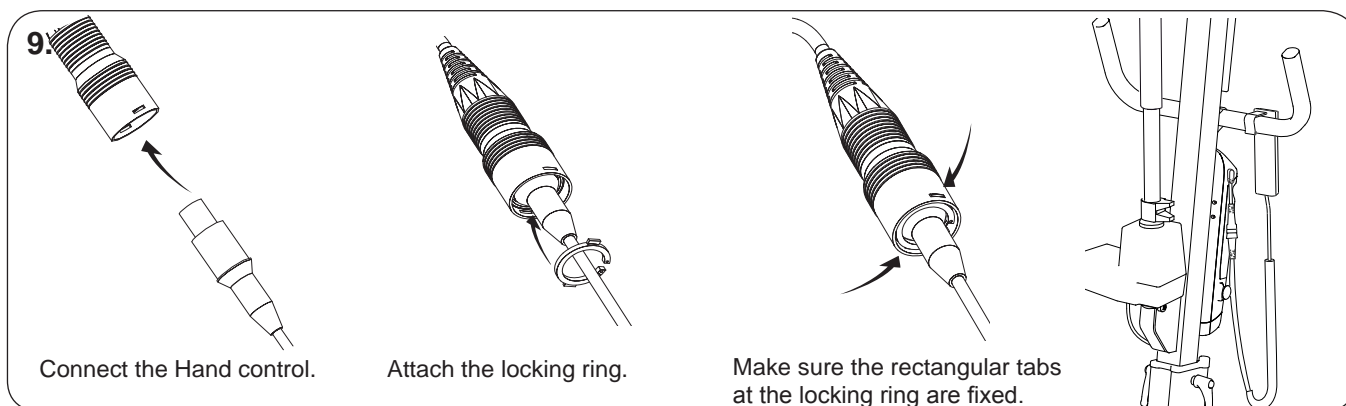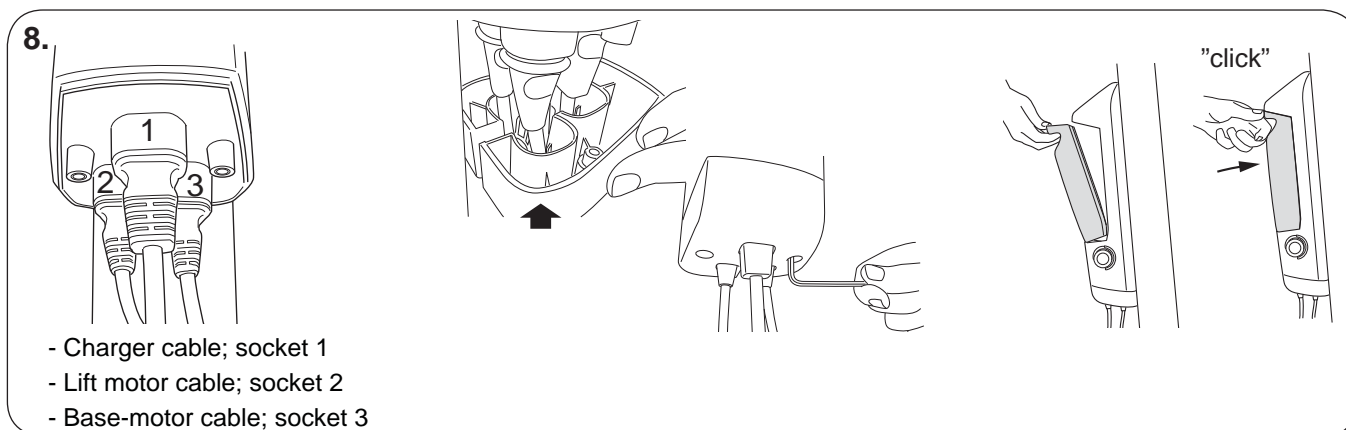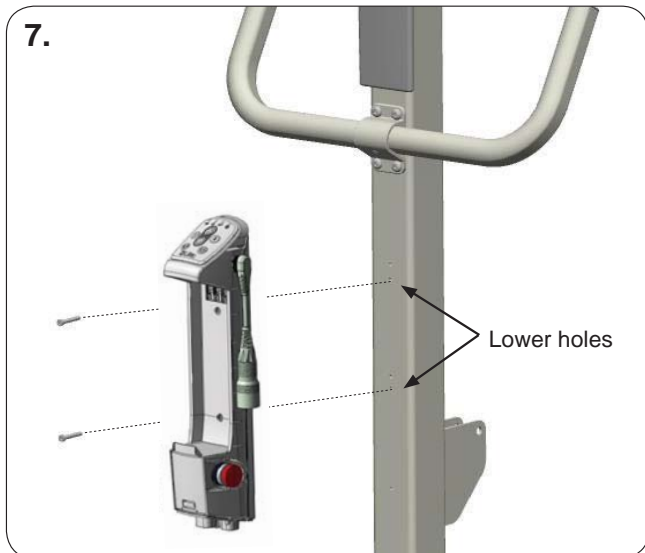

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490039

**To order this Spare Part:**

3SE650270-03

**S/N from the Viking M**

2012-10-24



**the Control Unit is intended for,  
is required!**
**Contains:**

Control Unit CBL20	1 (pre-configured)
Cable cover	1
Screw cable cover	2
Screw M5x35	2 (Viking)
Screw M5x50	2 (Golvo)
Allen Key 2,5 mm	1


**Fits:**

Viking XL	2040013
Viking L	2040014
Viking M	2040015
Golvo 8008 LowBase	2000019
Golvo 8000	2000014
Golvo 8008	2000015
Golvo 8000 Classic	2000024
Golvo 8008 Classic	2000025








**Qty:**

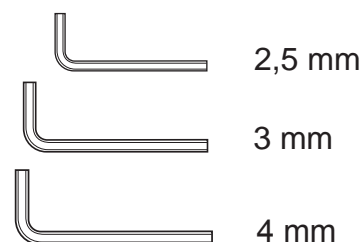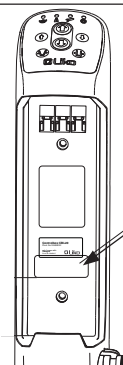**Tools required:**

**1.**

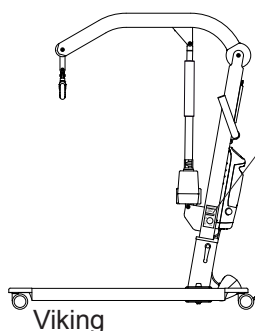
**Before use!**  
Make sure the  
S/N on the label  
on Control  
Unit CBL is the  
same as on the  
lift.

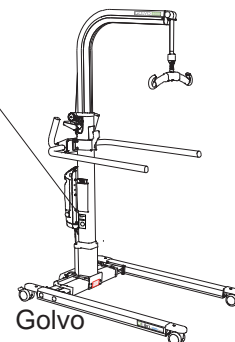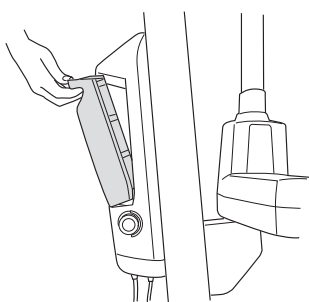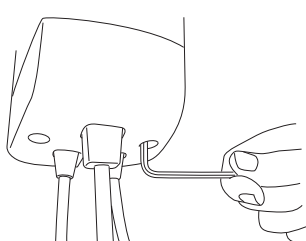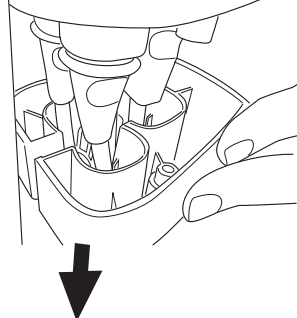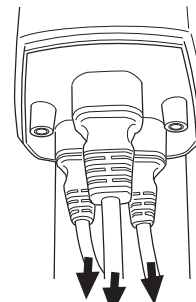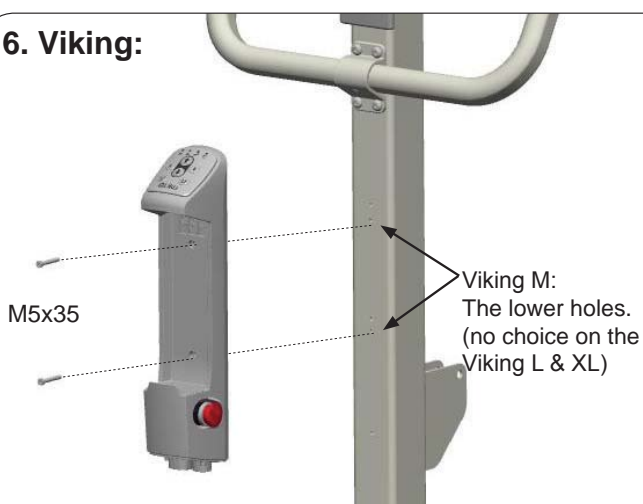
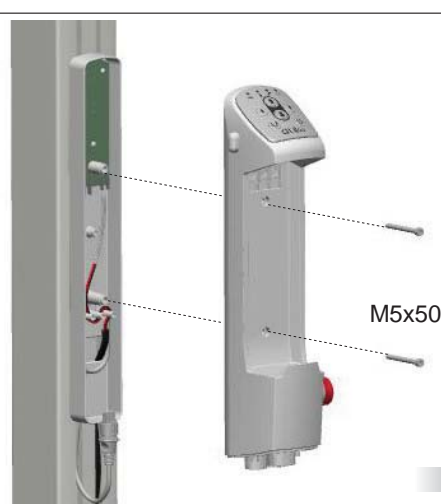


S/N:


**2.**

**3.**

**4.**

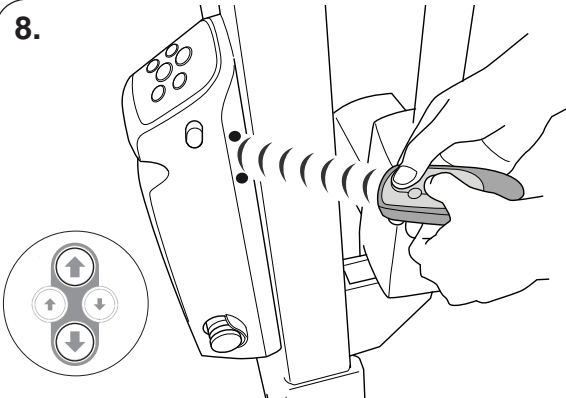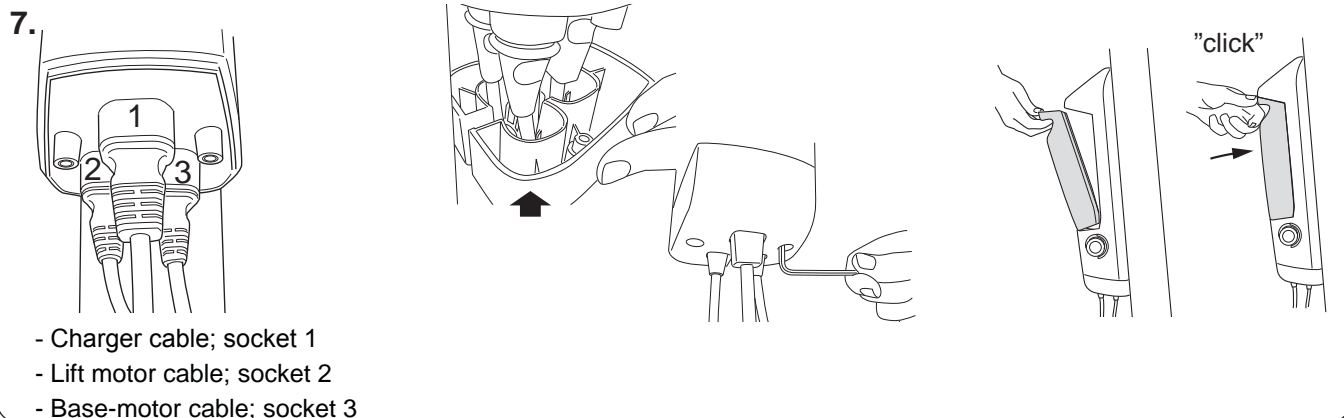**5.**

**6. Viking:**

**Golvo:**


## Assembly Instruction Monteringsanvisning

Prod. No. 20490039



**A.** To activate IR control: Hold the hand control IR about 20 cm / 8 inch. from the control unit's IR receiver and press the large UP and DOWN buttons on the hand control simultaneously for about 2 seconds. A signal is heard when the function is activated. The lift is now ready to use with Hand control IR.

**B.** Check to determine if other lifts in the room are affected by your hand control. If they are: Use the large UP and DOWN buttons on the operation panel of the lift that is to be used. Press both buttons simultaneously until a signal sounds. The lift has now been assigned a unique communication code. Now, activate IR control according to procedure A, above.

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490054

3SE690227-02  
2013-01-31

### Contains:

Control Unit CBL20, cable	1 (pre-configured)
Cable cover	1
Screw, cable cover	2
Screw M5x35	2 (Viking)
Screw M5x50	2 (Golvo)
Allen Key 2,5 mm	1

### Fits:


Viking XL
Viking L
Viking M
Golvo 8008 LowBase
Golvo 8000
Golvo 8008

### Qty:


1 (pre-configured)
1
2
2 (Viking)
2 (Golvo)
1

### Prod. No.

2040033
2040034
2040035
2000039
2000034
2000035

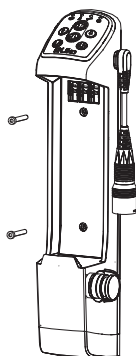


### Tools required:

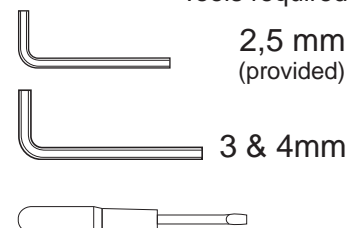


1.




**Before use!**  
Make sure the  
S/N on the label  
on Control  
Unit CBL is the  
same as on the  
lift.

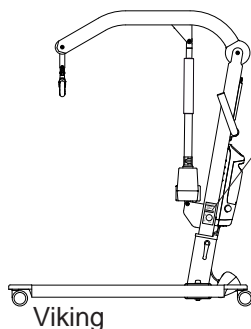


S/N:

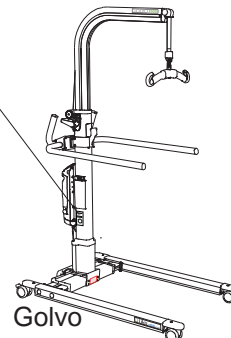


2.

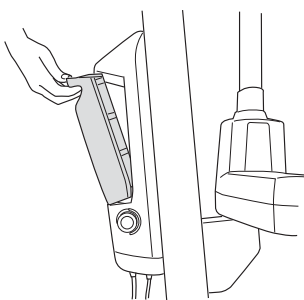


3.

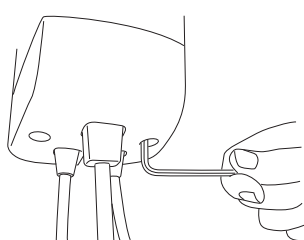


4.

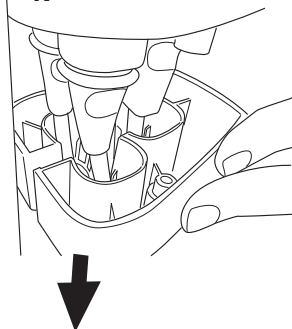


5.

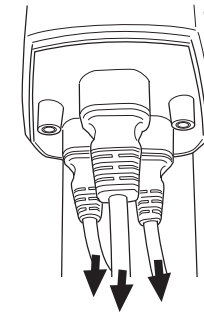

6. Viking:

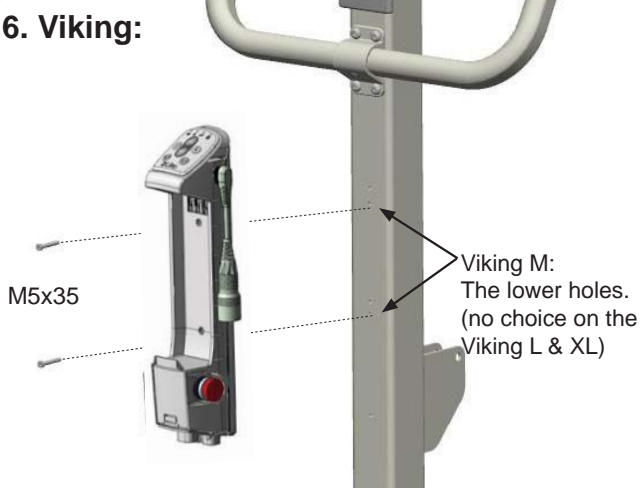

Golvo:

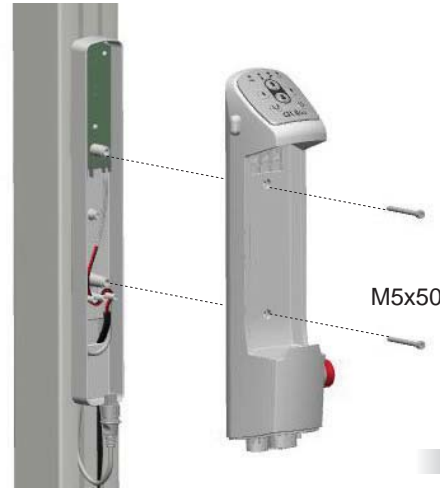


## Assembly Instruction Monteringsanvisning



Prod. No. 20490054

3SE690227-02

8.

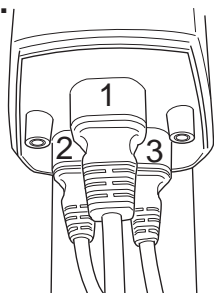


- Charger cable; socket 1
- Lift motor cable; socket 2
- Base-motor cable; socket 3

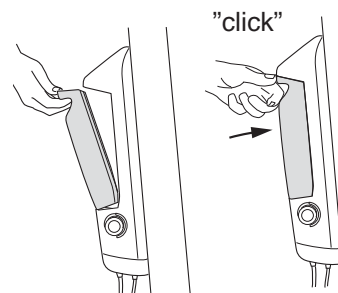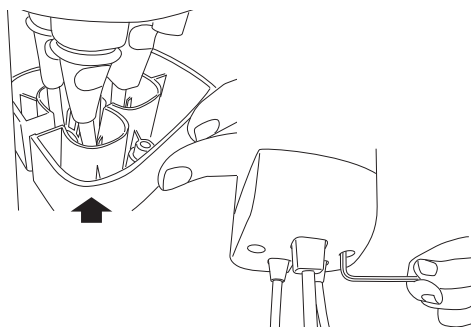


Battery  
Prod. No. 2006107


9.

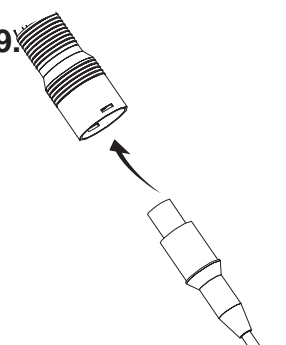


Connect the Hand control.

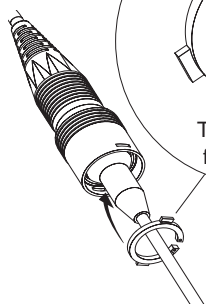


Attach the locking ring.

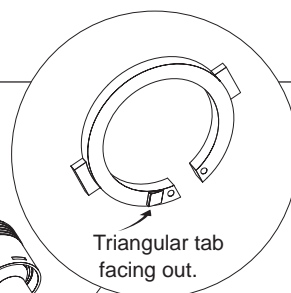

Triangular tab  
facing out.

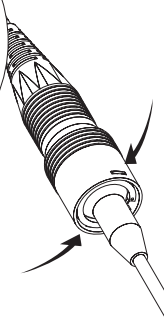

Make sure the rectangular tabs  
at the locking ring are fixed.

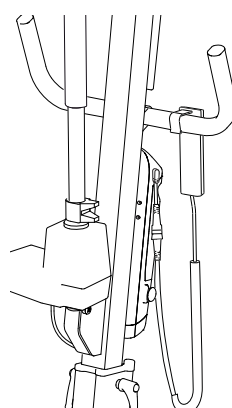

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.



## Assembly Instruction Monteringsanvisning

Prod. No. 20490033

3SE650214-02  
2010-04-13


**Contains:**

	Qty:
Actuator Viking M with Outer Tube	1
Sleeve bearing Mob.38	2
Coverhat Liko + Cover hat Liko bottom	4
Washer PS 6x12x1	4
Screw M6S M6x50	2
Locking nut M6 Nyloc	2

**Fits:**

Viking M (Prod. No. 2040015)

**Tools required:**

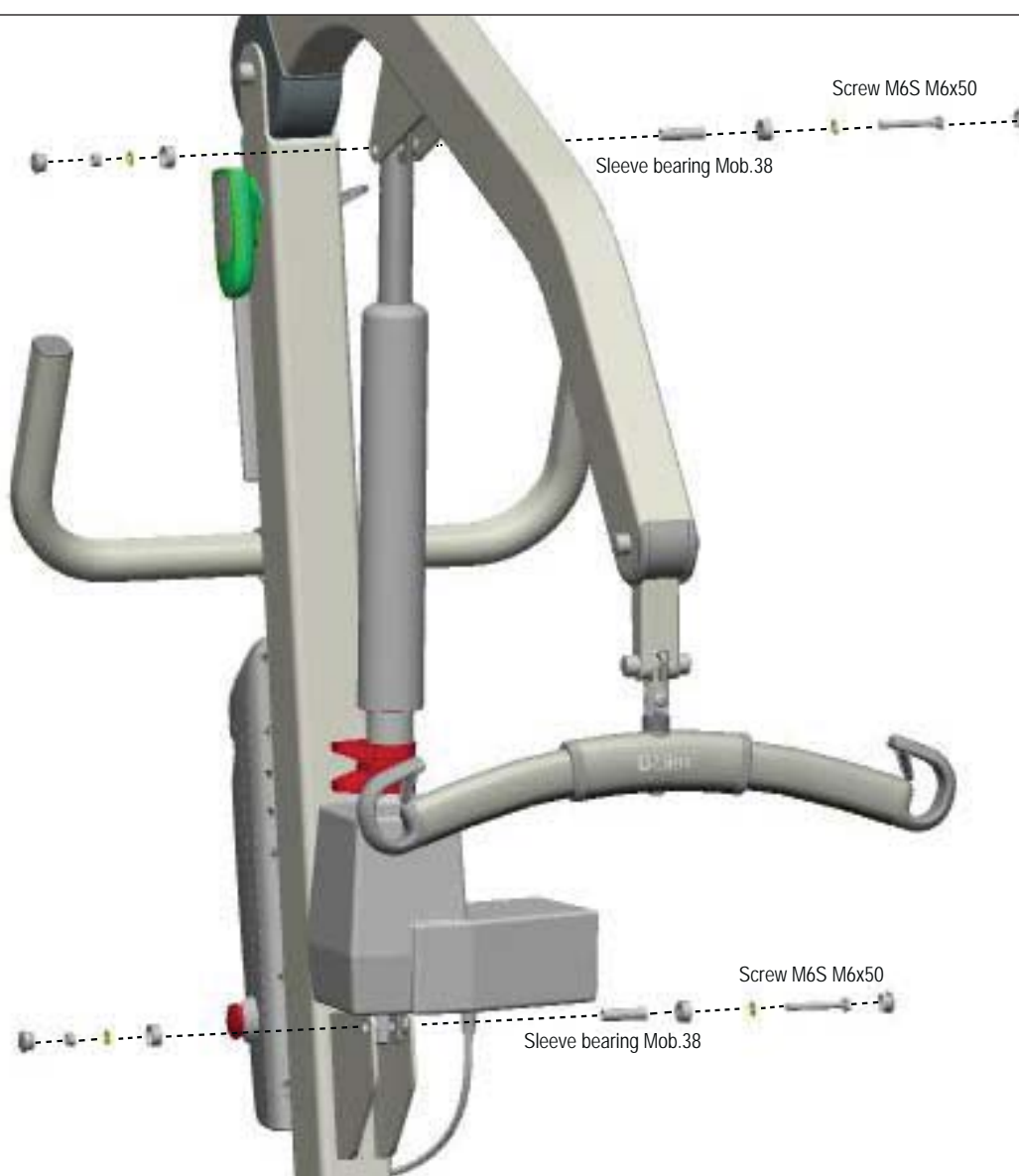2X  10mm

**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.
- Make sure the Outer Tube easy slides up and down.



## Assembly Instruction Monteringsanvisning

Prod. No. 21090018 Extension cable CBL20  
21090019 Extension cable CBL20, US


3SE650216-05  
2012-09-11

**Contains:**

Adapter cable charging

**Qty:**

1

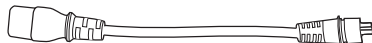
**Fits:**



Golvo 8000/8008  
Viking XL  
Viking L  
Viking M

**Prod. No:**

(all models)  
2040013 and 2040033  
2040014 and 2040034  
2040015 and 2040035

**Tools required:**


2,5 mm

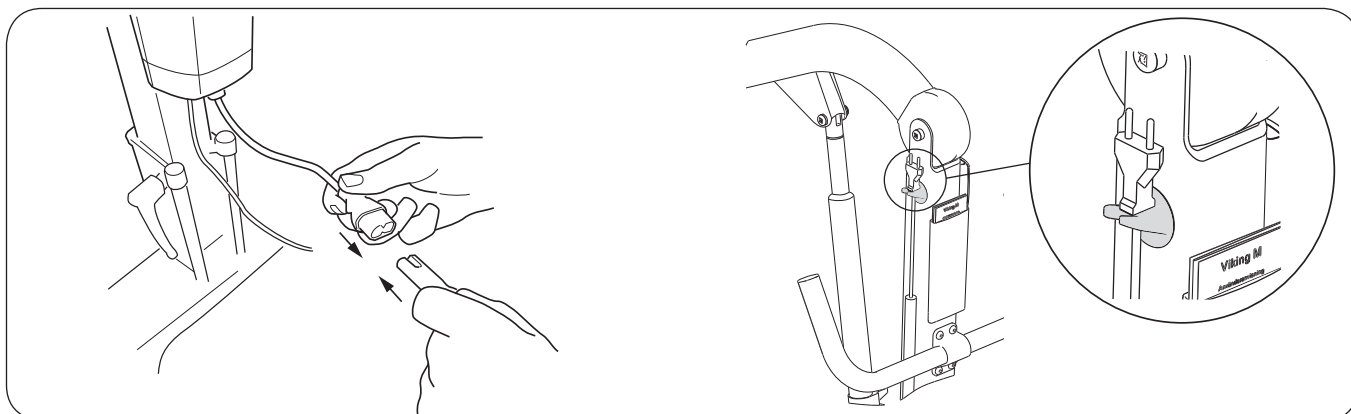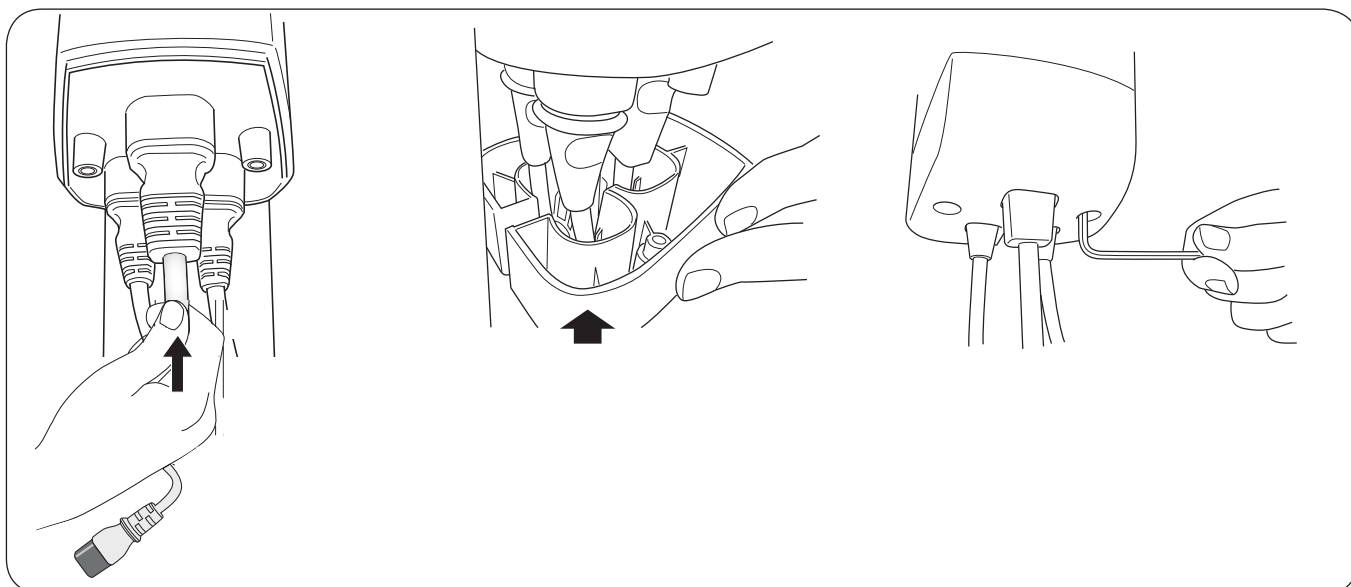

Connect the Charging cable to the Extension cable and the Extension cable to the wall socket (100-240 V AC). The battery is fully charged after 5 hours.

After completed charging, place the cable on the Charge cable hook.


**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning


Prod. No. 2006107

**Contains:**  
Batteri BAL20

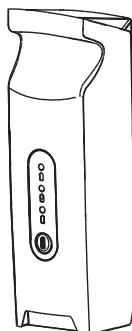**Qty:**  
1

**Fits:**

Viking M  
Viking M  
Viking L  
Viking L  
Viking XL  
Viking XL  
Golvo 8000/8008

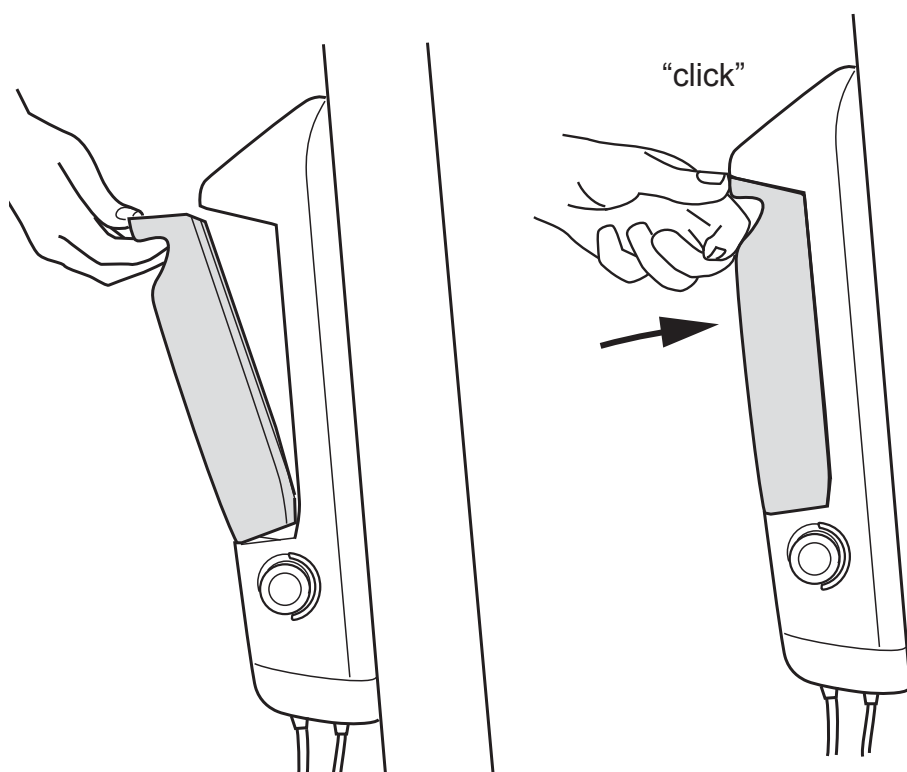**Prod. No.**

2040015  
2040035  
2040014  
2040034  
2040013  
2040033  
all models

**NOTE**

- Before use, charge the batteries according to the instruction guide.
- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.
- Old batteries are to be left at the nearest recycling station.



## Assembly Instruction Monteringsanvisning

Prod. No. 21090020

3SE650217-04

2011-03-04

**Contains:**




Belt Clips HBL20

**Qty:**

1

**Fits:**

Wireless hand control HBL20 Prod. No. 21090020

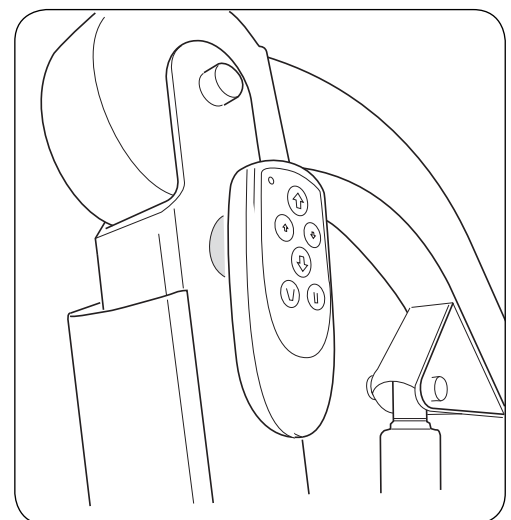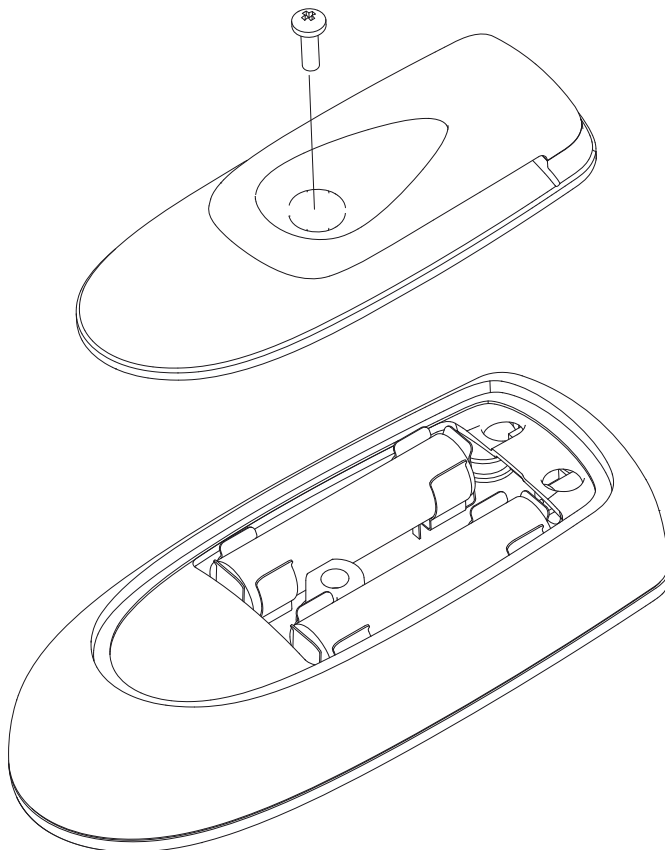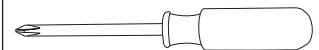**Tools required:**

**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 21090021

3SE650218-04

2011-03-04

**Contains:**

Clips with hook




**Qty:**

1

**Fits:**

Wireless hand control HBL20 Prod. No. 21090020

**Tools required:**

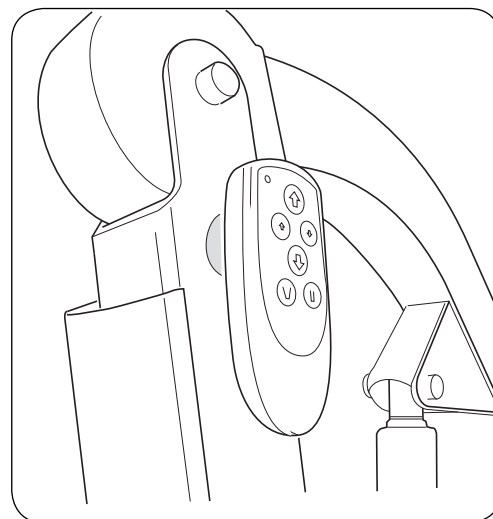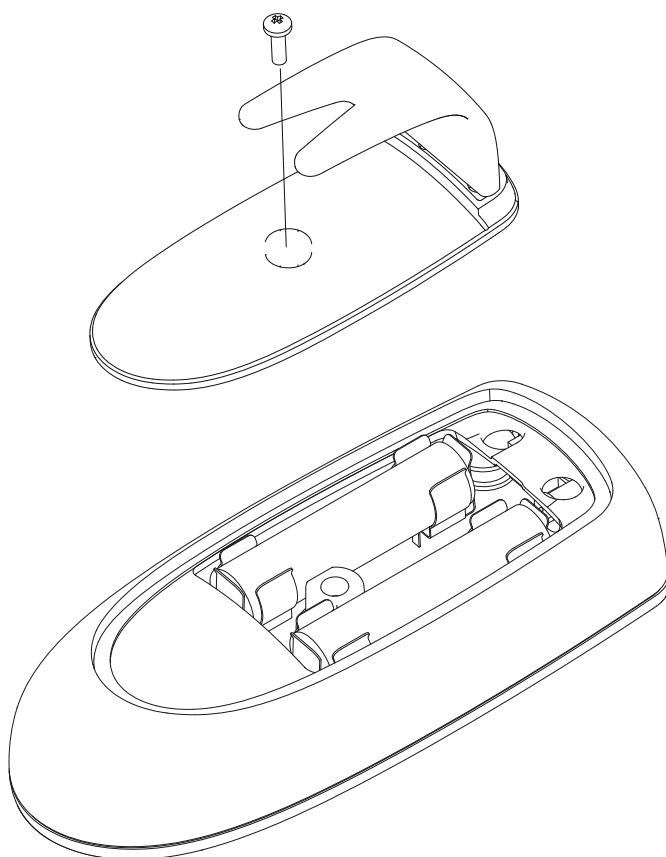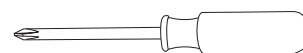

**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.


## Assembly Instruction Monteringsanvisning

Prod. No. 21090022

3SE650219-06  
2012-09-11

**Contains:**  
Charge cable hook


**Qty:**  
10

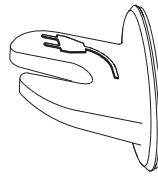


**Fits:**  
Golvo 8000/8008  
Liko M220  
Liko M230  
Viking XL  
Viking L  
Viking M

**Prod. No.**  
(all models)  
2050010  
2050015  
2040013 and 2040033  
2040014 and 2040034  
2040015 and 2040035


1:

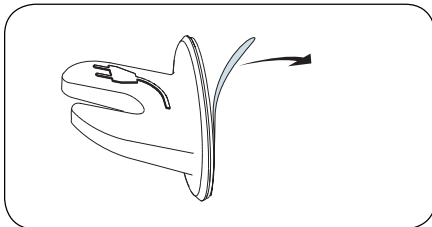


2: Viking XL

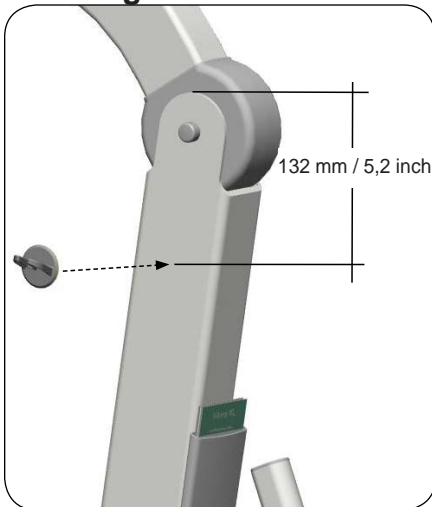


Viking L

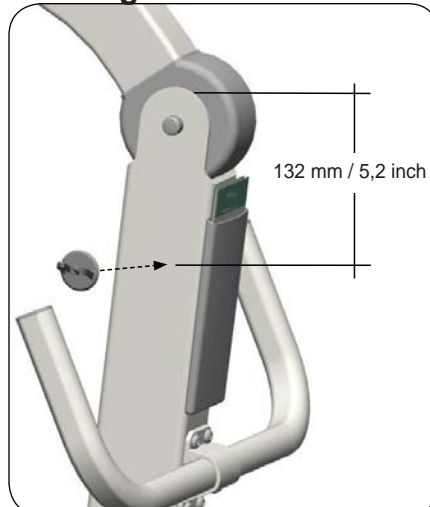


Viking M

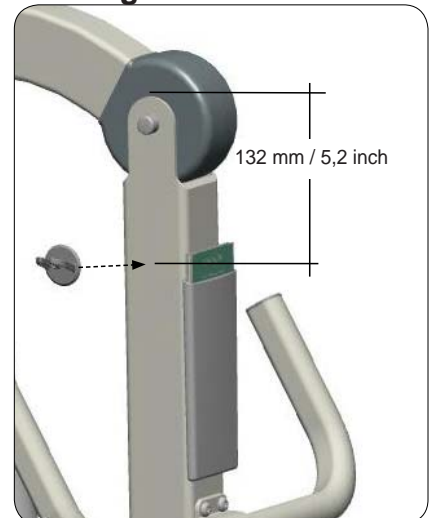

Golvo

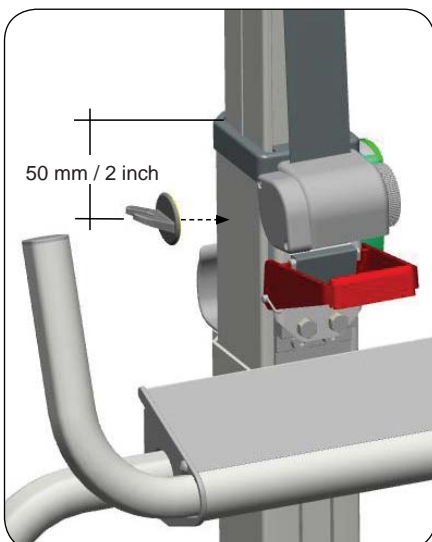

Liko M220 / M230

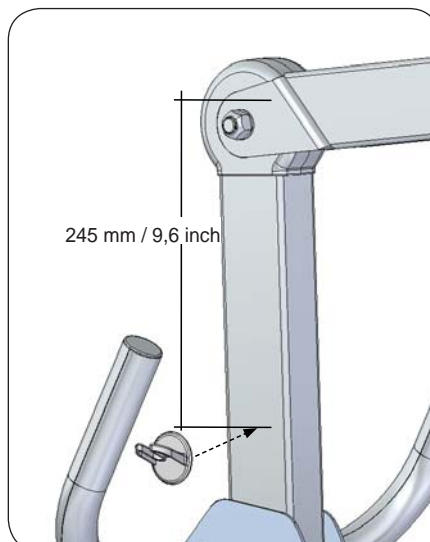

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 21090023

3SE650220-04  
2011-03-04

**Contains:**  
HandControl hanger

**Qty:**  
10


**Fits:**

Golvo 8000 / 8008


Viking M Prod. No. 2040015

Viking L Prod. No. 2040014


Viking XL Prod. No. 2040013

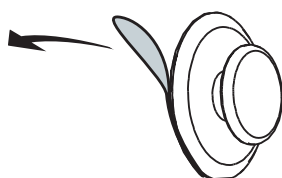

**Golvo**

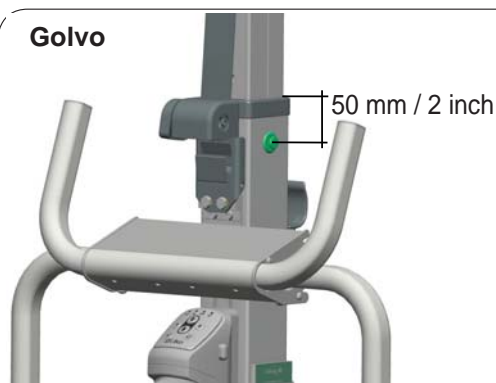

**Viking M, Viking L, Viking XL**

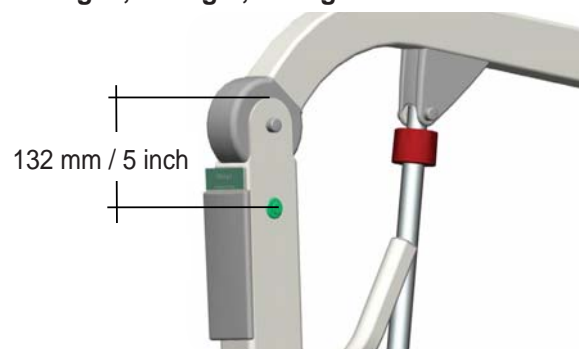

**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490036

3SE650259-02  
2010-06-14

### Contains:

Leg, right Mk II incl.  
link attachment and decal  
Leg, left Mk II incl.  
link attachment and decal

### Qty:

1

1


Nylon bearing 10x12x6  
Screw M6S M10x40  
Cover hat M10 grey U102  
Locking nut M10 nyloc

2

2

4

2

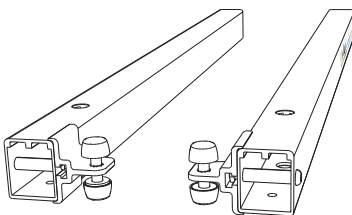

### Fits:

Viking M, from S/N 7100308  
(S/N below 7100308; Remove link attachment from spare part  
and use existing link attachment on lift.)


### Tools required

4mm

2x 17 mm

Kernite Unlock 2000

PTFE- grease

△ Before mounting; Press button for narrower base-width adjustment  to end position.



Grease the tip of the screws with  
Kernite Unlock 2000.

x2

Markings on the  
underside of the right leg!


Lubricate the inside of the  
Leg casing Viking outer,  
with flourflon PTFE- grease.

x2

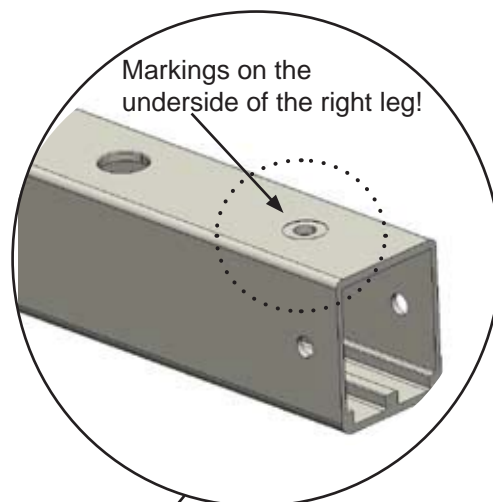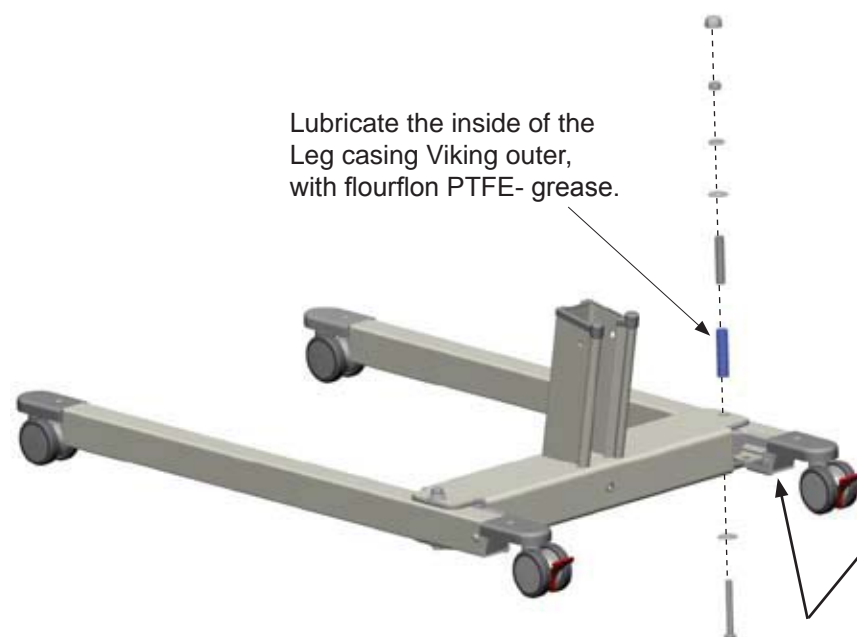


Assembly Instruction  
Monteringsanvisning

Prod. No. 20490036

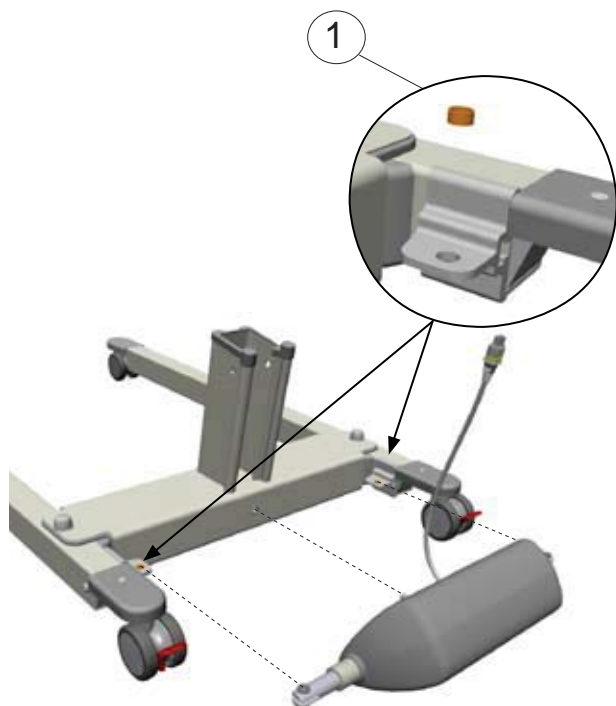

Pos. No	Name:	Pce
1	Nylon bearing 10x12x6	2

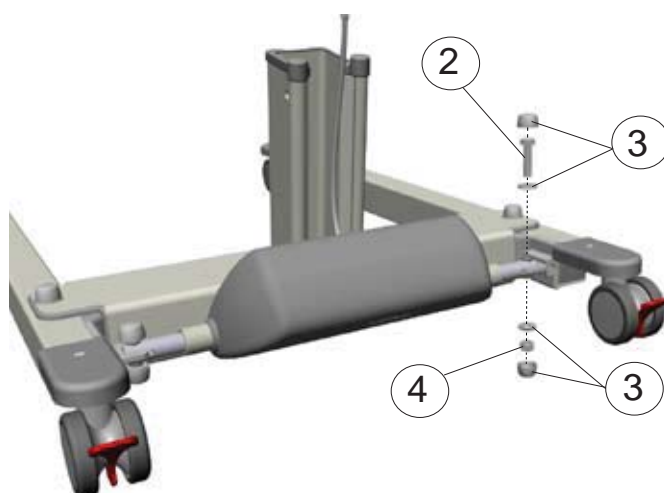

Pos. No	Name:	Pce
2	Screw M6S M10x40	2
3	Cover hat M10 grey U102	4
4	Locking nut M10 nyloc	2

**△ NOTE**

- Make sure the legs are mounted parallel to each other
- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.



# Assembly Instruction

## Monteringsanvisning


Prod. No. 20490038

3SE650269-01  
2010-04-12



**Contains:**  
Mast Viking M MKII  
(including handle and motor attachment)

**Qty:**  
1

**Fits:**  
Viking M (all models)

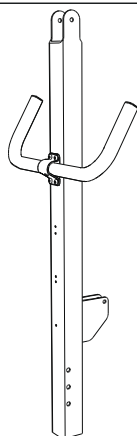

**Tools required**

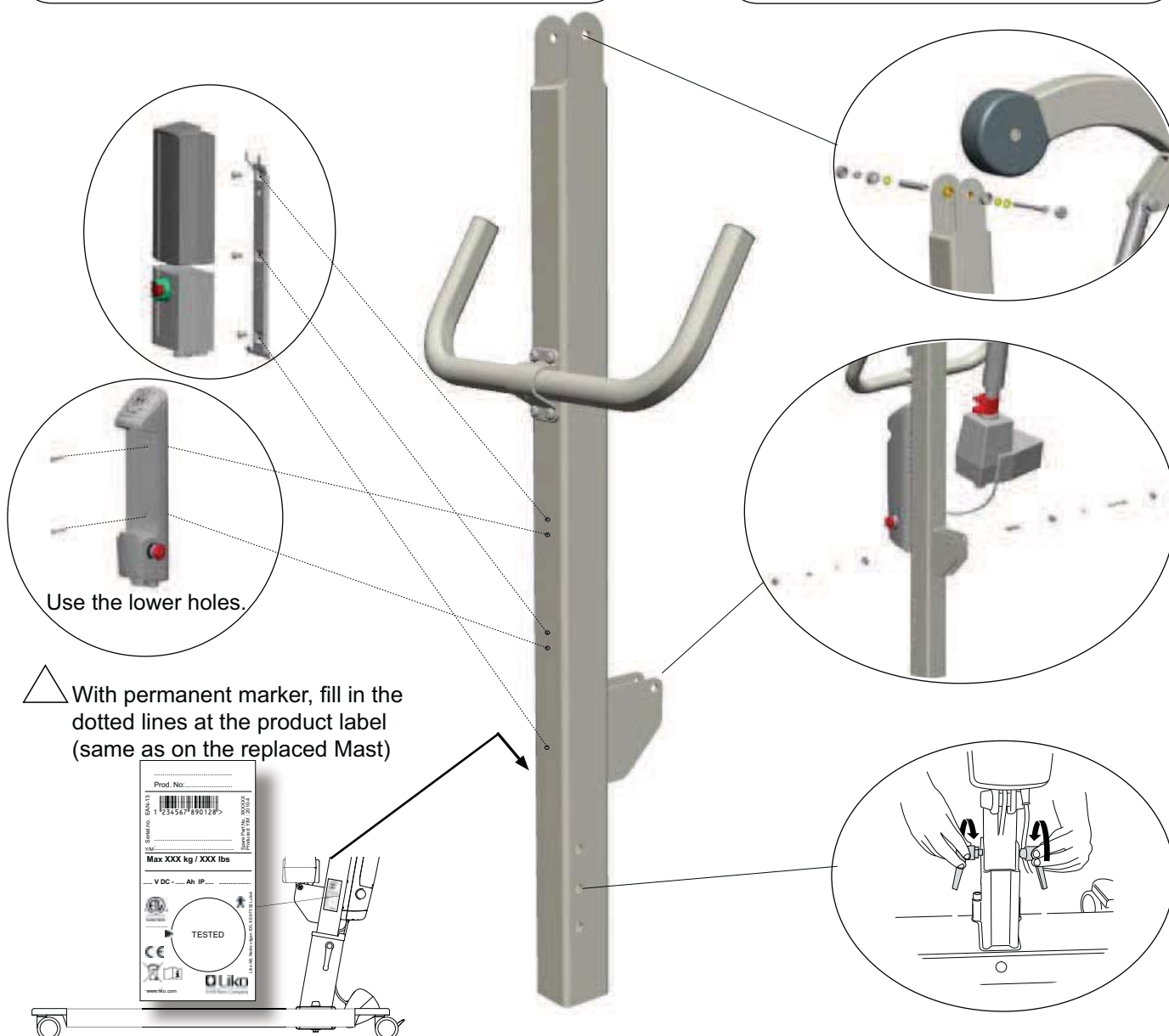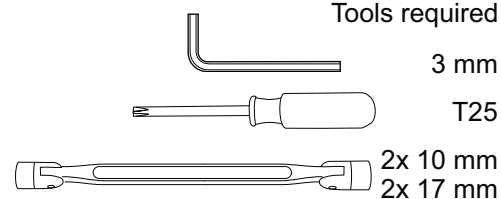

### NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.


## Assembly Instruction Monteringsanvisning

Prod. No. 20090077




3SE640217-01  
2010-05-27

**Contains:** Qty:  
Locking device 10

**Fits:**  
Control Unit CBL20

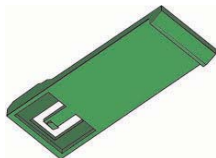

Tools required:

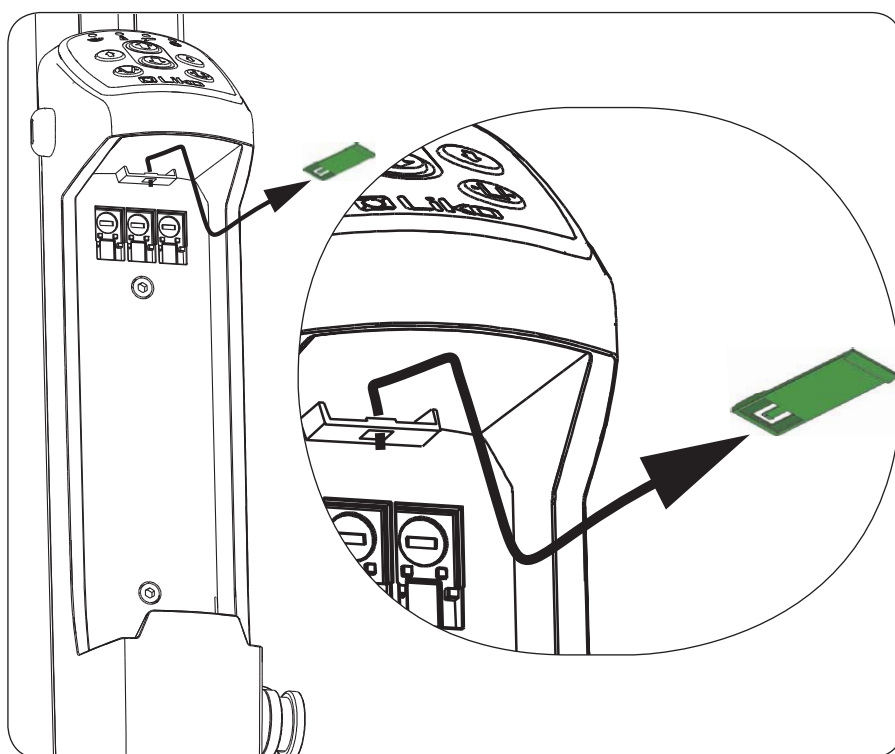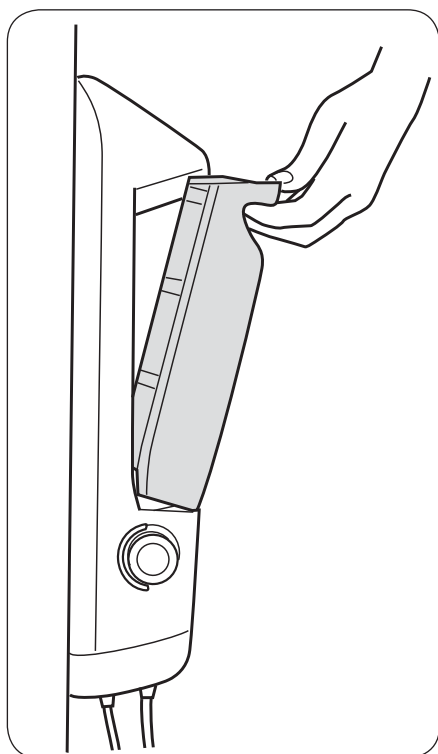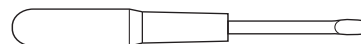

### ⚠ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

# Assembly Instruction


## Monteringsanvisning

Prod. No. 20490040

3SE650273-01  
2010-10-19


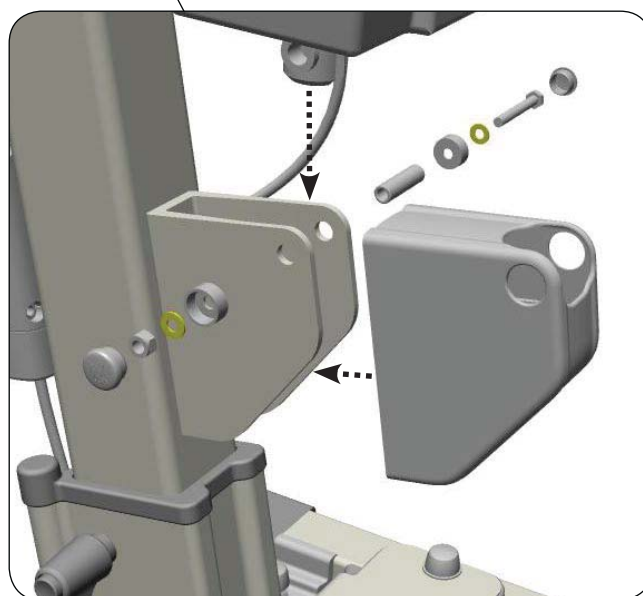
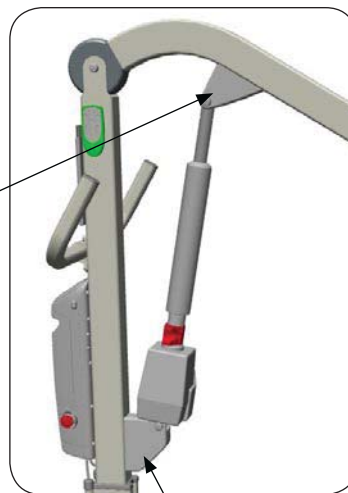
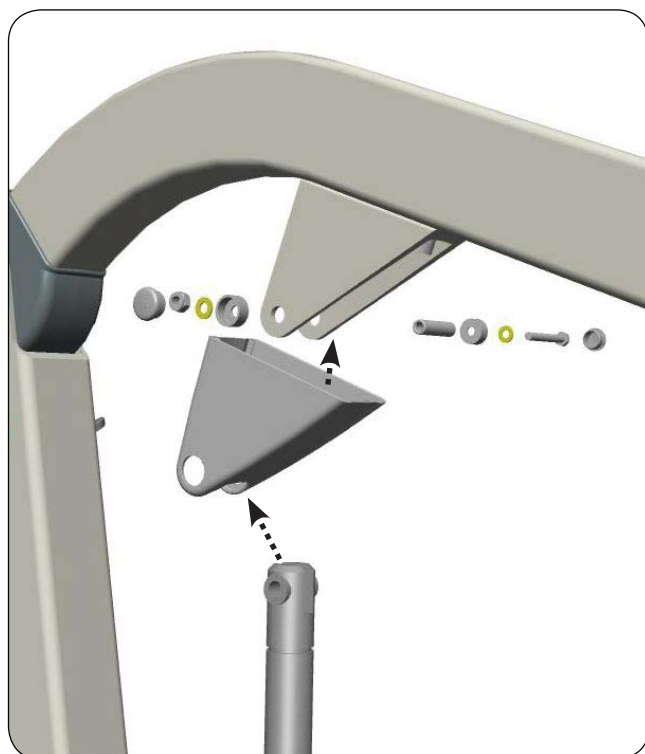
**Contains:**

Bracket cover, pillar  
Bracket cover, arm  
Sleeve bearing Mob.38  
Cover hat Liko bottom  
Washer PS 6x12x1  
Screw M6S M6x50  
Locking nut M6 Nyloc  
Cover hat Liko

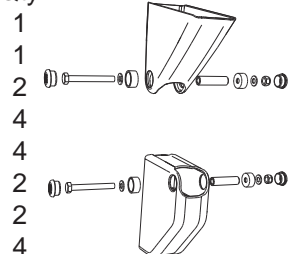**Qty:**

**Fits:**

Viking M (all models)  
Viking L, Prod. No. 2040004  
Viking XL (all models)

**Tools required:**

2 X  10 mm

**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning


Prod. No. 20490047 Leg cover L & XL  
20490048 Leg cover M

3SE650294-01  
2011-04-01


**Contains:**  
Leg cover, 3 pce

**Qty:**  
2

**20490047 Fits:**  
Viking L  
Viking XL

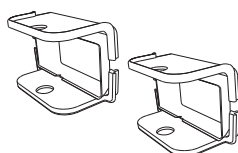


**20490048 Fits:**  
Viking M

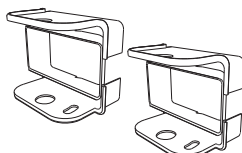

Tools required



2,5 mm

2x 17 mm  
Flourflon PTFE- grease


△ Before mounting; Press button for narrower base-width adjustment  to end position.

Prod. No.  
2040013, Viking XL  
2040014, Viking L  
2040015, Viking M

Prod. No.  
2040003, Viking XL  
2040004, Viking L  
2040005, Viking M

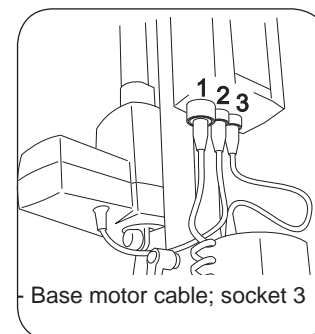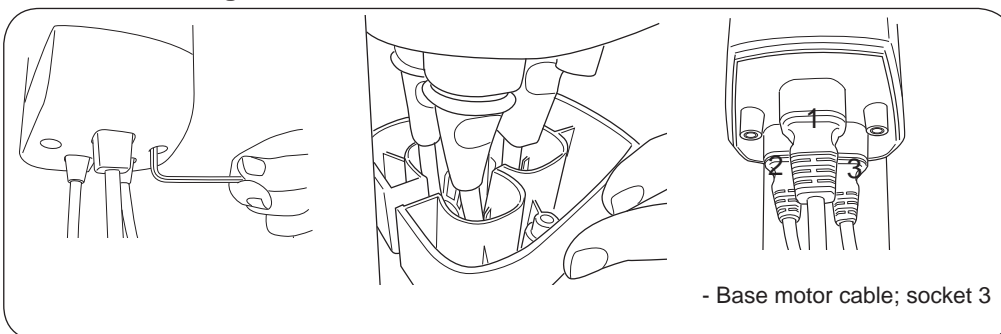

**Viking XL, Viking L, Viking M, All models:**

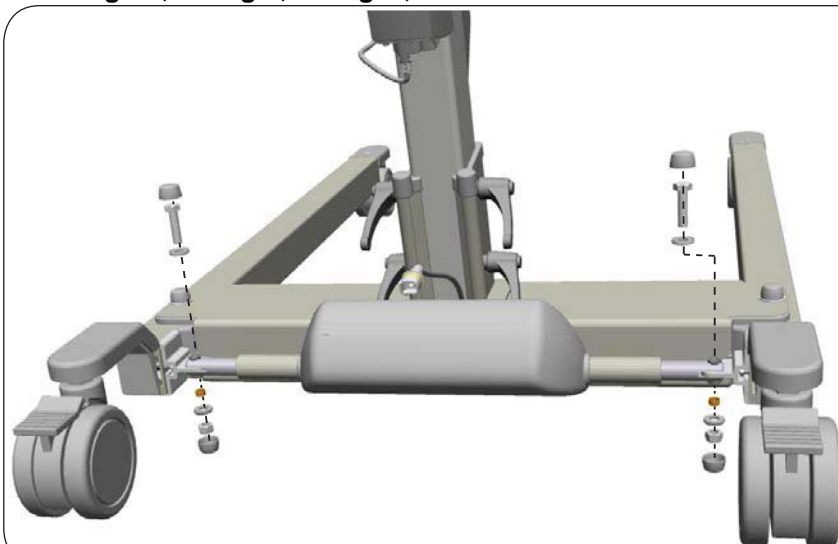


- Unfasten the base motor by removing the bolt connections.
- Remove the Base motor by pushing the motor axle in to the middle beam hole.



## Assembly Instruction Monteringsanvisning

Prod. No. 20490047 Leg cover L & XL  
20490048 Leg cover M

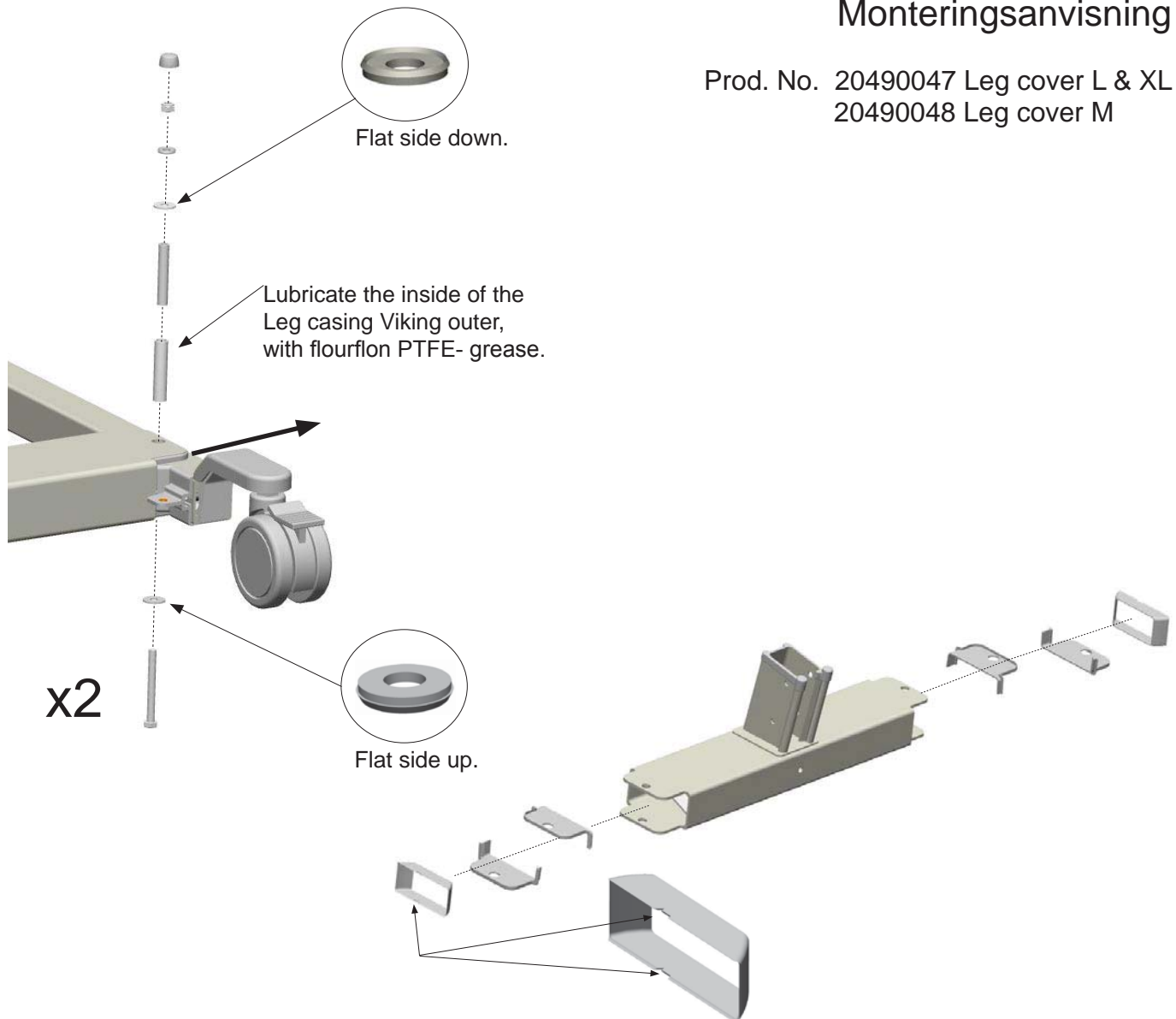

- Finish the assembly by mounting the base motor and bolt connections.  
(Picture showing Viking XL, same assembly for Viking M and Viking L.)

### **△ NOTE**

- Make sure the legs are mounted parallel to each other
- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.


## Assembly Instruction Monteringsanvisning

Prod. No. 20490049

3SE650295-01  
2011-04-01


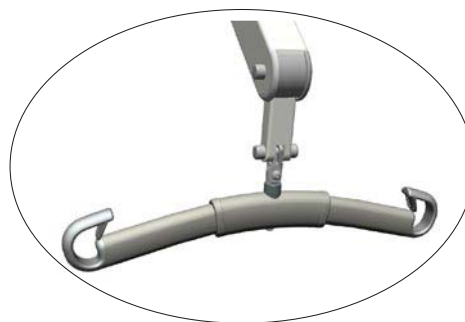
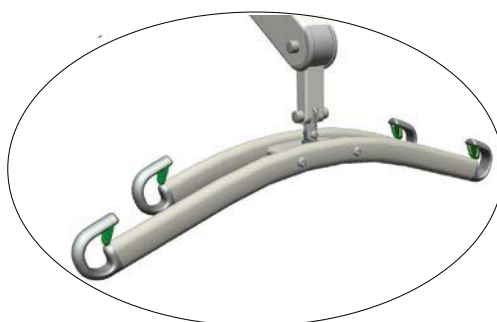
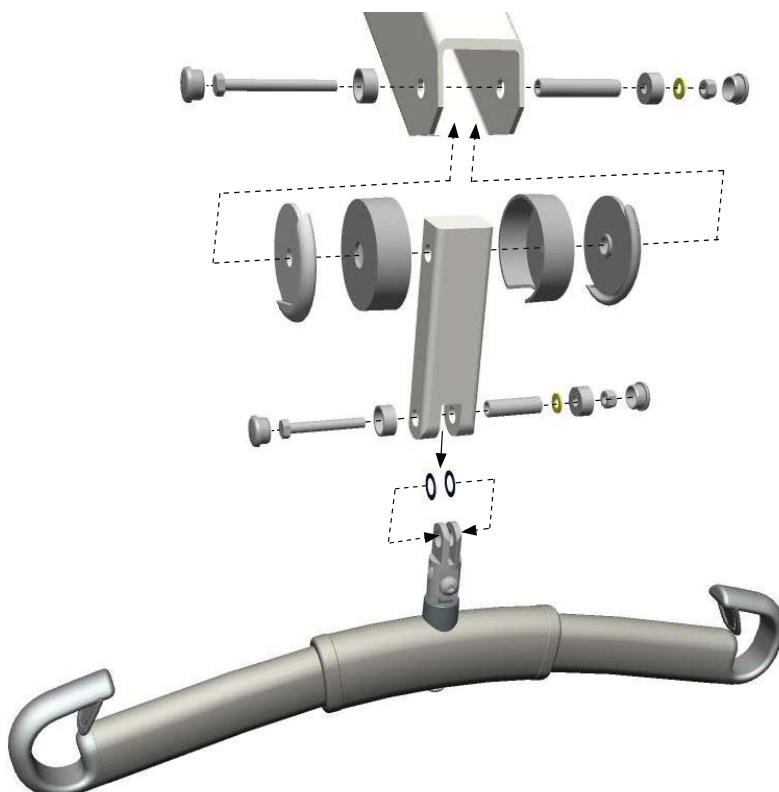
**Contains:**
**Qty:**

Friction washer Link	2
Link Viking	1
Sleeve bearing Mob.34	1
Cover hat Liko, bottom	4
Screw M6S M6x45	1
Washer TBRB 6x12x1	2
Locking nut M6 Nyloc	2
Cover hat Liko	4
Cap I Viking	2
Cap Y Viking	2
Sleeve bearing Mob.50	1
Screw M6S M6x60	1

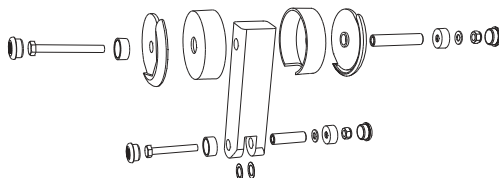
**Fits:**

Viking XL, Viking L, Viking M, Viking XS

**Tools required**

2x  10 mm

**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Liko/Hill-Rom and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20090080

3SE690208-01  
2011-09-23

**Contains:**


① Front fork locking light

② Nut M6LM M10 ISO

**Qty:**

**Fits 75 mm Wheels and 100 mm Wheels on:**

Golvo  
LikoLight  
Viking  
RollOn

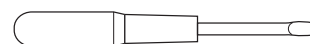**Tools required**


16 mm


**△ NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.



## Assembly Instruction Monteringsanvisning


Prod. No. 20090081

3SE690209-01  
2011-09-28


**Contains:**
**Qty:**

Fork

2

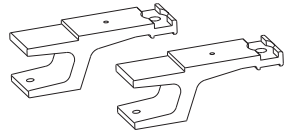





Fits:	Front Wheel:	Rear Wheel:
Golvo	X	X
Golvo LowBase	-	X
LikoLight	X	X
Viking XL	X	-
Viking L	X	-
Viking M	X	X
Viking S	X	X
Viking XS	X	X
RollOn	X	X

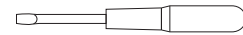**Tools required**


16mm



4 mm

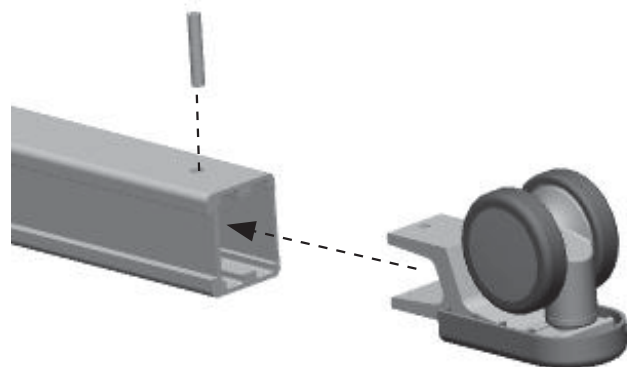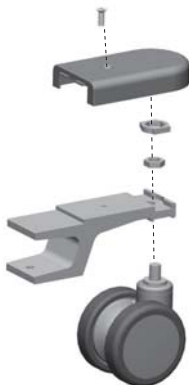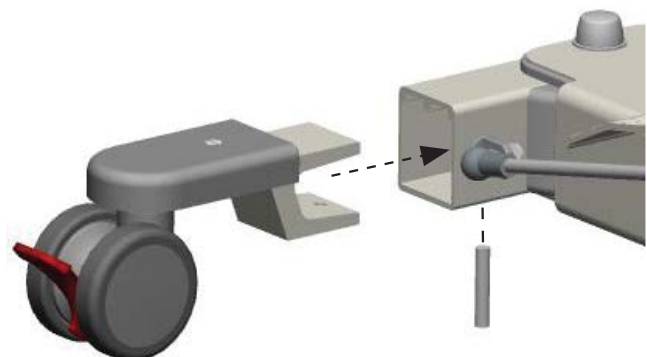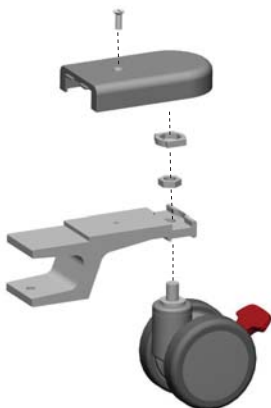**Assembly on front wheel:**

**Assembly on rear wheel:**

**NOTE**

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.



## Assembly Instruction


### Monteringsanvisning

Prod. No. 20490052

3SE690210-01  
2011-09-26

#### Contains:


Slingbar bolt casing R2R	Qty: 1
Slingbar Adapter	1
Threaded casing M6 10x16,5	1
Screw MRT M6x8 Ezloc A2	2

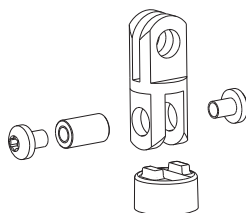

#### Fits:

Golvo (all models)  
Uno 200  
LikoLight  
Viking L, M, S, XS  
Universal Slingbar 350 / 450 / 600

#### Tools required

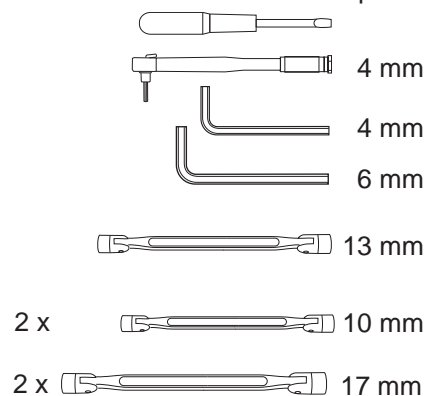


#### Golvo





#### Uno 200



#### LikoLight

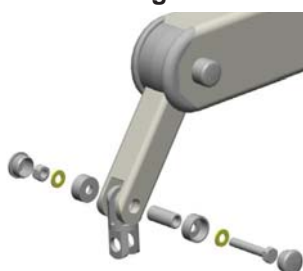

#### Viking

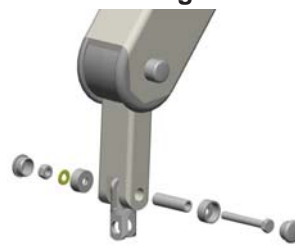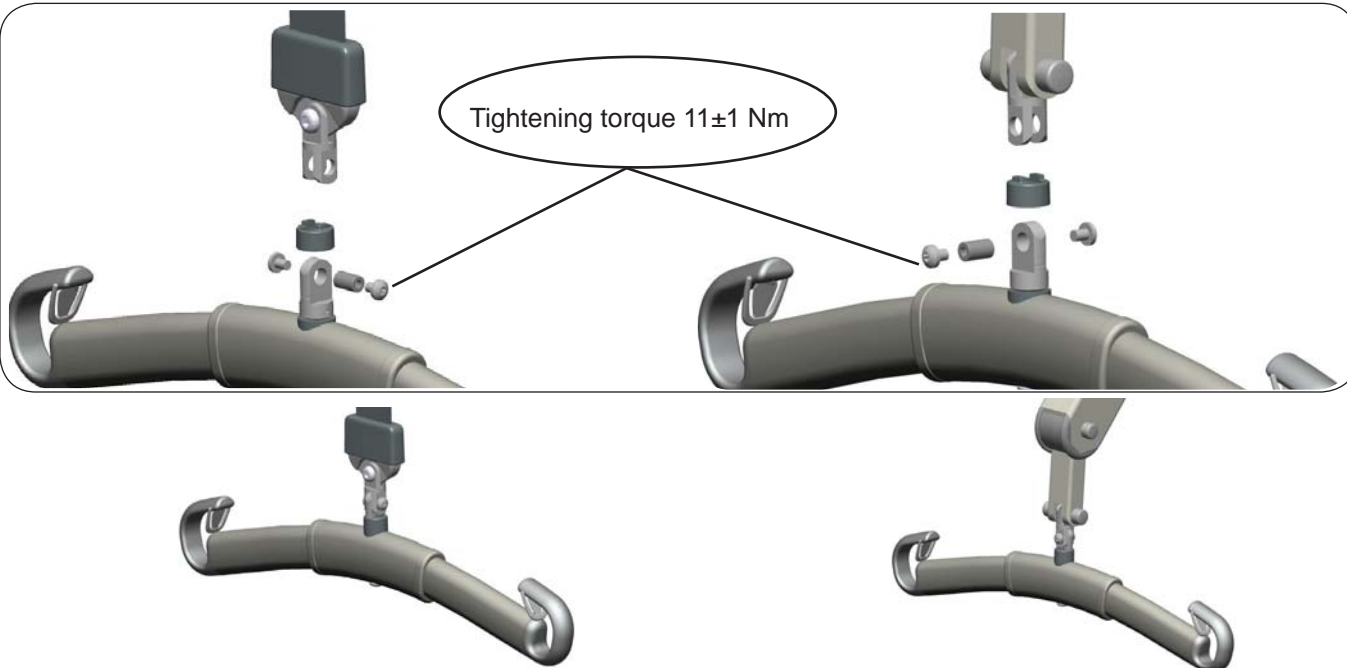
Tightening torque  $11 \pm 1$  Nm


#### NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## Assembly Instruction Monteringsanvisning

Prod. No. 20490057


3SE690231-02  
 2013-01-17

### Contains:

Locking Ring

### Qty:

10

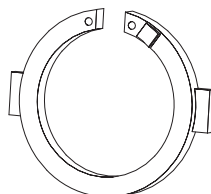

### Fits:

Viking M

### Prod No.

2040035

Viking L

2040034

Viking XL

2040033

Golvo 8000

2000034




Golvo 8008

2000035


Golvo 8008 LowBase

2000039


### Tools required

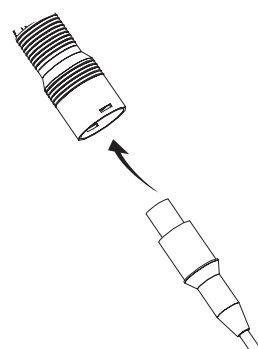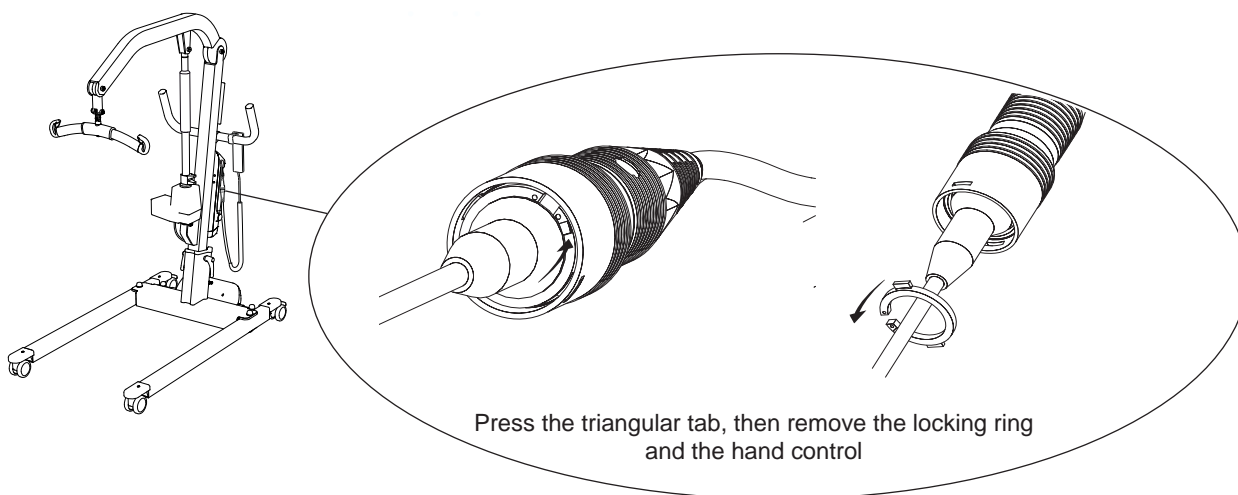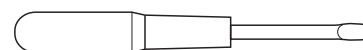

Connect the Hand control.

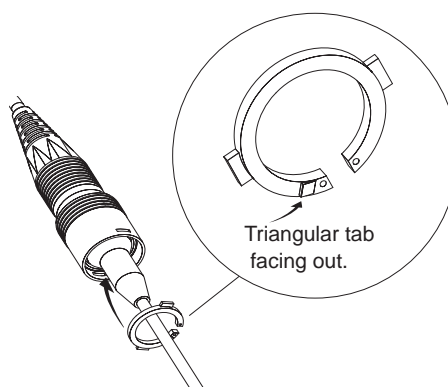

Attach the locking ring.

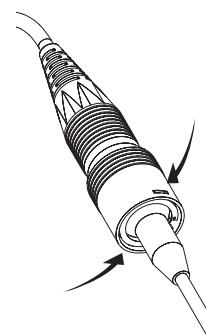

Make sure the rectangular tabs at the locking ring are fixed.

### △ NOTE

- Make sure the spare part is mounted according to the assembly instruction.
- Check all functions related to the changed spare part.
- Repairs and maintenance may only be performed by personnel authorized by Hill-Rom/Liko and using original Liko spare parts.

## PARAMETER Code Key

Liko Diagnostic System™ (operating data) is stored in Control Unit CBL

Parameter:	Information:
<b>TAAxT</b> [Ah]	Totally Accumulated Amphours (current x time)
<b>TAAxTR</b> [Ah]	Totally Accumulated Amphours, resetable (current x time) Can be reset by the manufacturer.
<b>PAAxT</b> [Ah]	Periodically Accumulated Amphours (current x time) Calculated continuously from the moment the lift is used until the first service.
<b>PAKT</b> [Months]	Periodically Accumulated Calendar time. Calculated between services.
<b>TADT</b> [Hours]	Totally accumulated drive time.
<b>AAS</b> [Starts]	Accumulated starts. The number of lifting motions counted from the moment the lift was used the first time.
<b>ASERV</b> [Services]	Accumulated services Number of performed service events made using LSS or LDS.
<b>IMED</b> [A]	Average power consumption during TADT. Overall accumulated operating time.
<b>ALA</b> [Charges]	Accumulated number of started chargings.
<b>AALA</b> [Charges]	Accumulated number of completed chargings
<b>AOLA</b> [Charges]	Accumulated numbers of not completed charging. Counts each disrupted charging event.
<b>AABT</b> [Removals]	Accumulated number of battery replacements. Counts each battery replacement where a used battery is replaced.
<b>AEUP</b> [Endpos. up]	Accumulated number of end positions during lifting motion up.
<b>AFSERV</b> [Services]	Accumulated number of reset Service Number of "Clear Service" performed manually.
<b>AEDWN</b> [Endpos. down]	Accumulated number of end positions during lifting motion down.
<b>SFSTAT</b>	N/A (This function is not in use)
<b>IDNR</b>	Identity number of the control box.
<b>AOLUP</b> [Overloads]	Accumulated number of stops due to overload.
<b>PADT</b> [Hours]	Periodically accumulated operating time.
<b>IMED 20</b> [A]	Average current consumption. Recounted for each lifting motion (up) which lasts more than 20 seconds.
<b>ELT</b> [Ah]	TAAxTR limit (constant) for ELT indication.
<b>ELTCNT</b> [Commands]	Number of ELT resets (only performed at replacement of actuator).
<b>CLIM_1</b> [A]	Current limit for lifting motion (up).
<b>CLCT</b> [ms]	Maximum allowed time to override CLIM_1 before cut-off
<b>CLIM_2</b> [A]	Current limit for widening motion (base-width adjustment).
<b>SINTT</b> [Months]	Indicates service intervals in calendar months.
<b>SINTAS</b> [Ah]	AxT limit for service event. (current x time)
<b>CYCLET</b> [Hours]	Cycle time is a constant for a medium driving cycle.
<b>PAOLUP</b> [Overloads]	Periodic accumulated number of stops due to overload.






Control Unit CBL

- The information listed above is stored in the Control Unit CBL.
- Replacing the Control Unit CBL without using the LSS may cause some data loss as this data is recordable by LSS and a computer. When data is stored in Liko database it can be retrieved if needed.
- Resetting the Service Interval using the LSS, LDS or Manually, will reset the Accumulated Amp-hours (PAAxT), Accumulated Time (PAKT), Accumulated Operating Time (PADT) and Accumulated number of stops due to overload (PAOLUP).

## RESET SERVICE INTERVAL PARAMETERS

## 6.2

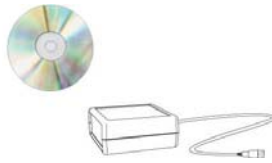After the Periodic Inspection has been performed Service Light and Service Interval Parameters\* in the Liko Diagnostic System™ has to be reset. Reset can be done using one of the following Service Tools/Methods:

Prod. No. 21090024	<b>1. Liko Service System, LSS</b> (Requires a computer) <i>Read and store the data from the Control Unit CBL into a computer. Data can also be transferred to a Liko Server. Can also reset the ELT-light (Expected Life Time). Use when Periodic Inspection or Preventative maintenance is performed.</i>  <i>Service Documents for the lift can be printed. Use for tracking and and transfer data when Spare Parts are changed.</i>	
Prod. No. 21090016	<b>2. LDS Service (LDS)</b> <i>Reads the data from the Control Unit CBL. Use when Periodic Inspection or Preventative maintenance is performed.</i>	
-	<b>3. Manual Service</b> <i>Resets the Service Interval Parameters*.</i> <b>△ Service Interval Parameters should only be reset after a complete Periodic Inspection</b> <i>Use the following combination of buttons on the control panel:</i> <b>A: Press and hold button 1 &amp; 2</b> <b>B: While holding 1 &amp; 2, press button 3, and hold all three buttons for three seconds.</b>  <i>(Works from Item No: CBL20001-05 on CBL (see label on CBL))</i>	

### IMPACT ON PARAMETERS

Changes of parameters when LSS, LDS or Manuel Service reset has been used:

Parameter:	1. LSS:	2. LDS:	3. Manual Service:	Check Points in PM ***
Service Light *	Unlit	Unlit	Unlit	
TAAxT	unchanged	unchanged	unchanged	x ***
TAAxTR	unchanged	unchanged	unchanged	
PAAxT **	0	0	0	x ***
PAKT **	0	0	0	
TADT	unchanged	unchanged	unchanged	
AAS	unchanged	unchanged	unchanged	x ***
ASERV	+1	+1	unchanged	x ***
IMED	unchanged	unchanged	unchanged	x ***
ALA	unchanged	unchanged	unchanged	x ***
AALA	unchanged	unchanged	unchanged	x ***
AOLA	unchanged	unchanged	unchanged	x ***
AABT	unchanged	unchanged	unchanged	x ***
AEUP	unchanged	unchanged	unchanged	x ***
AFSERV	0	0	+1	x ***
AEDWN	unchanged	unchanged	unchanged	x ***
SFSTAT	N/A	N/A	N/A	
IDNR	unchanged	unchanged	unchanged	x ***
AOLUP	unchanged	unchanged	unchanged	x ***
PADT **	0	0	0	x ***
IMED20	unchanged	unchanged	unchanged	
ELT	unchanged	unchanged	unchanged	
ELTCNT	unchanged	unchanged	unchanged	
CLIM1	unchanged	unchanged	unchanged	
CLCT	unchanged	unchanged	unchanged	
CLIM2	unchanged	unchanged	unchanged	
SINTT	unchanged	unchanged	unchanged	
SINTAS	unchanged	unchanged	unchanged	
CYCLET	unchanged	unchanged	unchanged	
PAOLUP **	0	0	0	x ***

\* Indicates Service Periodic Inspection interval reached.

\*\* Service Interval Parameters (the PAAxT, PAKT, PADT and PAOLUP are all used to determine Service intervals)



\*\*\* Parameters to use in the Preventative Maintenance



A **Hill-Rom** Company

## Liko Service System LSS

Prod. No. 21090024

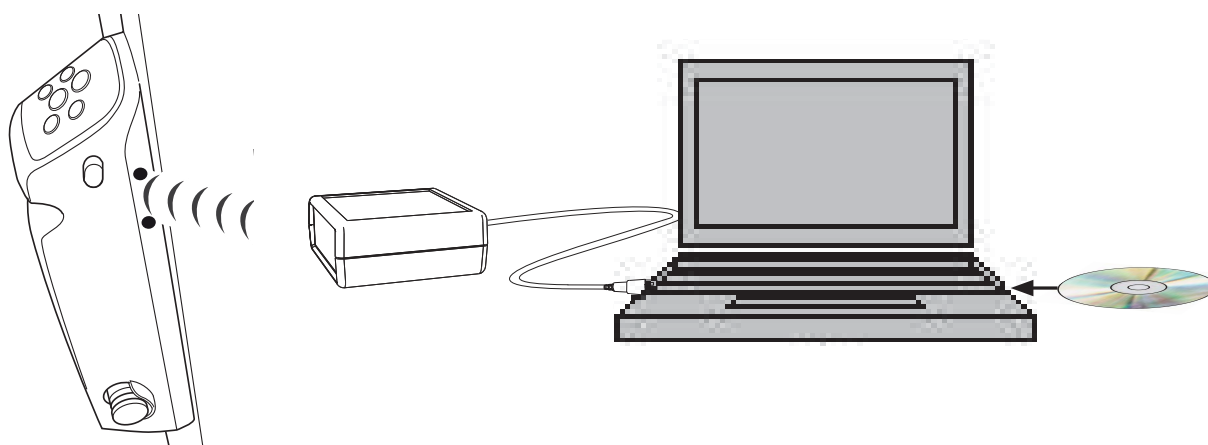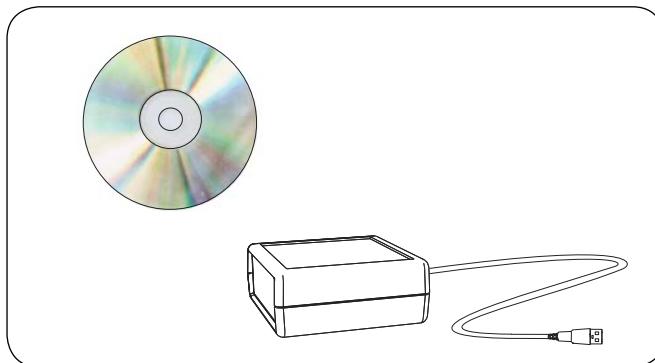


### Product Description

Liko Service System is a software developed to enable service technicians to conduct service in an easy way on all lifts equipped with the control unit CBL 20. It also gives the service technician the possibility to program the spare part on the field.

***Detailed information can be found in the instruction guide accompanying the product.***

## Liko Diagnostic System - Service

LDS Service Prod. No. 21090016

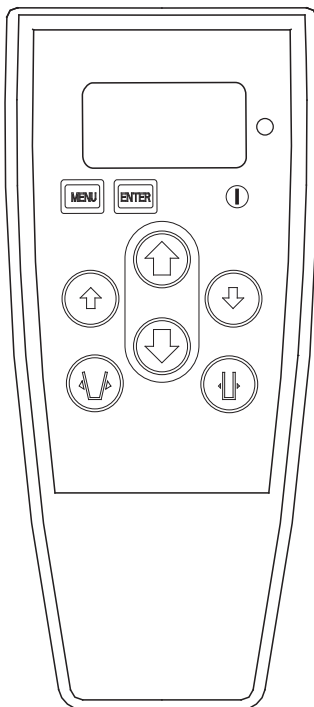

### Product Description

Liko lifts equipped with LDS (Liko Diagnostic System™) record and store information on how the lift is used. LDS Service can read the information stored in the Control unit CBL.

LDS Service can be used as hand control to operate Liko's IR controlled lifts.

LDS Service is specially developed for Liko's authorized service technicians to reset the Control Unit CBL service interval after completed service/periodic inspection.

***Detailed information can be found in the instruction guide accompanying the product.***



A **Hill-Rom** Company

## Periodic Inspection

	Viking XL	Viking L	Viking M
Prod. No.	2040003	2040004	2040005
	2040013	2040014	2040015
	2040033	2040034	2040035

Output (TAAxT):  Ah (Prod. No. 2040013-2040035)  
(Use Liko Service System or Liko Diagnostic System to read data from the CBL)

Lift type: .....

Prod. No: .....

Serial No: .....

Prod. Year: .....

### Customer references

Agreement No: .....

Name: .....

Address: .....

Post code: .....

The lift must be thoroughly inspected when the Service Light at the Control Unit CBL illuminates, or at least once per year. Viking with other control unit shall be thoroughly inspected at least once per year. Inspection and service must be carried out by Hill-Rom/Liko authorized personnel. Instructions for check points, from the next page.

### Check points:

	Approved	Not approved	Comments:
1 Product decal	<input type="checkbox"/>	<input type="checkbox"/>	.....
2 Visible damage to the lift surface finish	<input type="checkbox"/>	<input type="checkbox"/>	.....
<b>Base</b>			
3 Castor wheels, forks and fork covers	<input type="checkbox"/>	<input type="checkbox"/>	.....
4 Castor brakes	<input type="checkbox"/>	<input type="checkbox"/>	.....
5 Base covers	<input type="checkbox"/>	<input type="checkbox"/>	.....
6 Base opening and closing linkage	<input type="checkbox"/>	<input type="checkbox"/>	.....
7 Base frame castor play and levelling	<input type="checkbox"/>	<input type="checkbox"/>	.....
<b>Mast</b>			
8 Locking handles	<input type="checkbox"/>	<input type="checkbox"/>	.....
9 Lift arm hardware and covers	<input type="checkbox"/>	<input type="checkbox"/>	.....
10 Flexlink to arm hardware and covers	<input type="checkbox"/>	<input type="checkbox"/>	.....
11 Flexlink to slingbar hardware and covers	<input type="checkbox"/>	<input type="checkbox"/>	.....
12 Slingbar	<input type="checkbox"/>	<input type="checkbox"/>	.....
13 Mast actuator	<input type="checkbox"/>	<input type="checkbox"/>	.....
14 Battery and electrical cable installations	<input type="checkbox"/>	<input type="checkbox"/>	.....
15 Hand control	<input type="checkbox"/>	<input type="checkbox"/>	.....
16 Emergency stop function	<input type="checkbox"/>	<input type="checkbox"/>	.....
17 Emergency lowering device	<input type="checkbox"/>	<input type="checkbox"/>	.....
18 Control unit CBL20	<input type="checkbox"/>	<input type="checkbox"/>	.....
<b>Load Testing</b>			
19 Max. load test	<input type="checkbox"/>	<input type="checkbox"/>	.....
20 Mechanical emergency lowering load test	<input type="checkbox"/>	<input type="checkbox"/>	.....
<b>Accessory</b>			
21 Armrest	<input type="checkbox"/>	<input type="checkbox"/>	.....
<b>Documentation</b>			
22 Instruction guide	<input type="checkbox"/>	<input type="checkbox"/>	.....
23 Clear Service (CBL 20)	<input type="checkbox"/>	<input type="checkbox"/>	.....

Approval to use the mobile lift ☐ Approved ☐ Not approved

### Not approved:

If the mobile lift has one or more check points with result "Not approved" the mobile lift must not be used.

### To be actioned:

Actions according to the "Instructions for the check points" should be performed immediately. After performed actions sign below. If anything is unclear or if you have questions, please contact Hill-Rom/Liko or your local Hill-Rom/Liko representative. Contact information is to be found at [www.liko.com](http://www.liko.com).

Inspection performed by: \_\_\_\_\_ Date: \_\_\_\_\_

Final approval by: \_\_\_\_\_ Approval date: \_\_\_\_\_

Next inspection: \_\_\_\_\_


Inspection performed in accordance with ISO 10535:2006 Annex B- Periodic inspection

## Check points

Viking XL Prod. No. 2040003

Viking L Prod. No. 2040004

Viking M Prod. No. 2040005

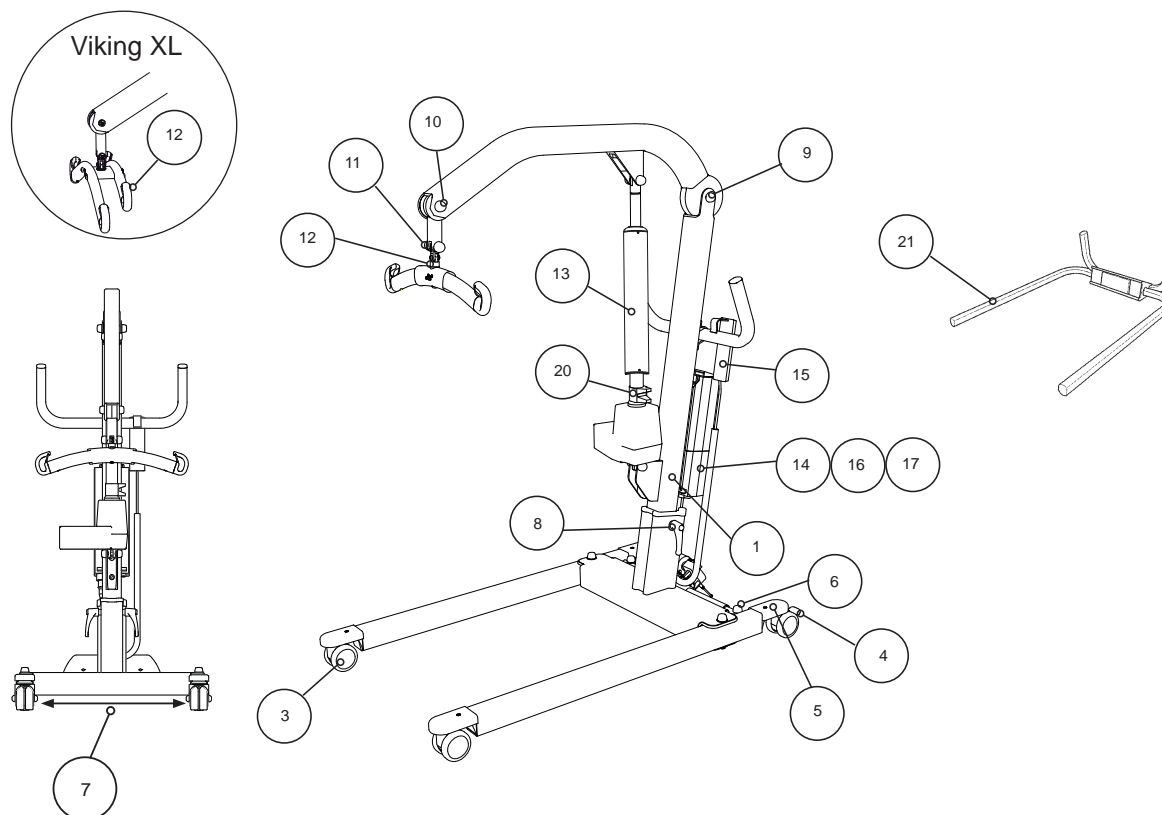


## Check points

Viking XL Prod. No. 2040013, 2040033

Viking L Prod. No. 2040014, 2040034

Viking M Prod. No. 2040015, 2040035

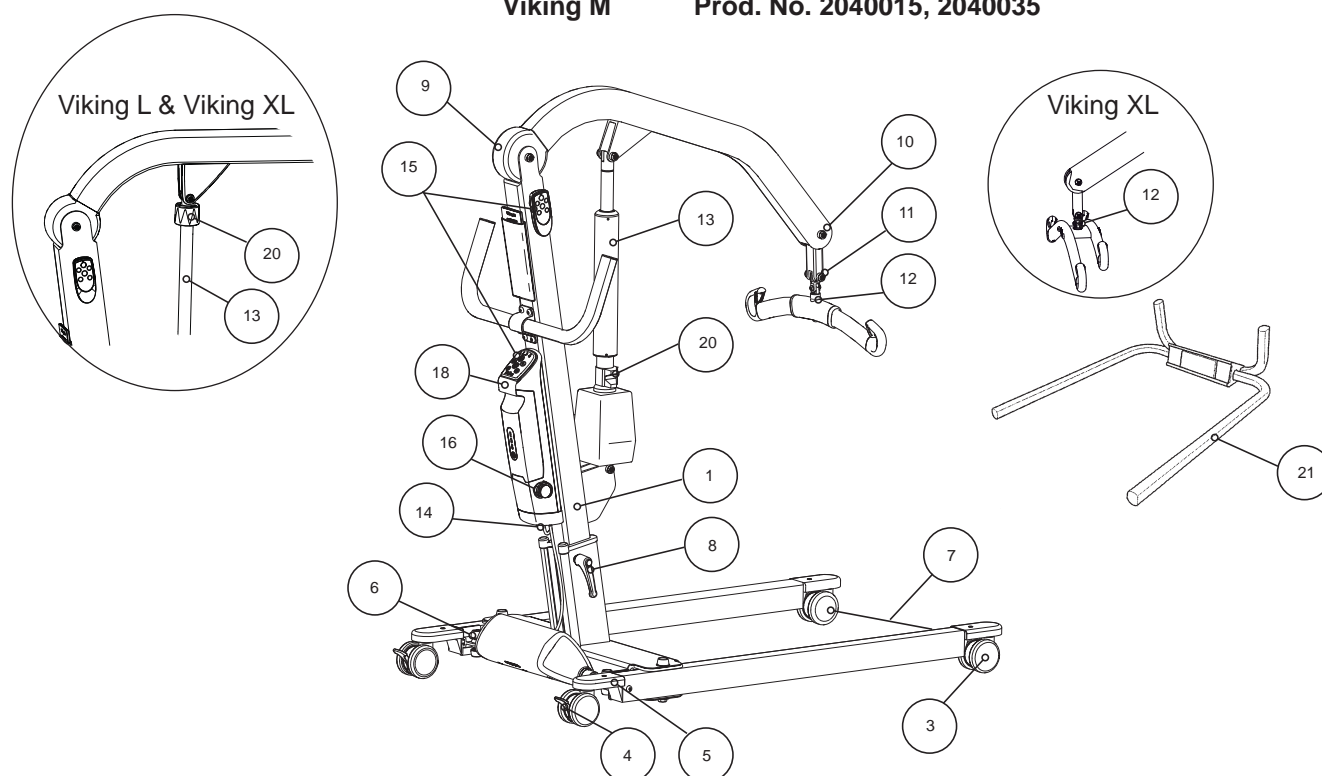



## Instructions for the check points

### 1 Product decal

- Verify presence of mast decal with model type and serial number.

### 2 Visible damage to the lift surface finish

- Check for scratches, dents, deformities or unusual surface wear.

### 3 Castor wheels, forks and fork covers

- Roll the lift, unloaded and with load, along the floor. Check to ensure that all castors roll and turn freely.
- Verify that the castor fasteners are tight. There should not be any play between the fork and the castor nut.
- Remove the front fork cover and inspect safety casing and bolt channel for cracks or fractures.

### 4 Castor brakes

- Lock the brakes. Make sure the wheels do not turn and the castor housing does not swivel when the lift is pushed.

### 5 Base covers

- Verify that the protective plastic covers are secure.

### 6 Base opening and closing linkage

- Inspect the linkage between legs and base motor. Make sure there is no cracks or deformations at the legs.
- Inspect the plastic protective cover shell for cracks, deformation or gaps.
- Open and close the base to maximum and minimum dimensions.
- Listen for particular noises and vibrations.

### 7 Base frame castor play and levelling

- Manually open and close legs at the front castors to measure wheel play. Base width wheel play should not exceed 35mm / 1,38 Inch total.

### 8 Locking handles


- Loosen & re tighten locking handles. Verify that threads are not damaged or binding.

### 9 Lift arm hardware and covers

- With the lift in it's lowest position, manually raise the lift arm. Verify that the shaft lowers and raises freely. If the arm is binding, the mounting screw is too tight.
- Inspect the plastic protective cover shells for cracks, deformation or gaps.
- Make sure the holes in the mast are not deformed on:

Product	Product No.	Up to S/N:
Viking XL	2040003	801 999
Viking L	2040004	7 200 795
Viking M	2040005	7 106 200

Verify that the diameter of the holes in the mast (where the arm and mast is connected) is not bigger than 12 mm/0,47 inch, It can be done by measuring the distance from cover hat, to top of the "ear". The length should be the same on each side ( $X^1=X^2$ )±1 mm.

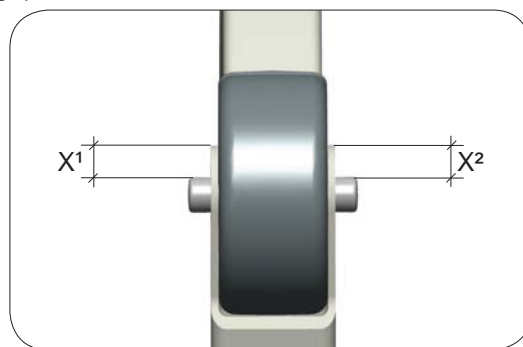

### 10 Flexlink to arm hardware and covers

- With the slingbar hanging in it's lowest position, manually raise the slingbar and flexlink arm.
- Verify that the flexlink arm lowers and raises freely. If the flexlink is binding the screw is too tight.
- Inspect the plastic protective cover shells for cracks, deformation or gaps.


### 11 Flexlink to slingbar hardware and covers

- With the slingbar hanging in it's lowest position, manually pivot the slingbar side-to-side. Verify that the flexlink arm lowers and raises freely. If the flexlink is binding, the mounting screw is too tight.
- Verify that hardware covering plastic caps are secure.
- If no accessories is attached, verify that the slingbar is assembled directly to the flexlink.

## Instructions for the check points

### 12 Slingbar

- Inspect visually to ensure that the slingbar is free from any scratches, sharp edges or deformities.
- Check that the unit rotates freely on its bearings.
- Make sure the lower slingbolt has a mark, as the picture shows. If not, the slingbar must be replaced.
- Make sure that both latches are mounted and fall back against the body of the slingbar.

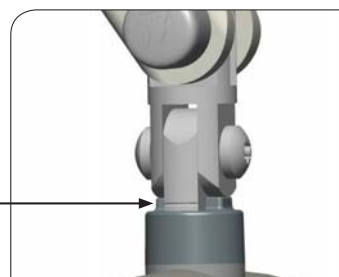

### 13 Mast actuator

- Verify that the upper and lower actuator fasteners to the mast are tight.
- Visual inspection of tubes, back fixture and piston rod eye.
- Check of abnormal noise.
- If the lift is equipped with Outer tube: Pull the Outer Tube to the top and bottom (with the lift arm in its highest position). Make sure the Outer Tube not get stuck in any position.
- Check of electric end stop

Prod. No. 2040003-2040005: By using test box 21090014 UK or 21090015 US

Prod. No. 2040013-2040015 and 2040033-2040035: Run the lift all the way up and down. Verify that the Indicator lamp for overload (orange light) not lights.

### 14 Battery and electrical cable installations

- Verify that all cables are properly inserted. Re-insert if uncertain.
  - Make sure the charge cable hook is mounted at the mast.
- Prod. No. 2040003-2040005: Verify battery charge (recharge battery or test with Prod. No. 2101100 Battery tester with diodes).
- Prod. No. 2040013-2040015 and 2040033-2040035: Inspect that the cable cover is mounted correctly. Verify battery charge. During charging the lights shall shine alternatively (orange, orange, green). Completed charging is indicated by a flashing green light.

### 15 Hand control

- With the emergency stop out, verify that all functions on the hand control are working.
- Prod. No. 2040013-2040015 and 2040033-2040035: With the emergency stop out, verify that all functions on the operation panel are working.

### 16 Emergency Stop function

- Press the emergency stop button. With the emergency stop pressed in, verify the lift does not operate with the hand control buttons.
- Turn the red emergency button in the direction of the arrows. Verify the button releases from the locked position into the raised, open position.

### 17 Emergency lowering device (only on Prod. No. 2040003-2040005)

- With the emergency stop out, verify that the electrical emergency-lowering device functions.

### 18 Control unit CBL 20 ( Prod. No. 2040013-2040015 and and 2040033-2040035)

- With the emergency stop out, verify the function of the electrical emergency-lowering device at the operation panel.

### 19 Max. load test

- Run the lift with maximum load, all the way up and down.

Max load	kg	lbs
Viking M	205	450
Viking L	250	550

## Instructions for the check points

<b>Viking XL</b>	300	660
------------------	-----	-----

- Run the base in and out with maximum load.
- Make sure all four wheels have contact with the floor.
- Listen for peculiar noises and vibrations.

### 20 Mechanical emergency lowering load test

- Raise with maximum load (see table above) and with (50 kg/110 lbs) until the arm is parallel to the ground.  
Prod. No. 2040003-2040005, 2040015, 2040035: Lift red emergency handle on the actuator upwards and hold, verify that the load lowers.
- Prod. No. 2040013, 2040014, 2040033, 2040034: Manually turn the red actuator collar in the direction of the arrows to lower.

### 21 Armrest (accessory)

- Verify that both arms rotate freely and hang vertically when not in use.
- Rotate arms to raised position. Perform a load test of the armrest with (40 kg/88 lbs).

### 22 Documentation

- Verify that the Instruction guide is provided.

### 23 Clear Service (CBL 20)

- Only products with Control Unit CBL20 (Prod No 2040013 - 2040015 and 2040033-2040035)



**Service indicator** (orange light)  
- Illuminates when service is needed.

When the Periodic Inspection has been performed, reset the service indicator with LSS, LDS or Manual Service according to instructions "Reset Service Interval Parameters" in the Service Manual.



## Preventative Maintenance



A Hill-Rom Company

	Viking XL	Viking L	Viking M
Prod. No.	2040003	2040004	2040005
	2040013*	2040014*	2040015*
	2040033*	2040034*	2040035*

\* Lift equipped with Control Unit CBL

Lift type: .....

Product No: .....

Serial No: .....

Prod. Year: .....

Use Liko Service System (LSS) or Liko Diagnostic System (LDS) to read data from lifts equipped with Control Unit CBL.  
If you don't have the LSS, write down the values in the columns below. (With LSS you can store data)

## Customer references

Agreement No: .....

Name: .....

Address: .....

Post code: .....

TAAxT	Ah
PAAxT	Ah
AAS	Starts
ASERV	Times
IMED	mA
ALA	Charges
AALA	Charges
AOLA	Charges

AABT	Removals
AEUP	EposUp
AFSERV	Services
AEDWN	EposDn
IDNR	
AOLUP	OverLoads
PADT	Hours
PAOLUP	OverLoads

To ensure maximum life cycle, Preventative Maintenance should be carried out according to how much the lift has been used TAAxT (Totally Accumulated Amphours) or according to years in use, see table below. TAAxT can only be read from lifts equipped with Control Unit CBL. Inspection and service must be carried out by Hill-Rom/Liko authorized personnel.

## Check points:

## Maintenance according to used TAAxT (Ah):

Viking XL: (with Control Unit CBL)

Viking L: (with Control Unit CBL)

Viking M: (with Control Unit CBL)

## Maintenance according to years of use:


Maintenance according to use:											
	Viking XL: (with Control Unit CBL)	204	408	612	816	1020					
	Viking L: (with Control Unit CBL)	182	364	546	728	910					
	Viking M: (with Control Unit CBL)	140	280	420	560	700					
Maintenance according to years of use:		1	2	3	4	5	6	7	8	9	10
1	Surface treatment.....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
2	Front castors.....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
3	Rear castors / brakes .....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
4	Pivot bolts .....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
5	Base opening and closing linkage .....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
6	Base motor linkage.....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
7	Lift arm.....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
8	Flexlink / Lift arm joint / Slingbar bolt.....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
9	Battery. ....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
10	Actuator .....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
11	Control Unit CBL.....	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

**Lubricate with PTFE or silicon grease**

When maintenance is performed; mark specified square with "X"  
When part is replaced; mark specified square with "R"


Interval/Month: \_\_\_\_\_ Date: \_\_\_\_\_ Serviced by: \_\_\_\_\_

## Check points

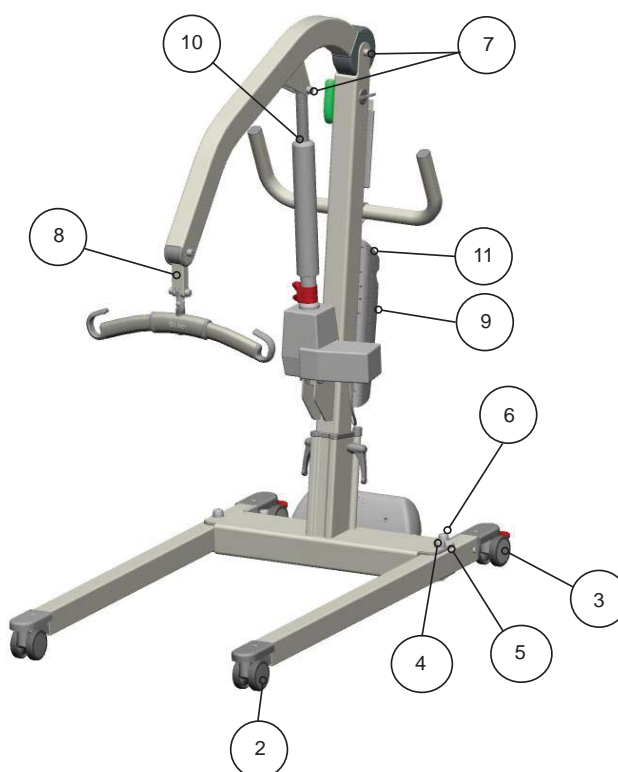

### Instructions regarding the check points

#### 1 Surface treatment

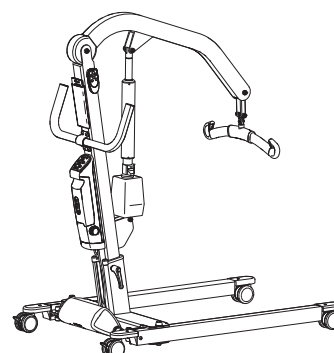



Visually inspect the lift for external damage or wear.

If needed, clean the patient lift with warm water or ethanol.

**Warning! Do not use chemicals containing phenol or chlorine, since the aluminium parts may be damaged.**

Instructions regarding the check points


2 Front castors

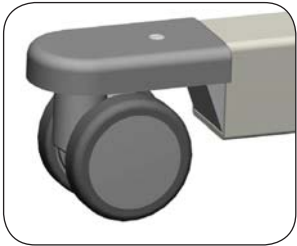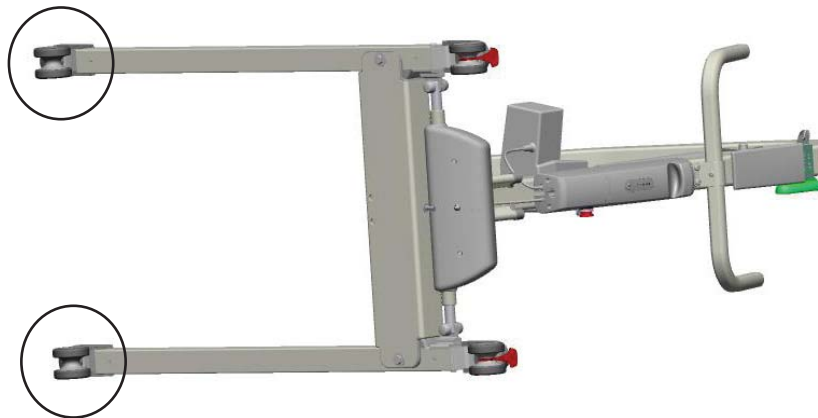

Check and clean front castors. If wheels are in poor condition, replace them. (For Spare Part No, see the Spare Parts List)

Front castors needs to be replaced after:	Viking XL 1020 Ah	Viking L 910 Ah	Viking M 700 Ah
or Replace according to years in use:	10	10	10


3 Rear castors / brakes

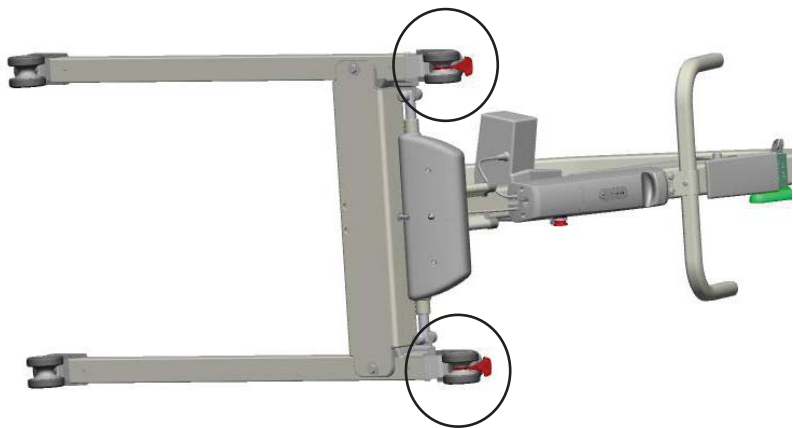

Check and clean rear castors. If wheels are in poor condition, replace them.  
(For Spare Part No, see the Spare Parts List)  
Check the brakes. They should lock in all directions.

Instructions regarding the check points

4 Pivot bolts

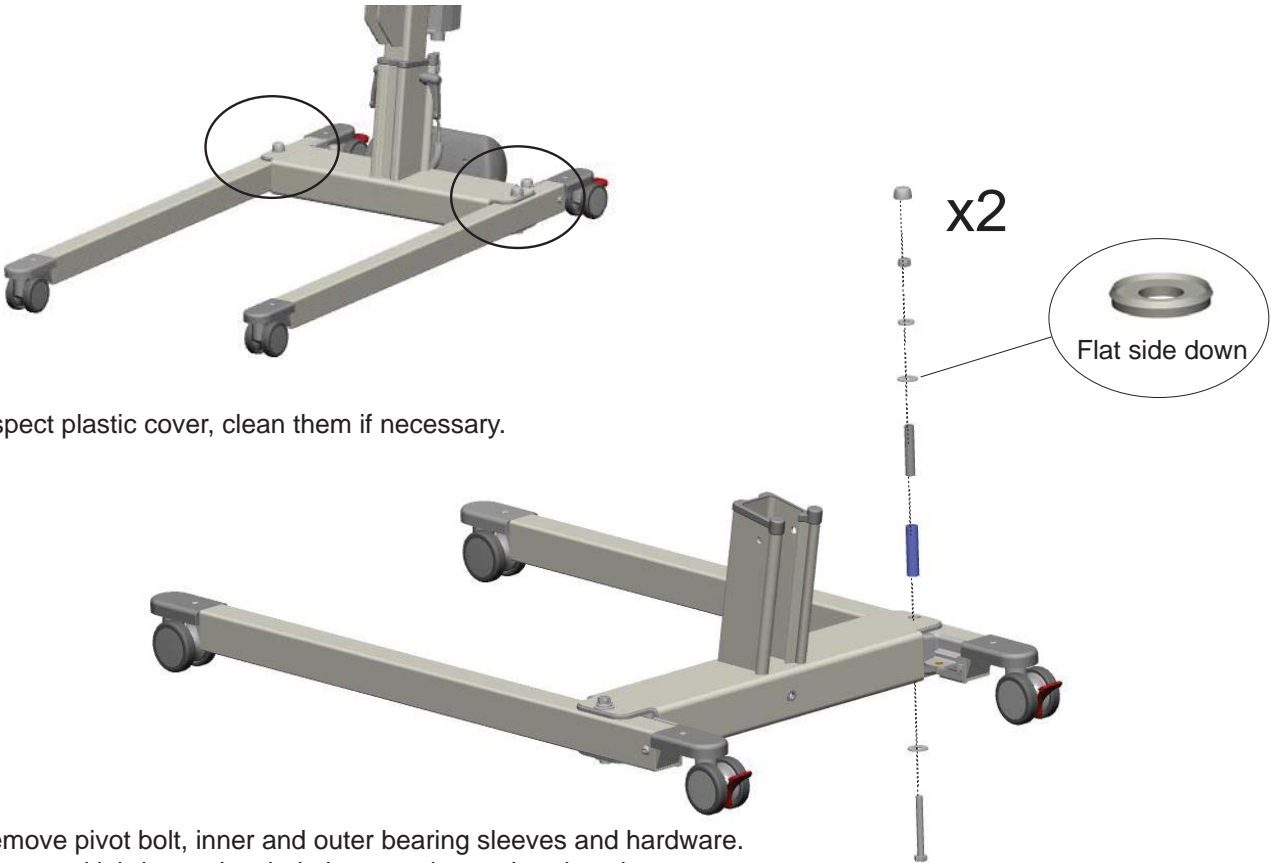

Inspect plastic cover, clean them if necessary.

Remove pivot bolt, inner and outer bearing sleeves and hardware.  
Clean and lubricate pivot bolt, inner and outer bearing sleeves.

5 Base opening and closing linkage

Remove hardware covers from middle beam and actuator.

Remove leg from middle beam and actuator.

- (a) Remove pivot bolt, inner and outer bearing sleeves and hardware.
- (b) Remove link arm cover, bolt and nut.


Remove plastic covers and inspect middle beam edges for burrs. File smooth if required.

Clean and inspect the three pieces of plastic cover. Replace if cracked or deformed. (For Spare Part No see the Spare Parts List)

Clean, lubricate and reassemble pivot bolt, bearing sleeves and hardware.

Tighten pivot bolt and apply plastic hardware covers.

Reconnect actuator to link arm. Apply plastic hardware covers.

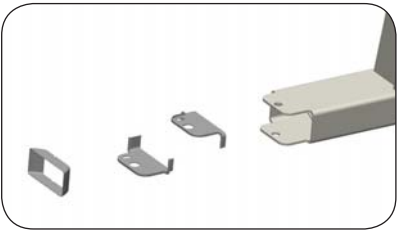


<b>Leg covers needs to be replaced after:</b>	<b>Viking XL</b>	<b>Viking L</b>	<b>Viking M</b>
<b>or</b>	<b>612 Ah</b>	<b>546 Ah</b>	<b>420 Ah</b>
<b>Replace according to years in use:</b>	<b>6</b>	<b>6</b>	<b>6</b>



Instructions regarding the check points

6 Base motor linkage

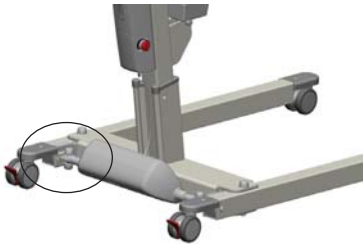


Remove link arm cover, bolt and nut.


Clean, lubricate and reassemble pivot bolt, bearing sleeves and hardware.

Inspect the legs at the linkage. If the linkage are in poor condition, replace them with Restoration set motor attachment.  
(For Spare Part No, see the Spare Parts List)

Reconnect actuator to link arm. Apply plastic hardware covers.

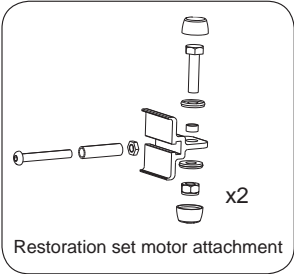


7 Lift arm

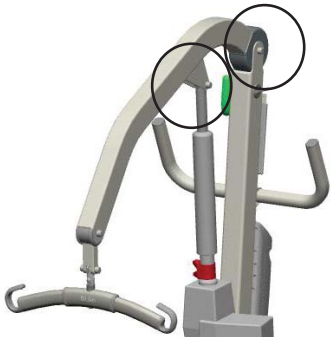


Remove hardware covers.

Remove and lubricate lift arm bolt.  
Visually inspect lift arm, plastic covers for wear, cracks or deformation.  
Reassemble in reverse order.

Raise the arm and then release, so that it falls to the lowest position.  
If the arm does not fall to the lowest position, the screw is too tight.

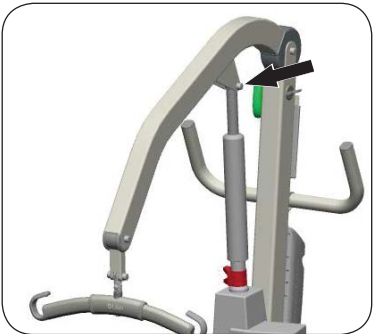


Remove and lubricate actuator hardware. Visually inspect actuator plastic bushings for wear. Replace if worn. Reassemble in reverse order.

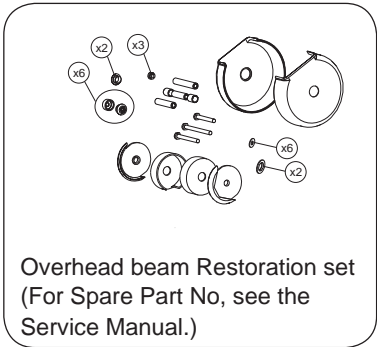

Overhead beam Rest. set needs to be replaced after: or Replace according to years in use:	Viking XL 1020 Ah	Viking L 910 Ah	Viking M 700 Ah
	10	10	10


Instructions regarding the check points

8 Flexlink / Lift arm joint / Slingbar bolt

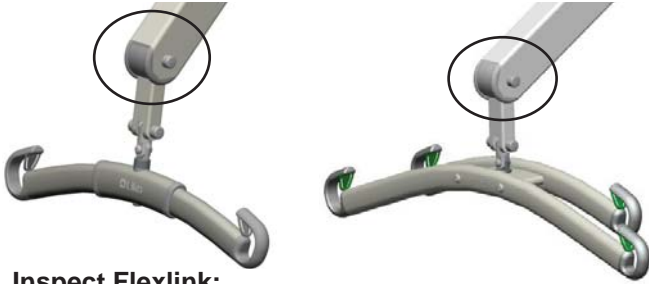


**Inspect Flexlink:**

1. Remove covers.
2. Remove, clean and lubricate flexlink. Inspect shell covers for cracks or gaps.
3. Pull out the Flexlink and check for wear. Re-assemble in reverse order.
4. Lift up the slingbar and then release, so that it falls to the lowest position. If the link does not fall to the lowest position, the screw is too tight.

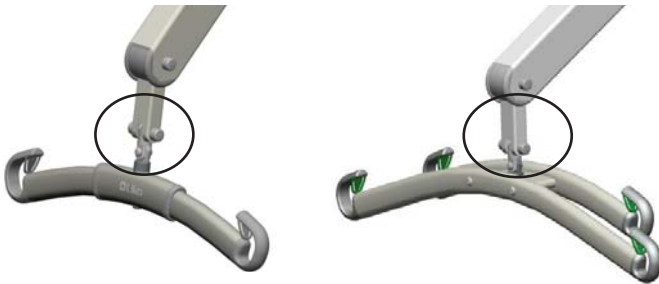

**Inspect Slingbar bolt:**  
(Universal SlingBar and Universal TwinBar)

1. Check slingbar bolt for wear.
2. Remove slingbar centerbolt and bearing, wipe of any dirty, dust or residue. (Universal Twinbar)
3. Reassemble in reverse order pivot slingbar centerbolt within the Flexlink. If the slingbar is binding, the screw is too tight.

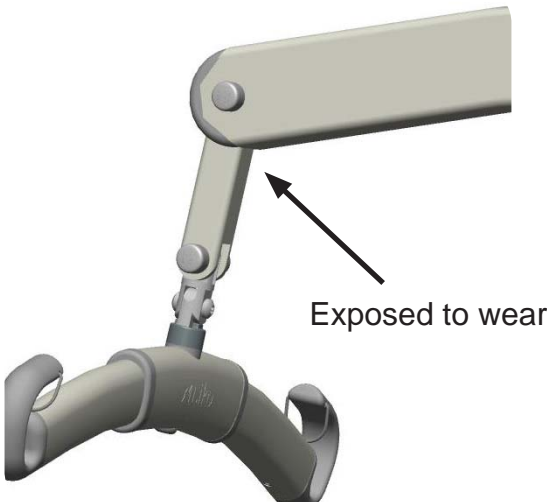


**Flexlink needs to be replaced after:**  
**or**  
**Replace according to years in use:**

Viking XL 1020 Ah	Viking L 910 Ah	Viking M 700 Ah
10	10	10

## Instructions regarding the check points

### 9 a) Battery to Control UnitCBL

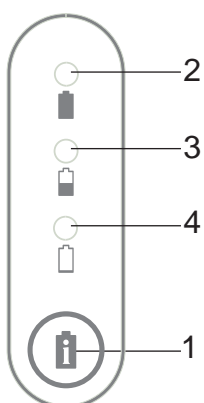

#### Battery capacity

Check battery capacity by pressing the i button (1).

The following information is displayed:

**2 – Green light:** More than 50% of battery capacity remaining.

**3 – Yellow light:** 25% -50% of battery capacity remaining.

**4 – Yellow light:** 0% -25% of battery capacity remaining.


The indicator lamps light for about 30 seconds before switching off.

Battery capacity is also displayed during lift operation.



Maximum charge is reached after about 5 hours. When the batteries are fully charged, the charger switches automatically to maintenance charging.

The lamps blink alternately (yellow/yellow/green) during charging. A blinking green lamp indicates that charging is complete. When operating time is noticeably shorter than normal, the batteries should probably be replaced. Discontinue charging and replace the batteries with 2006107 Extra battery box.

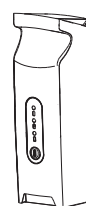


Prod. No.  
2006107 Extra battery box

<b>Batteries needs to be replaced after:</b>	<b>Viking XL</b>	<b>Viking L</b>	<b>Viking M</b>
	<b>408 Ah</b>	<b>364 Ah</b>	<b>280 Ah</b>
<b>and:</b>	<b>816 Ah</b>	<b>728 Ah</b>	<b>560 Ah</b>
<b>or</b>			
<b>Replace according to years in use:</b>	<b>4 and 8</b>	<b>4 and 8</b>	<b>4 and 8</b>

### 9 b) Battery to "Jumbo" Control Box

1. Remove the charger cable holder.
2. Remove the battery.
3. Inspect V-shaped metal contacts on the batterypack casing for uniformity. Check the battery with the battery tester Part. No. 2101100. A signal should be audible for more than 5 sec. If not, replace battery with spare part 2006106.
4. Inspect V-shaped metal contacts on control box for uniformity. Verify that plastic housing for metal contacts is not damaged/deformed. Tighten screws.
5. Place the removable battery pack into position on top of the control box. Adjust until you hear a clicking sound.

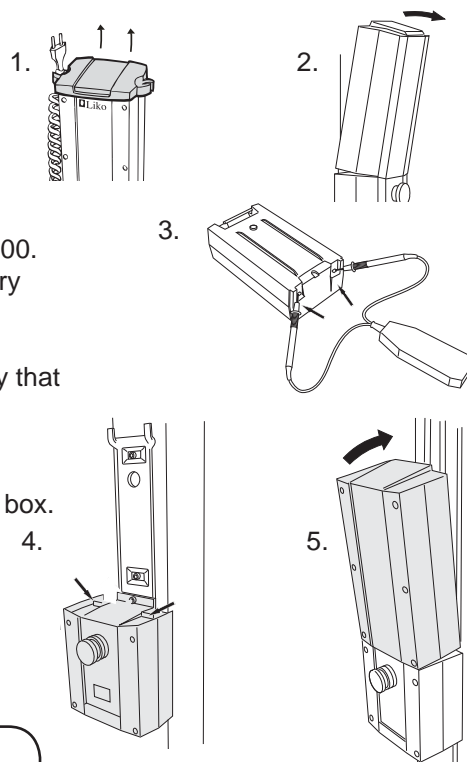

**Replace according to years in use: Every 3rd year**

## Instructions regarding the check points

### 10 Actuator

1. Visually inspect the actuator and the Outer tube (Outer Tube only on Viking M) for wear, cracks or deformation.
2. Replace the actuator after 700 Ah. (For Spare Part No, see the Service Manual.)




	Viking XL	Viking L	Viking M
<b>Actuator needs to be replaced after:</b>	<b>1020 Ah</b>	<b>910 Ah</b>	<b>700 Ah</b>
<b>or</b>			
<b>Replace according to years in use:</b>	<b>10</b>	<b>10</b>	<b>10</b>

### 11 Control Unit CBL

Use LSS or LDS to check that the correct current limit is programmed in CBL20:

	CLIM_1 Power limit for lifting motion (up) (actuator)	CLIM_2 Power limit for widening motion (base motor)
<b>Viking M</b>	8,5 [A] (CBL version 0-3: 9,2 [A])	6 [A]
<b>Viking L</b>	6,4 [A]	6 [A]
<b>Viking XL</b>	8,0 [A]	6 [A]




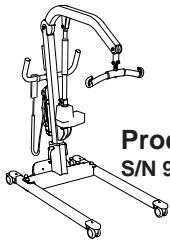

**Prod. No. 2040035**  
 S/N 9 200 001 - 9 300 000



**Prod. No. 2040015**  
 S/N 7 500 401 - 7 600 500

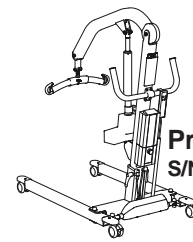

**Prod. No. 2040005**  
 S/N 7 100 101 - 7 200 200

Changes	Model 2040035 S/N	Model 2040015 S/N	Model 2040005 S/N	Project
---------	-------------------------	-------------------------	-------------------------	---------

## 2012

New model of Viking M with fixed handcontrol is introduced. Prod. No 2040035	9 200 001	-	-	1707
---	-----------	---	---	------

## 2010

Covers to motor brackets introduced.	7 505 000	-	791
Spring in Outer Tube altered. New screws to fasten the handle. Diagonal profile has been shortened.	7 503 472	-	791

## 2009

OuterTube also introduced on Prod. No. 2040005.	-	7 108 587	974
---	---	-----------	-----

## 2008

4 mm shorter middle beam. Link attachment for base motor adjusted.	7 502 500	-	671
Viking M prod. no. 2040015 with new control box and IR hand control.	7 500 401	-	556

## 2007

New linkage between legs and motor.	7 107 500	633
New washers (PS 6x12x1) arm-mast	7 107 064	EFR 758
Seal tape at Sling bolt, introduced on Slingbars from s/n 1231718.	-	493
New washers arm-mast PS12x18x1	7 106 528	EFR 694
Holder for Quick Reference Guide, and tape has been added. New washers (TBRB 6,4x12x1,6) in the pivot point on the mast.	7 106 201	519
Old latch back.	7 105 310	619
New Composite Latch.	7 105 034	467



Changes	Model 2040005 S/N	Project
---------	-------------------------	---------

**2006**

Washer 8,4/20x2,0 has been added to base-width adjustment.	7104347	
New pivot points between arm-mast, actuator-arm, mast-actuator, arm- link and link-slingbar (M6 screw and nuts).	7 103 094	370

**2005**

New Locking handles.	7 102 079	
New wheels and forks 75mm. Emergency guard for control box.	7 102 064	351
Base motor moved 3 mm back to avoid touching the middle beam.	7 101 100	213
New current limit in control box.	7 101 023	

**2004**

New Slingbar "Universal Slingbar 450".	7 100 862	226
Viking M delivered with the old Slingbar Standard 450.	7 100 382	
Width of base increased with 40 mm. Viking M is from now on equipped with a new Adapter.	7 100 308	





A  Company

[www.liko.com](http://www.liko.com)

---

**Manufacturer:**

Liko AB  
SE-975 92 Luleå  
Sweden  
[info@liko.se](mailto:info@liko.se)